# CEREC Ortho SW 1.2

## Update Description (EN)

### Installation notes:

- Perform a backup of all patient data before starting the installation process.
- Do not perform an installation on several CEREC units or computers at the same time if these are within the same network.
- Before starting the installation process, please check the patient database directory in the software (Configuration/Settings/Patient Database). The path should always point to a directory, never to the root of a drive, i. e. never to "D:\", but rather to "D:\Data".

### Changes:

- Tools for standard model space analysis (Bolton, Nance, Moyers) are available for optional use within the MODEL Phase. Analysis results can be exported as a standard PDF file.
- During model analysis, models are generically recolored, showing teeth as standard white and gingiva as pink. This allows a less distracting and more pleasing presentation to the patient.
- New EXPORT Phase
  - Includes a new dedicated export button for Dentsply Sirona Ideal Smile Aligner (outside the U.S.)
  - Has been restructured to make export options easier to find, grouping aligner manufacturers, other appliance manufacturers, and file exports into separate groups. The last used options are always available at the top of the screen for easy access.
- The Set Model Axis step now shows the maxilla by default, and the center line has been lengthened to facilitate orientation of the median palatine suture.
- The software can be installed on other workstations to allow model space analysis to be performed while the CEREC unit is in use for scanning.

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