Sidexis 4

- Installationsvoraussetzungen V4.4
- Installation Requirements V4.4
- Conditions d' installation V4.4
- Requisitos de instalación V4.4
- Requisiti di installazione V4.4

www.dentsplysirona.com

Unbedingt vor der Installation lesen!
It is essential to read this document before installation!
Ce document doit impérative être lu avant l'installation!
¡Leer obligatoriamente antes de la instalación!
Leggere assolutamente prima dell'installazione!

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Dear customer,

We are delighted that you chose to place your trust in Dentsply Sirona's innovations and wish to congratulate you on the purchase of your new dental product.

Your product is marked with the adjacent symbol. This means that the instructions for use are made available in electronic form on our online portal.

You can easily download the Operating Instructions online from <http://www.dentsplysirona.com/ifu>. If you use a terminal device which functions with a QR app, you have the additional advantage of calling up the Operating Instructions anytime, anywhere. Simply scan the QR code and it will connect you directly to our online portal.

If you would prefer to have the instructions for use in paper format, please request them at <http://www.dentsplysirona.com/ifu>. Simply fill out the form online and we will be delighted to send you a printed copy, free of charge. The instructions for use will be received within the next 7 days. We would like to thank you in advance for your cooperation.

We hope that you thoroughly enjoy using your Dentsply Sirona products.

Your Dentsply Sirona Team

<http://www.dentsplysirona.com/ifu>

General requirements for the installation of Sidexis 4

- Availability of a secure Internet connection:
A secure connection to the Internet must exist during the installation.
- User account with administrator rights:
A Windows user with administrator rights must be logged in for the installation.
- In order to be able to run Sidexis 4, read, write, change and execute access rights for the \Sidexis and \PDATA directories are necessary.
- Update and patch management:
The latest patch version is required for each operating system.

Please keep your operating system and other Microsoft components up-to-date with the latest security updates, even after installation.

- Parallel operation of Sidexis 4 on multiple computers:
If other software packages are run in parallel on the same computers, their system requirements must be added.
 - Sidexis 4 requires an installed MS SQL Server. Sidexis 4 is compatible with MS SQL Server versions 2017, 2019 and 2022. The MS SQL Server 2017 Express is included and installed by the Sidexis 4 setup if a compatible version has not already been installed.
 - A LAN bandwidth of 1 GB/s is required to work efficiently on the network. Please note that parallel-running applications and services can reduce the bandwidth available for Sidexis 4 and thus impair the speed of data processing in the network. The latest patch version is required for each operating system.

Notes on virtualization:

Sidexis 4 is released for specific Windows operating systems. It is assumed that the released operating system runs in a stable and high-performance system environment. Whether this system environment is based on a virtual machine or a physical computer is the responsibility of the respective IT manager of the customer.

Services can be easily virtualized. When applications are virtualized, the virtualization of the graphics plays the most important role → The virtualization must provide smooth graphics performance when the maximum number of stations are running.

It must be noted that virtualization results in higher resource consumption. Therefore, setting only the minimum system requirements does not lead to a satisfactory operating behavior of applications. For this reason, the resources made available to the individual virtual machines must be limited; we advise against dynamic settings. Sidexis 4 checks the available memory, for example, so that transactions are canceled before the operating system can "dynamically" expand the available memory.

The performance thresholds of the virtualization used must be known to the operator and must not be exceeded in real operation.

Hardware requirements

To meet the different requirements depending on the size of the practice or clinic, the recommended hardware requirements for the server PC are differentiated according to the number of workstations. This allows a better working speed (performance) to be achieved depending on the size of the practice.

The hardware requirements for the workstations are independent of the size of the practice. Information on how to achieve better performance on workstations can be found in section "Tips for better system performance".

Server PC

	Minimum requirements	recommended for up to 20 workstations	recommended for up to 50 workstations	recommended for more than 50 workstations
Operating system	Windows 10 Pro (64-bit) ^{1,2} Windows 10 Enterprise (64-bit) ^{1,2} Windows 11 Pro (64-bit) ^{1,3} Windows 11 Enterprise (64-bit) ^{1,3} Windows Server 2016 ⁴ Windows Server 2019 ⁴ Windows Server 2022 ⁴	Windows Server 2016 ⁴ Windows Server 2019 ⁴ Windows Server 2022 ⁴	Windows Server 2016 ⁴ Windows Server 2019 ⁴ Windows Server 2022 ⁴	Windows Server 2016 ⁴ Windows Server 2019 ⁴ Windows Server 2022 ⁴
MS SQL server	MS SQL Server 2017 Express (included in the Sidexis 4 setup)	MS SQL Server 2017 Express (included in the Sidexis 4 setup) or higher version	MS SQL Server 2017, 2019 or 2022 (not included in the scope of delivery)	MS SQL Server 2017, 2019 or 2022 (not included in the scope of delivery)
CPU	2 GHz / 2-core processor	2.3 GHz / 4-core processor	2.3 GHz / 4-core processor	2.4 GHz / 12-core processor (e.g. XEON E5-2673v3)
RAM	4 GB	8 GB	16 GB	32 GB

	Minimum requirements	recommended for up to 20 workstations	recommended for up to 50 workstations	recommended for more than 50 workstations
Hard disk	> 1 TB	> 1 TB	> 1 TB free space	> 1 TB
	Sufficient disk space must be ensured even during operation.			
Graphics card	any	any	any	any
Network	1 Gbps Ethernet with DNS	1 Gbps Ethernet with DNS	1 Gbps Ethernet with DNS	5-10 Gbps Ethernet with DNS

- Using these systems for server operation is not recommended, as they are not designed by Microsoft for this purpose. Slower performance compared to a server operating system is to be expected.
- Tested with: Version 21H2
- Tested with: Version 21H2
- with Desktop display, Core is not supported.

Workstation PC

	Minimum for 2D stations	Minimum for 3D stations
Operating system¹	Windows 10 Pro (64-bit) ^{1,2} Windows 10 Enterprise (64-bit) ^{1,2} Windows 11 Pro (64-bit) ³ Windows 11 Enterprise (64-bit) ³	
CPU	2 GHz / 2-core processor	2.3 GHz / 4-core processor
RAM	4 GB	8 GB
Graphics card	512 MB, DirectX 9.0c compatible or Intel onboard graphics ⁴	2 GB, DirectX 10 compatible or Intel onboard graphics ⁴
Hard disk	At least 5 GB of free space	
Network	1 Gbps Ethernet with DNS	
Screen resolution	At least 1280x1024 pixels, recommended: 1920 x 1080 pixels	
Optical drive	DVD ROM; DVD RAM (for use with wrap & go)	

- For 64-bit operating systems, installation using Boot Camp (Apple IOS) is also approved.
- Tested with: Version 21H2
- Tested with: Version 21H2

4. To ensure that the interaction with the 3D rendered volume is reliably "judger-free", graphics cards are recommended with at least the following Passmark GPU benchmark values:
NVIDIA: Passmark > 1000
AMD: Passmark > 1500
Intel Onboard: Passmark > 540

The Passmark values can be found at www.videocardbenchmark.net if the name of the graphics card is known.

Tips for better system performance

- More working memory (RAM) improves the performance for most tasks. RAM should be increased especially if other applications are run simultaneously with Sidexis 4 in the same system environment.
- Use an SSD instead of a conventional hard disk
- A good graphics card with its own graphics memory is advantageous for working with 3D data. The Intel on-board graphics card does not have its own graphics memory and uses the RAM of the PC. A corresponding increase of the RAM is therefore recommended.

Compatibility

Sidexis 4 practice devices

All devices in the current imaging systems portfolio of Sirona / Dentsply Sirona are supported

Supported systems	Systems no longer supported
Intraoral devices	
Xios AE, Xios XG Supreme, Xios XG Select, Xios Plus (Ethernet Wallbox), Schick AE, Schick 33, Schick Elite	Xios, Xios Plus (USB), Sidexis IO sensors
XIOS Scan	
Extraoral devices	
Orthophos XG	Orthophos DS
Orthophos XG 3D	Pan Elite
Orthophos SL, Orthophos S, Orthophos E	
Axeos	
Galileos with Facescan	

- Please observe the installation requirements for the individual products.
- Sidexis 4 is only tested and released with the respective current PC software components. An update of all PC software components to the latest version is required (may require an update of the device software, and thus recalibration of the X-ray unit)

Cameras

Integration is possible for the cameras listed below:

Supported systems	Systems no longer supported
SiroCam F, AF, AF ⁺	SiroCam 3, C, digital

Plugins

- The necessary plugins must be installed for operation of the cameras.
- The installation requirements for the individual products must be met.

The video module (v2.0.39) can be found in the Sidexis 4 installation menu under "Additional components". Older versions (up to and including v2.0.37) are no longer supported.

Where necessary, newer versions are available for download from Siroforce (<https://siroforcemobile.dentsplysirona.com>) under menu item "Service information / software → Treatment centers → PC Software → Sidexis 4 Plugins → Video Plugin → Software Download".

Integration of Dentsply Sirona treatment centers:

Dental unit plugins are available for Sidexis 4. When connected to Dentsply Sirona treatment centers, they enable

- the connection of the Intraoral camera
- the additional display of the ApexLocator on a monitor*
- the automatic time synchronization*
- the patient-specific chair positions*
- the automatic head rest adjustment*

*Not available for all Dentsply Sirona treatment centers

The dental unit plugins (v1.3.8) can be found in the Sidexis 4 installation menu under "Additional components". Older versions (up to and including v1.3.6) are no longer supported.

Where necessary, newer versions are available for download from Siroforce (<https://siroforcemobile.dentsplysirona.com>) under menu item "Service information / software → Treatment centers → PC Software → Sidexis 4 Plugins → Dental Unit Plugins → Software Download".

Other devices

- VistaScan image plate scanners can be used via a plugin from Dürr Dental (Dürr Bridge).
- Third-party intraoral cameras can be connected to Sidexis 4 (version 4.1.3 or higher) via WDM (Windows Driver Model). To do this, install the video module from the Sidexis 4 setup.
- The Sidexis SyncServices of REINKE SOFTWARE ENGINEERING (RSWE) and Sidexis 4 are completely compatible as of version 4.1.3. This requires a current version of Sidexis SyncServices (available from RSWE).

Sidexis extensions that are not yet supported

The following Sidexis XG extensions are not yet supported by Sidexis 4 version 4.4:

- TSE version

Important note: Should the customer continue to need these functions, then Sidexis XG is the more suitable product for these customers.

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SIRONA Dental Systems GmbH