

Integrity® Multi•Cure

Temporary Crown & Bridge Material


Technique Guide



- 1.** Take matrix using VPS or Alginate impression material.



- 2.** Dispense Integrity® Multi•Cure material directly into the matrix. Place matrix into the patient's mouth within 45 seconds.

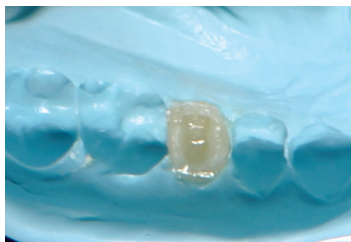


- 3.** Remove the provisional crown or bridge from the mouth after approximately 90 seconds.



- 3a.** Optional
Light cure each unit 20 seconds or place in a curing oven for 60 seconds.



- 4.** After light curing or 5 minutes from start of mix, adjust and trim the provisional with a slow speed acrylic bur and/or a high speed diamond bur. Finish and polish.

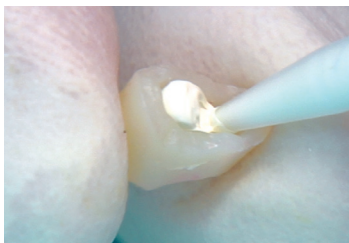

- 5.** Cementation: Apply a thin, uniform layer of Integrity® TempGrip® Cement to the entire internal surface of the Integrity® Multi•Cure material restoration.

Light-Cure Technique

0'45"
working time

0'45" - 1'30"
setting time in the mouth

1'30" - 2'15"
matrix removal from mouth

after mouth removal
light cure each unit 20 sec

after light cure
remove oxygen inhibition
layer; trim margins;
final finishing

Self-Cure Technique

0'45"
working time

0'45" - 1'30"
setting time in the mouth

1'30" - 2'15"
matrix removal from mouth

2'15" - 5'00"
final setting

from 5'00"
remove oxygen inhibition
layer; trim margins;
final finishing

Save 2-3 minutes!

Ordering Information

Integrity® Multi•Cure Temporary Crown & Bridge Material Cartridge Refills

1 Cartridge Material (76 g);
15 Mixing Tips
605.78.361 Shade A1
605.78.362 Shade A2
605.78.363 Shade A3
605.78.365 Shade A3.5



Integrity® TempGrip® Temporary Crown & Bridge Cement

666450 Syringe Refill Package
2 Syringes (9 g each);
20 Mixing Tips



For Dental Use Only



Caution, consult
instructions for use

www.dentsply.eu

©2017 Dentsply Sirona, Inc.
All rights reserved. (07/11/2017)
K107162-00