

+
WE
KNOW
ENDO.

MAILLEFER

TruNatomy™

True, Natural, Anatomy



Dentsply
Sirona

Instrumentation

Standard cases



ReadySteel
K-File
10 TruNatomy
Irrigation
Needle

Scout 2/3 of the canal
in the presence of a
lubricant.



TruNatomy
Orifice Modifier
20.08

Use gentle 2-5 mm
amplitude in-and-out
motions until the coronal
3rd exponent is shaped.



ReadySteel
K-File
10 TruNatomy
Irrigation
Needle

Scout with
lubricant to
estimated working
length and irrigate.

Agitate your irrigation solution using the
EndoActivator with a Small tip (15.02).

Motor settings: 500 rpm | 1.5 Ncm



TruNatomy
Glider
17.02v

TruNatomy
Irrigation
Needle

TruNatomy
Prime
26.04v

Optional



TruNatomy
Small
20.04v

TruNatomy
Medium
36.03v


Irrigation

Agitate your irrigation solution using the EndoActivator with a Small tip (15.02).

Advance passively to WL using gentle amplitudes. Do not brush.
Recapitulate and irrigate between each TruNatomy file.

Obturation options

Standard cases

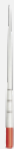

TruNatomy
Absorbent Point
Prime



TruNatomy
Conform Fit
Gutta-Percha
Prime



Maillefer Instruments Holding Sàrl
Chemin du Verger 3, CH-1338
Ballaigues, Switzerland
dentsplysirona.com

Files, gutta-percha, paper points

Optional



TruNatomy
Absorbent Point
Small & Medium



TruNatomy
Conform Fit
Gutta-Percha
Small & Medium



Produits Dentaires SA
Rue des Bosquets 18, CH-1800
Vevey, Switzerland

Irrigation Needle