



Update-Description

inLab CAD SW 22.1.1


This document describes changes and improvements compared to previous version inLab CAD 22.1.0

Date: September 2022

General

New features

Integration of PrimeTaper implants for CEREC Guide 3

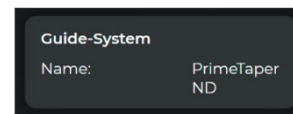

PrimeTaper Implant EV

Connectivity Connect Case Center URL China – Transfer of Primescan and Omnicam data via Connect Case Center to inLab Software in China



New materials

- KATANA Zr ONE
- CEREC Tessera Abutment (only design parameter adjustment) – Not USA yet
- Indication extension multilayer top for GC Initial LiSi and GC Initial LRF

Quality improvement and stability

Dentsply Sirona continuously improves the quality of its products. As part of this release, several improvements have been made and known software bugs have been fixed. We always recommend using the latest software version.

Bugfixes and Quality

- Random freezes during reconstruction have been fixed.
- Connect Case Center downloaded into CAD 22 the “DXD conversion to RST format failed” error appears was resolved.
- Unhandled exception after use of shape – anatomical – occlusal on gingiva has been fixed.
- Atlantis DXD file (core file) import sometimes contains artifacts which can lead to problems during the import into inLab and CEREC. That behavior has been fixed.
- UX improvement on patient details which now fit the teeth numbers for bridges.
- In some situations, cases cannot be saved nor exported. A critical situation has been found and fixed.
- Sometimes Reduce function cannot be used properly if “hide neighbors” checkbox is checked. This problem has been resolved.
- An unhandled error occurs when switching to “Click Scanbody Head” from “Set Jaw Line”. This behavior has been fixed.
- Crashes when loading multilayer implant bridge case with gingiva are prevented.
- Misleading info for connector details has been corrected.
- Preface abutment rotation for Ankylos has been fixed.
- Wrong display of the model in Check Model step on cases from Connect Case Center has been corrected.
- In special situation the software cuts away a tooth during virtual seating. This behavior has been fixed.
- System error when trying to Save As a case with Enter from “Patient” field has been corrected.

Dentsply Sirona

Sirona Dental Systems GmbH
Fabrikstraße 31, 64625 Bensheim, Germany
dentsplysirona.com

Procedural Solutions

Preventive
Restorative
Orthodontics
Endodontics
Implants
Prosthetics

Enabling Technologies

CAD/CAM
Imaging
Treatment Centers
Instruments