

CEREC SW 5.1.2

GRAPHICS CARD DRIVER (EN)

A new graphics card driver has been tested and approved for use with CEREC SW 5.1.2. This can optionally be installed as a replacement for the previous graphics card driver.

- To ensure optimal functionality of your CEREC SW 5.1.2, the use of a graphics card driver approved for this purpose is necessary. Please only use graphics card drivers that have been tested by Dentsply Sirona for use with CEREC SW and the installation application Dentsply Sirona offers for them.

Graphics card drivers validated for CEREC SW 5.1.2 are the versions:

- 18.7.1 (24.20.12019.1010)
- 19.5.2 (26.20.11015.5009)
- **20.4.2 (26.20.15029.27017) (new)**

- These three graphics card drivers have also been approved for use with:

- CEREC SW 5.0.0 - 5.1.2
- Connect SW 5.0.0 - 5.1.1
- Ortho SW 2.0.0 - 2.0.1

- To install the graphics card driver version 20.4.2, please download the installation application AMD-20.4.2-Win10-64bit-CleanSetup.exe from my.cerec.com.

- Installation time <10 min, 2 automated restarts

- When the software is started, it automatically checks whether your driver is correct. If you receive a message that this is not the case, please contact customer service.

- We recommend the following graphics card drivers for Omnicam users:

- **18.7.1** for Windows10
- **17.4.4** for Windows7