

Atlantis® suprastructures compatibility chart



Manufacturer/ Company	Implant system	Atlantis® suprastructures prosthetic screw/ ASA screw availability	Screw Order REF number from Manufacturer/ Company*	Torque Ncm	
BioHorizons	IMPLANT LEVEL				
	Internal/ Tapered 3.5, 4.5, 5.7	x	PXAS	30	
Biomet 3i	IMPLANT LEVEL				
	Certain™ 3.25, 4.0, 4/3, 5.0, 5/4, 6.0	x	IUNIHT	20	
	Certain™ Prevail™ 3/4/3 (3.4), 4/5/4, 5/6/5, 4/3, 5/4, 6/5	x	IUNIHT	20	
	Certain™ XP 4/5, 5/6	x	IUNIHT	20	
	ABUTMENT LEVEL				
	Low Profile Abutment	x	LPCTSH	10	
Camlog	IMPLANT LEVEL				
	Screw-line Implant 3.3	x	J4005.1601	20	
	Screw-line/Root-line Implant 3.8, 4.3	x	J4005.1601	20	
	Screw-line/Root-line Implant 5.0, 6.0	x	J4005.2001		
Dentsply Sirona	IMPLANT LEVEL				
	Astra Tech Implant System*				
	OsseoSpeed® EV 3.0 Green	x	25203	25	
	OsseoSpeed® EV 3.6 Purple	x	25204	25	
	OsseoSpeed® EV 4.2 Yellow	x	25205	25	
	OsseoSpeed® EV 4.8 Blue	x	25206	25	
	OsseoSpeed® EV 5.4 Brown	x	25207	25	
	OsseoSpeed® EV Profile 4.2	x	25205	25	
	OsseoSpeed® EV Profile 4.8	x	25206	25	
	OsseoSpeed®/OsseoSpeed® TX 3.0 - Yellow	x	24189	15	
	OsseoSpeed®/OsseoSpeed® TX 3.5, 4.0 - Aqua	x	24449	20	
	OsseoSpeed®/OsseoSpeed® TX 4.5, 5.0 - Lilac	x	24209	25	
	OsseoSpeed® TX Profile 4.5/5.0	x	24209	25	
	OsseoSpeed® 3.5 - Yellow (Previous)		24131		
	OsseoSpeed® 4.0 - Blue (Previous)	x	24209	25	
	Xive*				
	Xive® S 3.0	x	46-4304	24	
	Xive® S 3.4, 3.8, 4.5, 5.5	x	46-4305	24	
		ABUTMENT LEVEL			
	Astra Tech Implant System*				
UniAbutment EV	x	25481	15		
Angled Abutment EV	x (ASA not available)	25481	15		
20° UniAbutment	x	22435	15		
45° UniAbutment	x	22435	15		
Angled Abutment 20°	x (ASA not available)	22435	15		
Ankylos*					
Ankylos® Balance Base Narrow D4.2	x	3105 6022	10		
Ankylos® Balance Base D5.5	x	3106 6022	10		
Xive*					
Xive® TG 3.4, 3.8, 4.5	x	46-4300	24		
Xive® MP 3.4, 3.8, 4.5	x	46-4300	24		
Xive® MP 5.5	x	46-4305	24		
Keystone Dental	IMPLANT LEVEL				
	PrimaConnex SD 3.3,3.5	x	45060K	30	
	PrimaConnex RD 4.0,4.1	x	45060K	30	
	PrimaConnex WD 5.0	x	45060K	30	
Nobel Biocare	IMPLANT LEVEL				
	NobelActive NP 3.5	x	36917	35	
	NobelActive RP 4.3, 5.0	x	36918	35	
	NobelReplace NP 3.5	x	29474	35	
	NobelReplace RP 4.3	x	29475	35	
	NobelReplace WP 5.0	x	29475	35	
	NobelReplace 6.0	x	29475	35	
	ABUTMENT LEVEL				
	Multi-Unit Abutment RP	x	29285	15	

* Note: We recommend to check screw REF number and torque Ncm with the manufacturers, excluded Dentsply Sirona. Atlantis suprastructures assortment can vary from country to country. Please check Atlantis WebOrder for availability. All trademarks, company names and implant designs are the property of their respective owners.

Please turn over →

Manufacturer/ Company	Implant system	Atlantis® suprastructures prosthetic screw/ ASA screw availability	Screw Order REF number/ from Manufacturer/ Company*	Torque Ncm
Straumann	IMPLANT LEVEL			
	Bone Level 3.3 NC	x	025.2900	35
	Bone Level 4.1, 4.8 RC	x	025.4900	35
	Standard/Standard Plus 4.8 RN (Regular Neck)	x	048.356	35
	Standard/Standard Plus 4.8 WN (Wide Neck)	x	048.356	35
	ABUTMENT LEVEL			
	RN Abutment Level	x	048.350	15
	WN Abutment Level	x	048.350	15
	Bone Level Multi-Base Abutment D3.5, 4.5, 6.5	x	025.0900-04	15
	Bone Level Multi-Base Angled Abutment	x	025.0900-04	15
	Screw-Retained Abutment 3.5	x	023.4763	15
Screw-Retained Abutment 4.6	x	023.4763	15	
Zimmer Dental	IMPLANT LEVEL			
	Tapered Screw-Vent™ 3.5, 4.5, 5.7	x	MHLAS	30
	Screw Vent™ 3.3, 3.7, 4.5	x	MHLAS	30
	ABUTMENT LEVEL			
Tapered Abutment	x	SCTS	20	

* **Note:** We recommend to check screw REF number and torque Ncm with the manufacturers, excluded Dentsply Sirona. Atlantis suprastructures assortment can vary from country to country. Please check Atlantis WebOrder for availability. All trademarks, company names and implant designs are the property of their respective owners.

Turnaround times

- Atlantis® suprastructures are available in titanium and cobalt-chrome.
- Turnaround times vary depending on material.
- When the case is received at the Atlantis production facility and the manufacturing process has started, your Atlantis WebOrder account will be updated with the case estimated shipment date.

Standard turnaround times are as follows:

Turnaround times (in-house business days)		
	Titanium	Cobalt-chrome
Atlantis® Bridge	6	10
Atlantis® Hybrid	6	10
Atlantis® Bar	6	10
Atlantis® 2in1	10	N/A

Determining return arrival date

1. Select your choice of delivery method and calculate the number of days it will take to ship the case from the dental laboratory to Dentsply Sirona Implants (Waltham, MA).
2. Add on the days the case will be in-house at the Atlantis production facility. Calculation of in-house production time begins when the case is received at the Atlantis production facility before noon.
3. Dentsply Sirona Implants will ship the case via selected method in Atlantis WebOrder.

Note:

- Design approvals to process the case must be granted by 10:00 am your local time.
- Material, on-hold and case information issues must be resolved by 8:00 pm EST.
- Weekends and holidays are excluded.
- Time needed for **design approval** or resolving **on-hold information** may delay the case.

Example

- Order and all case materials for an Atlantis Bar in titanium is received on Monday before noon.
- Design for approval is sent out.
- If approval takes place the next day by 10 am (your local time), the case will be shipped to the dental laboratory by next Tuesday.

Additional information

The **Atlantis suprastructure prosthetic screw** is offered for all implant systems. One screw per implant site is provided with the Atlantis order. Next to the Implant selection in Atlantis WebOrder, it is possible to order up to (1) additional prosthetic screw per implant site.

Atlantis suprastructures are always compatible with the implant/abutment screw from the respective manufacturer's system. Additional screws can be ordered from the implant system manufacturer. As a service, all screws are listed with the manufacturers REF number.

Note: We recommend to check the REF number with the implant system manufacturer.