



Achieve Coronal Access

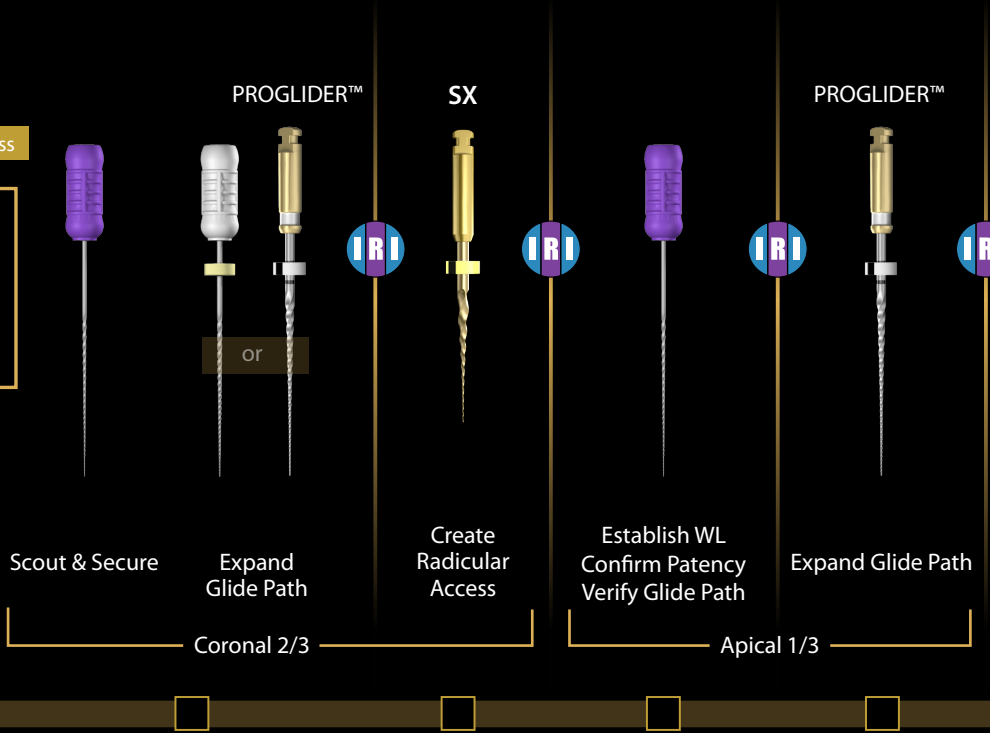

- Use ProLube® Root Canal Conditioner and small hand files when manually scouting and securing canals
- Use NaOCl when mechanically pre-shaping or shaping canals
- Use ProGlider™ Rotary Glide Path File & ProTaper Gold SX files in one or more passes to expand the glide path
- Use ProTaper Gold SX to relocate the coronal aspect of a canal away from an external root concavity, eliminate triangles of dentin, or produce more shape as desired
- Irrigate, recapitulate with a 10 file and re-irrigate after removing each rotary file
- Frequently clean and inspect the flutes of the rotary files

 300 RPM / Torque 400-520 gcm



View the technique
www.TulsaDentalSpecialties.com/Gold



TREATMENT TECHNIQUE

SHAPING & FINISHING SEQUENCE

Expand Final Shape
as Desired

F3 F4 F5



- Use ProTaper Gold files in regions of the canal that have a confirmed, smooth and reproducible glide path
- Use ProTaper Gold files in a deliberate brushing motion, on the outstroke, to more optimally prepare canals that exhibit irregular cross-sections and to facilitate apical file progression
- Use any ProTaper Gold file in one or more passes to safely achieve length


View the technique

www.TulsaDentalSpecialties.com/Gold

Proceed to 3D Disinfection and Obturation




S1 S2

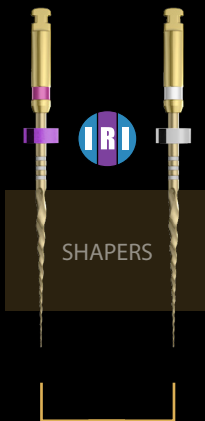

SHAPERS

Brush, Follow & Shape
to Length

F1 F2

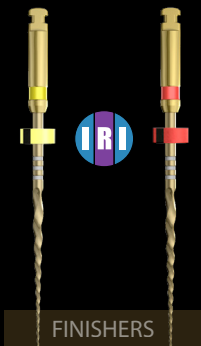

FINISHERS

Follow, Brush & Finish
to Length

Inspect Apical Flutes for Dentin to Verify Shape



300 RPM / Torque 400-520 gcm



STERILIZE
BEFORE USE



SINGLE USE ONLY



CONSULT
INSTRUCTIONS
FOR USE

Rx Only
For Dental Use Only

© 2014 DENTSPLY International, Inc. TCPTG Rev. 1 5/14

Manufactured By: **DENTSPLY Tulsa Dental Specialties**
DENTSPLY International, Inc.
608 Rolling Hills Drive
Johnson City, TN 37604
1-800-662-1202
1-800-597-2779 (fax)
www.TulsaDentalSpecialties.com
www.TulsaDentalSpecialties.com/patents

DENTSPLY
TULSA DENTAL
SPECIALTIES