

Conform Fit™ Gutta-Percha Points



FOR DENTAL USE ONLY

DIRECTIONS FOR USE **REF. WGGPC, PTNGP, PTGGP AND PTUGP**

0) COMPOSITION

Components:

- Zinc Oxide
- Titanium dioxide
- Synthetic gutta-percha
- Barium sulfate
- Coloring agents
- Inert fillers

1) INDICATIONS FOR USE

Products are only to be used in a clinical or hospital environment by qualified dentists.

Application field: Endodontic obturator. This product is acceptable for all patients requiring root canal therapy. Indicated to be the predominate material used to obturate or fill the cleaned, shaped and irrigated root canal space during root canal therapy.

2) CONTRAINDICATIONS

None known.

3) WARNINGS

None known.

4) PRECAUTIONS

- For Dental Use Only.
- Not to be sterilized with heat.
- Keep away from sunlight and heat.
- Maintain under room temperature (< 30°C (86°F)).

5) ADVERSE REACTIONS






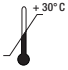
In the present technical state, no adverse reaction has been reported so far.

6) STEP-BY-STEP INSTRUCTIONS

The material should only be used by qualified dental professionals and those familiar with the techniques and procedures for use. After shaping and cleaning the root canal, obturation technique must be set by the professional. This procedure includes the use of Gutta-Percha cones and sealer. The main cone corresponds to the first cone to be introduced in the channel and should be selected in the same caliber of the last instrument used in the preparation to ensure the apical sealing. Other cones can be introduced into the channel depending on the technique used by the professional. A heated instrument can be used to soften the material and ensure perfect sealing and dimensional adaptation of the root canal.

7) DISINFECTION

Immerse the obturation devices in 5.25% NaOCl for 1 minute at ambient temperature. Do not use disinfecting solutions containing Phenol or any products which are not compatible with the treated filling material.

Symbols	EN
	Expiry date
	Single use only
	Nonreturnable if seal is broken
	Batch number
	Store in a cool, dry place
	Temperature Limitation
	Not made with natural rubber latex
	Rx Only
	For Dental use only

Manufacturer



DENTSPLY Tulsa Dental Specialties
 DENTSPLY International, Inc.
 608 Rolling Hills Drive
 Johnson City, TN 37604, USA
dentsplysirona.com

Made in USA