Vulcan[®] D-Series Burnout Furnaces

Digital Control

Owner & Operator's Manual

MODELS	D-130	D-550	D-1750	D-1750 EURO
100V	-----	9495132	-----	-----
100 – 120V	9495120	9495130	-----	-----
230V	9495121	9495131	9495140	9495141

Table of Contents

DESCRIPTION	PAGE
Safety	3
Technical Data	4
Installation Instructions	7
General Operation	8
Maintenance	12
Troubleshooting	13
Product Service	15
Warranty	16

FOR PROFESSIONAL USE ONLY.

Please read these operating instructions carefully before installing or operating this equipment.

- Use indoors only.
- Never operate the unit in close proximity to combustible materials or place materials on top of the unit.
- The unit must be electrically grounded to a three wire electrical outlet or receptacle. The electrical service provided must be a dedicated line of the proper size according to local electrical codes.
- Unit must be placed in a position that allows the power cord to be easily disconnected from the wall or inlet socket.
- Do not attempt to maintain the unit until you have read and understand this operation manual.
- Disconnect the line cord before attempting to service the unit.
- Do not operate the unit controls with tongs or other tools.
- Do not use solvents or liquid cleaners on the control panel.
- Do not cover the top of the unit.
- If the unit is not operated in the manner as specified in this manual, the protection provided by the unit may be impaired.

CAUTION: RISK OF DANGER!

BURN HAZARD IS PRESENT WHEN DOOR IS OPENED. USE CAUTION AFTER OPENING DOOR.
SURFACES MAY BE HOT!

OSHA AND CALIFORNIA PROPOSITION 65: MUFFLE DUST EXPOSURE

In keeping with the policy of DENTSPLY Ceramco to build safe products, comply with all National and State statutes and keep you, the valued customer informed; the services of a Certified Industrial Hygienist firm were employed to test and evaluate the lab operator's exposure to respirable refractory ceramic fiber (RCF) and cristobalite (a form of crystalline silica) present in the furnace muffle.

The findings of this test revealed that levels of exposure during the normal operation of this equipment, as outlined in the operator's manual, were far less than the Permissible Exposure Limit set by the Federal Government.

When it becomes necessary to replace the muffle, the person doing this work is recommended to wear a HEPA filter respirator and protective gloves as a precautionary matter.

Seal used muffle in a plastic bag and dispose of in accordance with local, state and Federal regulations.

Because this product and many similar products on the market today contain crystalline silica and ceramic fibers, it is necessary under the statutes of California Proposition 65 that DENTSPLY Ceramco includes the following statement:

"This product contains substance(s) known to the State of California to cause cancer."

Material Safety Data Sheets for RCF materials supplied upon request.

SYMBOL TABLE

	- Alternating current	Courant alternatif
	- On (Supply)	Marche (alimentation)
	- Off (Supply)	Arrêt (alimentation)
	- Caution, Hot Surface	Attention, surface chaude
	- Protective Conductor	Terminal Borne de masse, châssis
	- Caution, Risk of Danger	Attention risque de DANGER

TECHNICAL DATA

PARAMETER

Wide Operating Temperature Range: 25°C (77°F) – 1100°C (2012°F).

Programmable Heat Rate: 1 – 30°C/min (1°F – 54°F/min).

Programmable Hold Time: 0.00 – 60:00 hrs.

Muffle Temperature Uniformity: ± 8°C (± 15°F) at steady state.

ELECTRICAL 50-60 HZ:

MODEL	VOLTS	AMPS	WATTS	Watts to Maintain 1000°C (1832°F) (Air Exchange setting @ 0)
D-130 120V	100-120V	12A	1440W	525
D-130 230V	200-240V	6.25-7.5A	1250-1800W	525
D-550 100V	100V	14.7A	1470W	1050
D-550 120V	100-120V	16A	1920W	1050
D-550 230V	200-240V	7.7-9.2A	1540-2215W	1050
D-1750 230V	230V	19A	4370W	2200
D-1750 Euro	230V	19A	4370W	2200

DIMENSIONS:

Model	D – 130	D – 550	D – 1750
Length:	330mm (13.0")	430mm (17.0")	535mm (21.0")
Depth:	290mm (11.5")	400mm (15.0")	610mm (24.0")
Height:	350mm (14.0")	410mm (16.0")	545mm (21.5")

WEIGHT:

Model	D – 130	D – 550	D – 1750
Furnace	12kg (26lbs)	20kg (45lbs)	44kg (97lbs)
Shipping	15kg (32lbs)	25kg (54lbs)	60kg (132lbs)

OPERATING TEMPERATURE (Ambient):

5°C – 40°C (41°F – 104°F)

FUSE:

Model	D – 130	D – 550	D – 1750
Mains:	100-120V:12A 230V:2x 10A	120V:16A 230V:2x 10A	2x30A Breaker
PCB:	1 Amp	1 Amp	1 Amp


FEATURES:

- High Performance Hybrid Muffle.
- Longer Life, more Durable than Fiber Construction.
- Faster Heating and Cooling than Fiber.
- Digital Temperature Controller (programmable Temperature, Rate, Hold Time, Alarm, °C/°F operation).
- Integrated Door safety Switch.
- Durable Stainless Steel Front Panel.


TECHNICAL DATA (Contd.)

TEMPERATURE RISE AND COOL PERFORMANCE (No Load, max rate):

D-130 Heating and Cooling Curves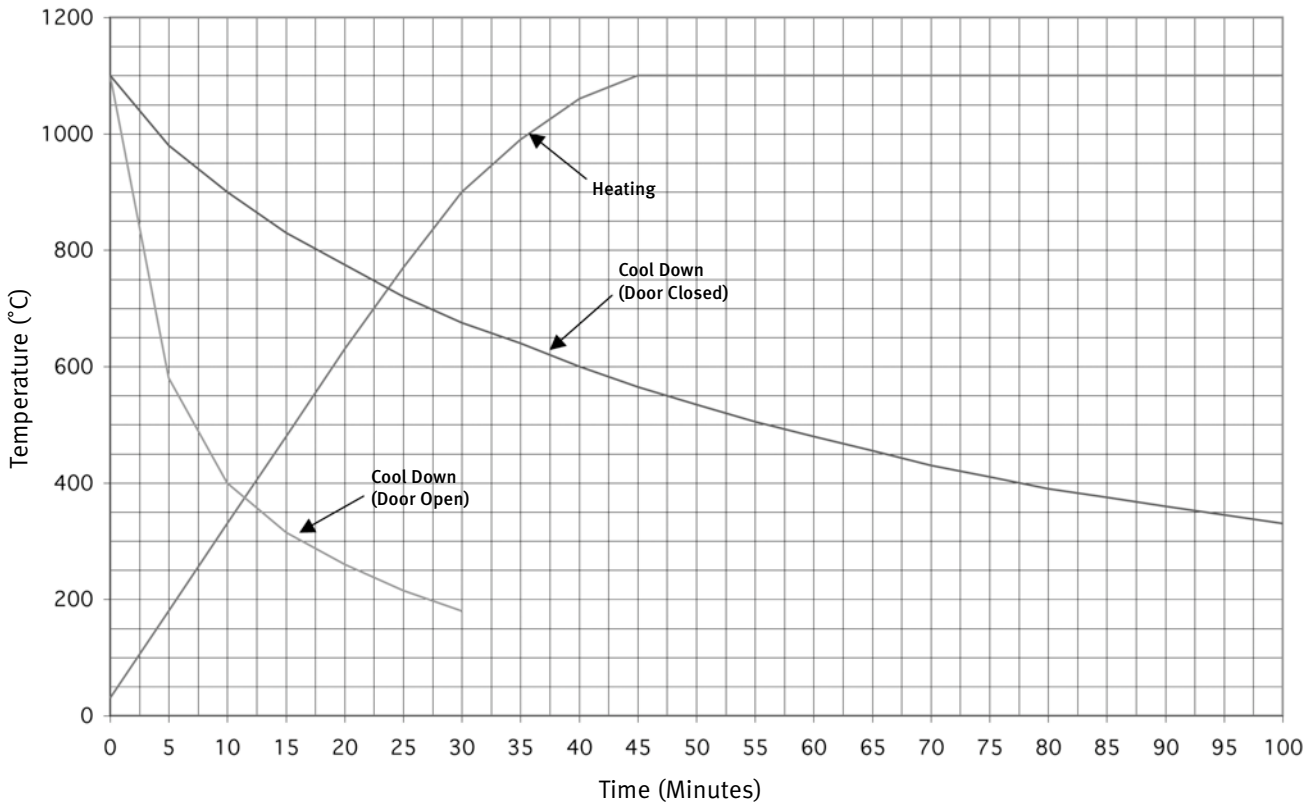


D-550 Heating and Cooling Curves

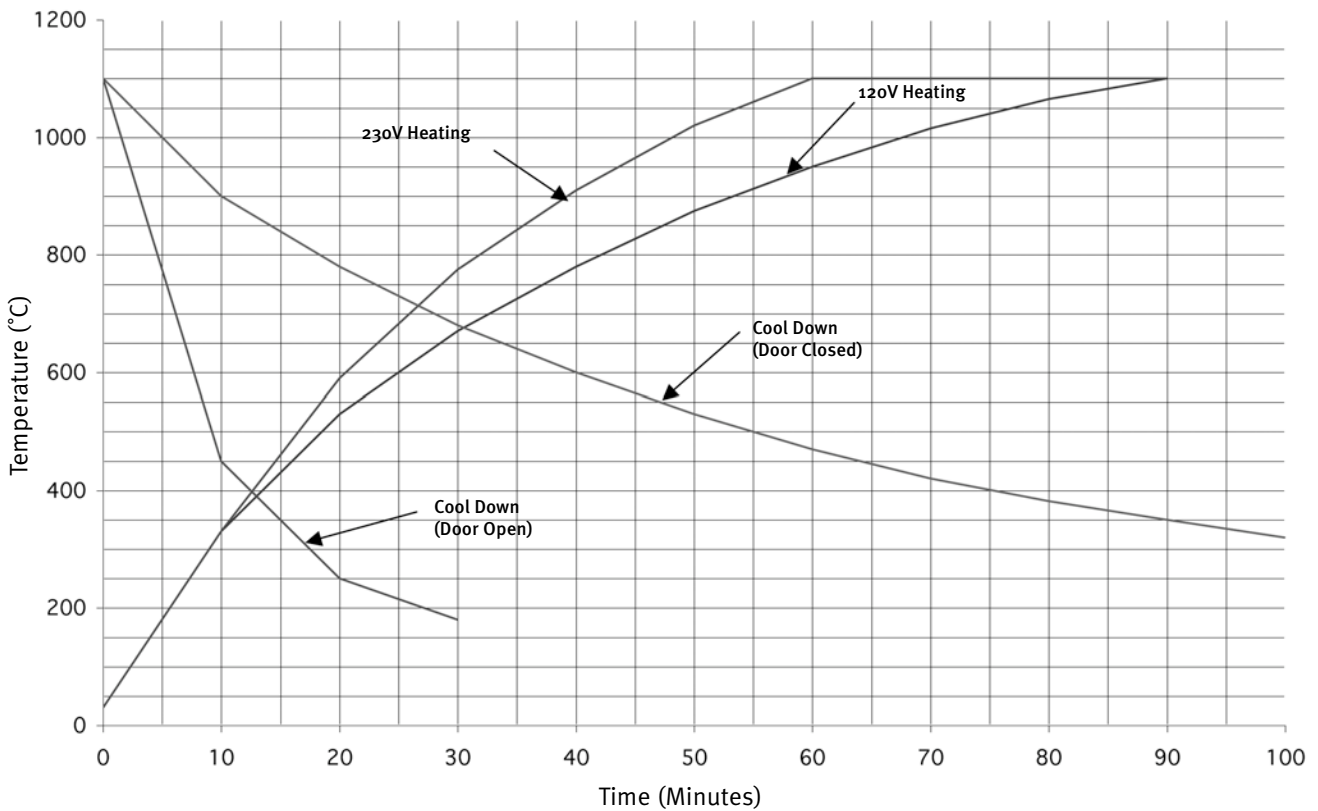TECHNICAL DATA (Contd.)

D-1750 Heating and Cooling Curves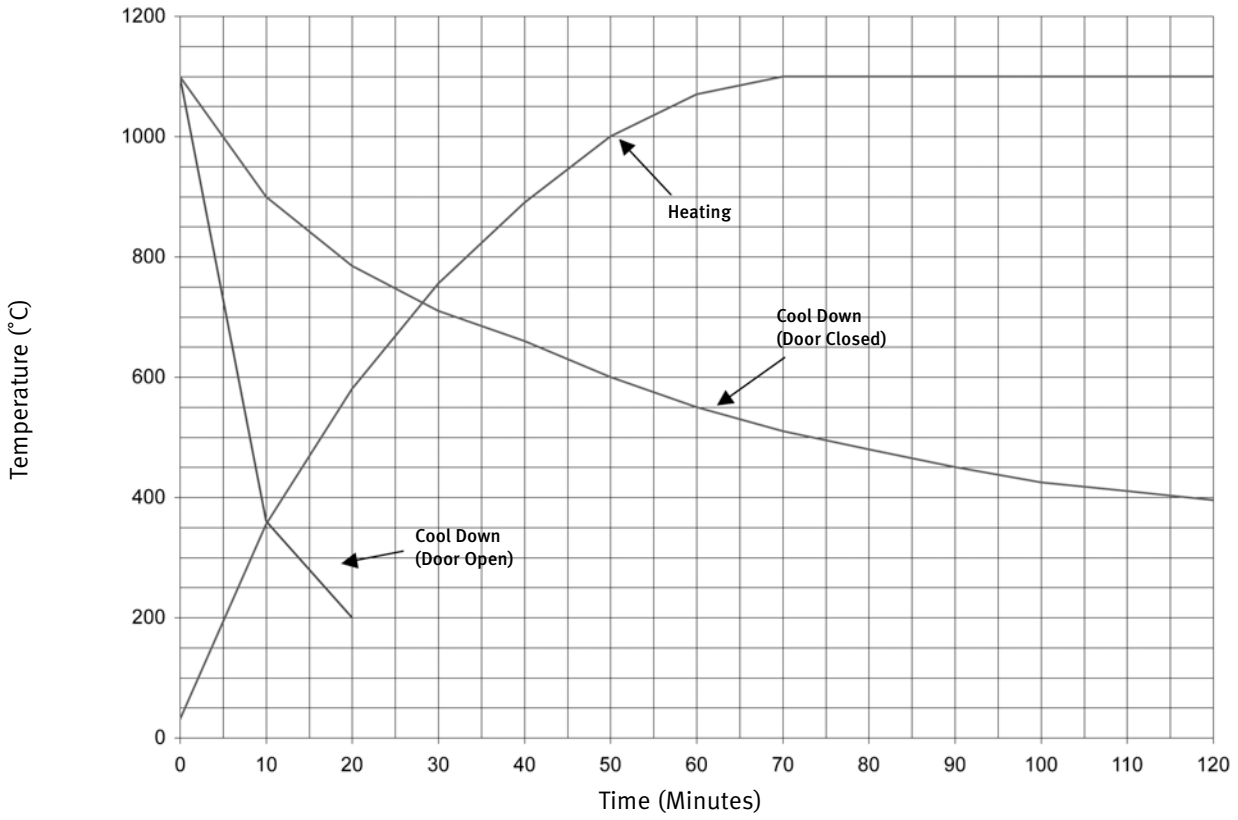

INSTALLATION INSTRUCTIONS

Shipping damage should be reported to the carrier as soon as detected.

The Unit Package contains the following:

- One furnace complete with power cord
- Owner's & Operator's Manual
- Ceramic Floor tray (shipped in the muffle)
- Two exhaust port plugs (for heat treating applications only)

UNPACKING:

Save the box and other packing material for future use in transporting the unit.

- Open the box.
- The Model 1750 requires two people to lift. Carefully lift the unit from the box by lifting on packaging supports. Support unit from the bottom to prevent slipping.
- Remove packaging supports.
- Remove plastic bag.
- IMPORTANT! The furnace is shipped with packing material inside the muffle. Remove this material before connecting power to the furnace.

INSTALLATION:

1. Remove all packing material from in and around the furnace. The furnace should be located at least 15cm (6") away from walls, shelves and heat sensitive materials. Open the furnace door and remove the packing material from inside the furnace. NOTE: The furnace front panel will show some discoloration around the muffle due to the calibration and burn in cycles performed at the factory.
2. The furnace should **not** be located directly under shelves and other airflow restrictions.
3. On high voltage (230 volt) units, connect the line cord packaged in the rear of the furnace to the socket in rear of furnace.
4. a. (Combustion, Burnout or Reaction Processes) Position the furnace under a vent hood or connect the exhaust port to a ventilation system to prevent exposure to the exhaust fumes. The furnace exhaust port 25 mm (1") OD by 25 mm (1") long **[50 mm (2") OD by 20 mm (0.8") long on 1750 Model]** can be ducted into the exhaust hood for more effective ventilation. Stainless Steel flexible metal tubing can be used for this ducting.
4. b. (Heat Treating or Non-Reaction Processes) Position the furnace under a vent hood. Plug the exhaust port with the ball plugs provided in the furnace accessory kit. This will reduce heat loss and electricity requirements.
5. Connect the furnace to a power circuit or receptacle with an over current protection (circuit breaker or fuse) rating of 20 Amps on the low voltage model and 10 Amps on the high voltage model. The furnace should be the only load on this circuit. The 1750 North American model requires a 20A supply (breaker/receptacle). The 1750 Euro model requires a dedicated 32A supply (breaker/receptacle).
6. Turn on the furnace lighted power switch (right-hand side of the control panel).


WARNING:

Operating the Furnace without the Ceramic Floor Tray may damage the furnace.
Never operate the Unit in close proximity to combustible materials or place materials on top of the unit.
Do not use extension cords with this equipment.
Plug the power cord into a circuit rated a minimum of 20A.
Do not position the unit in a way that blocks access to the power cord inlet socket or the wall outlet.

GENERAL OPERATION

FUNCTION KEY DESCRIPTIONS:


Front Panel Layout: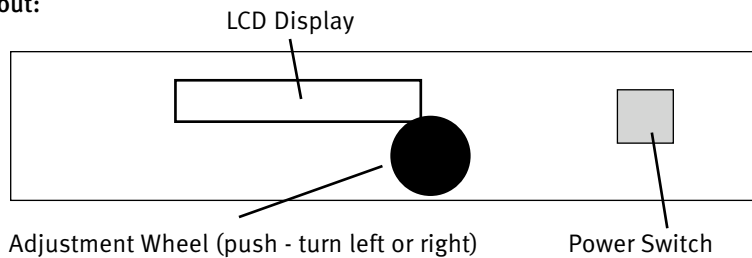

FUNCTION OPERATION:


Display Sequences at first power up:

- Plug the power cord into a wall outlet.
- Turn on the front panel power switch.
- The furnace control will go through a self-diagnostic test.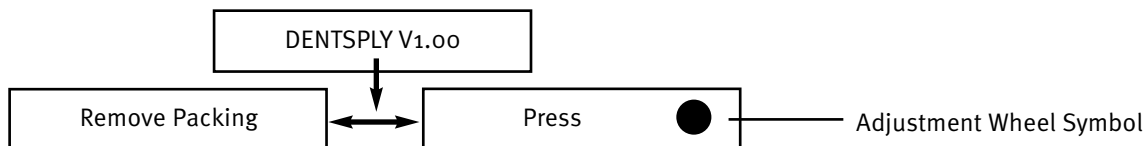

The Display shows the "Remove Packing" reminder and alternately the "Push the **Adjustment Wheel**" command.

- Remove the packaging material if not done so already and push the **Adjustment Wheel**.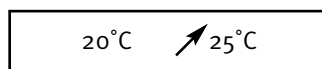

The furnace is now in operating condition, the arrow indicates temperature will increase to 25°C.

GENERAL OPERATION (Contd.)

Adjustment Wheel Operation:

The turning of the Adjustment Wheel is only detected by the control electronics after it is has been Pushed.


Note:

If after a **Push** or **Turn** the wheel is not active again within 7 seconds the display reverts back to its previous state.

Follow the examples below on how to program a new value (a counterclockwise turn decreases the value).


Changing Temperature:

One push prompts the operator to change the programmed muffle temperature. The left side of the display shows the current set temperature and the right side of the display accepts the new temperature to be controlled. A slow turn of the **Adjustment Wheel** will change the value by 1. A slightly faster turn will increase the value by 10, while fast turning of the wheel results in a value change by 100.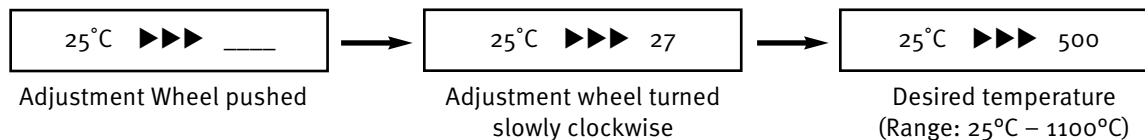


In order to store the new temperature value, push the **Adjustment Wheel**.

The display shows the actual furnace temperature and indicates the target temperature.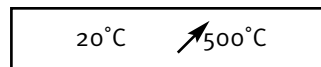

Changing the Heat Rise:

Two pushes prompt the operator to change the programmed heat rise. The left side of the display shows the current set heat rise and the right side of the display accepts the new heat rise to be controlled. A slow turn of the Adjustment Wheel will change the value by 1. Faster turning will increase the value by 10.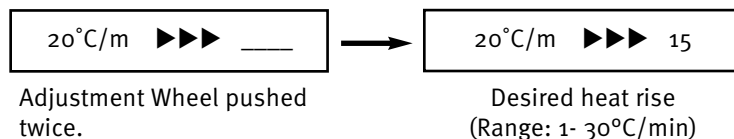


In order to store the new heat rise value push the **Adjustment Wheel**.

The display shows the actual furnace temperature and indicates the target temperature which is now controlled at 15°C/min.

GENERAL OPERATION (Contd.)


Activating the Hold Time:

Three pushes prompt the operator to activate a Hold time to count down after the heat rise finished. The left side of the display shows the currently programmed hold time while the right side of the display accepts the new hold time (up to 60:00 hours) to be controlled. A slow turn of the **Adjustment Wheel** will change the value by 1minute. Faster turning will increase the value by 10, very fast by hours.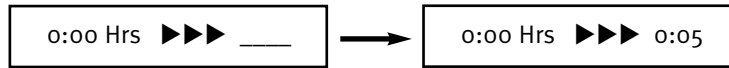

Activating the Alarm:

Four pushes prompt the operator to select the alarm function which is active at the Hold temperature when set. A value of 0 disables the alarm. Programming a value of 1 will emit a short beep every 4 seconds (2 = 0.5 sec on, 3 = 1 sec on, 4 = 2 sec on, 5 = short pause every 4 seconds)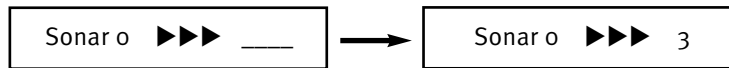


Adjustment Wheel pushed
Three times

Desired alarm value

Changing the Temperature Units:

Five pushes prompt the operator to select the temperature and the heat rise to be displayed in either °C or °F. Turning the Adjustment Wheel clockwise will show the °F unit.

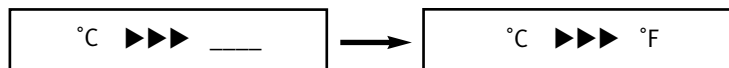Adjustment Wheel pushed
four times

Clockwise turn

GENERAL OPERATION (Contd.)

Display at the Programmed Temperature:

Once the furnace muffle reaches its programmed temperature the display shows either: A. the programmed hold time or B. Hold or C. Hold with the alarm symbol.

500°C 0:59 Hrs

A) Hold time count down

500°C Hold

B) No alarm function set
or hold time.

500°C Hold))

C) Alarm function active

Note Push the **Adjustment Wheel** to deactivate the alarm function

After opening the muffle door to retrieve the load the display shows the actual temperature and the target temperature. The target temperature is again controlled by the programmed heat rise.

430°C ↗ 500°C

During the Hold countdown the display shows the time remaining although the temperature decreases after opening the muffle door.

430°C 0:15 Hrs

CAUTION: Use caution when removing materials from muffle.

MAINTENANCE/ERROR MESSAGES

WARNING:

This equipment contains dangerous voltages. Maintenance and repair work should only be performed by an authorized service technician of DENTSPLY Ceramco.

To keep the equipment in good working order you should follow the guidelines below.

- Examine the equipment regularly for mechanical damage.
- Do not use solvents or liquid cleaners on the control panel.

TEMPERATURE CALIBRATION:

Every VULCAN furnace is calibrated in the factory at 1000°C. Under normal use the furnace should not require calibration. The electronics used in the VULCAN furnaces are very stable and will have minimal drifts over the life of the furnace. Thermocouple replacement could be a potential requirement for calibration if high accuracy is required.

For example:

A furnace is operating at a stable temperature and a separate thermocouple is inserted in the furnace and a digital thermometer measures the muffle temperature. The display shows 875°C which is the programmed temperature but the digital thermometer reads 868°C (need +7°C offset adjustment).

Software V1.10

Turn the trimmer potentiometer, RP3, which is located above the thermocouple connector on the “D” controller board clockwise until the display reads the same temperature as the meter. Allow the furnace temperature to stabilize for 5 – 10 minutes before making further adjustments.

Software V 2.20

Turn off the furnace then turn on again while pushing the adjustment wheel until the display shows the current offset.

Example:

Offset = -12°C

Using the example above, release the adjustment wheel and turn counter clockwise to add the +7°C offset needed. The new Offset reading is: Offset = -5°C.

Press the adjustment wheel to enter this value. The digital thermometer should read 875°C after the temperature stabilizes.

CLEANING:

- Clean the unit as needed.
- Vacuum dust and dirt from the furnace rather than blow. This will minimize the amount of air born dust particles.
- Clean exterior surfaces with a soft, damp cloth using a mild detergent and water.
- DO NOT IMMERSE IN WATER!

TROUBLESHOOTING (Contd.)

Service of this equipment should be done only by qualified personnel. Failure to comply may result in personnel injury and/or property damage.

Problem	Cause	Solution
No power to unit:	Power switch. No line voltage. Blown fuse.	Place switch in ON position. Check line voltage. Check and replace fuse.
Slow heating: (Ramp rate not achieved)	Rate programmed too low. Large load. Ramp rate may not be achieved at higher temperatures.	Push the Adjustment wheel twice and check the Rate value. Reduce load size or adjust ramp rate lower. Adjust ramp rate lower based on heating chart.
Door too loose or tight:	Force drag on door arms.	Adjust the Hex screw on the upper rear corners to tighten or loosen.
Displays "Wrong TC":	The thermocouple was incorrectly installed.	Reverse thermocouple leads on the circuit board.
Displays "Thermocouple":	The thermocouples opened.	Replace thermocouple.
Displays "Temperature":	The muffle temperature exceeded 1150°C.	Call DENTSPLY Service Department.

TROUBLESHOOTING (Contd.)

ACCESSORIES and REPLACEMENT PARTS

Description		Part Number
Tongs: 25cm (10") Stainless Steel	9390014	
Tongs: 30cm (12") Stainless Steel	9390015	
Tray: Bottom, Model 130		9353053
Tray: Bottom, Model 550		9353057
Tray: Bottom, Model 1750		9353060
Shelf: free standing Model 550		9493327
Shelf: Model 1750		9493396
Muffle Hardening Agent		9491006
Temperature Sticks: 705°C (1300°F)		9199073
Temperature Sticks: 815°C (1500°F)		9199074
Exhaust port ball plug	D - 130, 550	9491093A
	D - 1750	9492456
Service Manual		9363193
Heating Plate:	D -130	9493340
	D - 550 (120V)	9493470
	D - 550 (100/230V)	9493364
	D - 1750	9493392
Muffle Assy Complete:	D -130	9493422
	D - 550 (120V)	9493471
	D - 550 (100/230V)	9493423
	D - 1750	(Factory Repair)
Thermocouple:	D -130	9493342
	D - 550	9493342
	D - 1750	9491747

PRODUCT SERVICE

WARNING:

This equipment is designed with safety features to protect the operator and must not be modified in any form. Only qualified individuals should repair this piece of equipment. Failure to observe these precautions may result in burns or electrical shock.

Three methods of product service are available:

- Telephone assistance available at the number listed below,
- Return the unit for servicing using the instructions below,
- Call DENTSPLY at the phone number below and obtain a service manual for a nominal fee.

BEFORE RETURNING THE UNIT:

- Call DENTSPLY for an RMA (Return Material Authorization) number. This is used to track and identify your unit. Equipment received without this number may not be identifiable.
- Equipment damaged in shipment as a result of improper packing may not be paid by the carrier.

DENTSPLY will not be responsible for damages resulting from improper packing.

Ship prepaid to:

DENTSPLY Ceramco

ATTN: Equipment Repair

PR Number _____

470 West College Ave.

York, PA 17404 USA

Phone: 800.835.6639 Option #1 (US Customers)

717.849.4502 (International Customers)

Fax: 909.795.5268

Email: equipmentrepair.ca@dentsply.com

WARRANTY

WARRANTY: Except with respect to those components parts and uses which are hereinafter described, DENTSPLY Ceramco warrants this furnace to be free from defects in material and workmanship for a period of two years from the date of sale. DENTSPLY Ceramco 's liability under this warranty is limited solely to repairing or, at DENTSPLY Ceramco 's option, replacing those products included within the warranty which are returned to DENTSPLY Ceramco within the applicable warranty period (with shipping charges prepaid), and which are determined by DENTSPLY Ceramco to be defective. This warranty shall not apply to any product which has been subject to misuse; negligence; or accident; or misapplied; or modified; or repaired by unauthorized persons; or improperly installed.

INSPECTION: Buyer shall inspect the product upon receipt. The buyer shall notify DENTSPLY Ceramco in writing of any claims of defects in material and workmanship within thirty days after the buyer discovers or should have discovered the facts upon which such a claim is based. Failure of the buyer to give written notice of such a claim within this time period shall be deemed to be a waiver of such claim.

DISCLAIMER: The provisions here-in stated DENTSPLY Ceramco sole obligation and exclude all other remedies or warranties, expressed or implied, including those related to **MERCHANTABILITY** and **FITNESS FOR A PARTICULAR PURPOSE**.

LIMITATION OF LIABILITY: Under no circumstances shall DENTSPLY Ceramco be liable to the buyer for any incidental, consequential or special damages, losses or expenses.

LIMITATION OF ACTIONS: The buyer must initiate any action with respect to claims under the warranty described in the first paragraph within one year after the cause of action has accrued.

Corporate and Sales Office:

DENTSPLY Ceramco
DENTSPLY International
570 West College Avenue
York, PA 17404-0872 USA
PH: 800.487.0100

FAX: 800.735.1101
www.dentsply.com

Product Service Office:

DENTSPLY Ceramco
DENTSPLY International
470 West College Ave.
York, PA 17404 USA
PH: 800.835.6639 Option #1 (US Customers)
717.849.4502 (International Customers)
FAX: 909.795.5268
Email: equipmentrepair.ca@dentsply.com

DENTSPLY

CERAMCO®

Manufactured for:
DENTSPLY International Inc.
DENTSPLY Ceramco
570 West College Ave.
York, PA 17405-0872
1-800-243-1942
www.prosthetics.dentsply.com or www.dentsply.com

Distributed by:
DENTSPLY Canada
161 Vinyl Court
Woodbridge, Ontario
L4L 4A3
905-968-5374

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Printed in U.S.A.