

Castone[®] Dental Stone

TYPE III WHITE

 **25 Lbs.** (11.4 kg.)
ReOrder #99047



+D001990470+

 **50 Lbs.** (22.7 kg.)
ReOrder #99046



+D001990460/

 **100 Lbs.** (45.4 kg.)
ReOrder #99049



+D0019904900

For Dental Use Only

Caution: Inhalation of powder may be hazardous to your health.

CASTONE is a high quality, high strength dental stone used for dental casts, investing and articulator mounting.

Directions:

For Manual Mixing- Add 100g powder to 30 cc water in a clean, dry mixing bowl. Soak for 30 seconds, then vigorously spatulate for 30 seconds.

For Mechanical Mixing- Add 100g powder to 30 cc water in a clean, dry mixing bowl. Soak until powder is wet. Mix for 20 seconds, 200 to 300 turns of the blade.

Distributed By:
DENTSPLY Trubyte
DENTSPLY International Inc.
York, PA 17405-0872

1-800-786-0085

www.dentsply.com

Lot No.

DENTSPLY
TRUBYTE

CREAM

 **25 Lbs.** (11.4 kg.)
ReOrder #99043



+D001990430.

 **50 Lbs.** (22.7 kg.)
ReOrder #99044



+D001990440_

 **100 Lbs.** (45.4 kg.)
ReOrder #99045



+D001990450\$

Properties:

Powder/Water Ratio	100 g: 30 cc
Compression Strength	Wet 3,000 psi Dry 8,000 psi
Setting Time	8-10 minutes
Setting Expansion	0.08%

Storage: Gypsum materials are subject to deterioration when exposed to the atmosphere. Store this product at 59°F to 86°F. Keep container tightly closed once seal is broken to protect contents from moisture and high humidity.

NOT RETURNABLE FOR CREDIT IF SEAL IS BROKEN.

Made & Printed in U.S.A.

4235-E Rev. A (2/99)

Exp. Date