MIDWEST[®]

DENTSPLY
PROFESSIONAL

Maintenance Coupler

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MIDWEST[®] Maintenance Coupler (REF 780161)

For Use With Midwest[®] Stylus[™]
High Speed Handpieces

Installation, Maintenance and Operation Instructions

INDICATIONS FOR USE:

The Midwest maintenance coupler is designed for use with the Midwest Stylus fiber optic high speed handpiece. It is to be used with a hose that supplies air during the process of maintaining your high speed handpieces.

CONTRAINDICATIONS:

Do not use this coupler for operative procedures. This coupler is for maintenance use only.

WARNINGS:

- Do not sterilize coupler. Do not use disinfectant on coupler.
- NEVER lubricate a Stylus[™] EasyCare[™] handpiece (easily identified by red band at base of handpiece). Stylus[™] EasyCare[™] handpieces are pre-lubricated and do not require cleaning,

lubrication or expelling procedures. See Stylus™ EasyCare™ directions for use for more details.

Operation

1. Thread maintenance coupler on purge station or air hose used for high speed handpiece maintenance.
2. Disconnect handpiece from coupler and operatory hose.
3. Push button to remove bur.
4. Scrub and rinse handpiece per maintenance instructions provided in the Stylus directions for use.
5. Insert maintenance coupler inside handpiece.
6. Run handpiece (without bur) until it reaches normal speeds or until all excess lubricant is expelled (a minimum of 20 seconds.) Expel using purge station or air hose used for high speed handpiece maintenance. Wipe off excess lubricant on outside of handpiece with dry gauze pad or paper towel.

Warranty

The Midwest[®] maintenance coupler is guaranteed to be free of defects in materials and workmanship for a period of twelve (12) months from its date of purchase — provided that the coupler has been operated and maintained as prescribed in these instructions (at the discretion of DENTSPLY) and has not been subjected to apparent misuse, abuse or accident. All questionable, defective, or loose parts must be returned to the dealer, with coupler and proof of purchase.

There are no warranties, expressed or implied beyond the description on the face hereof. DENTSPLY neither assumes, nor authorizes any person to assume for it, any other liability for incidental and consequential damages resulting from the use of the coupler. To receive reimbursement for a defective coupler, you must send the defective coupler, within the warranty period to the dealer with specification of the defect and receipt of purchase.

Repair—Within The United States

Midwest Air Repair™ Service

Should your Midwest Maintenance Coupler need repair, call 1-800-800-7202 to schedule a free pick-up.

Outside the United States:

Contact your DENTSPLY Representative

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Copyright 2013 DENTSPLY International
DENTSPLY Limited
Made in U.S.A.
Code No. 86978-0413 Printed in U.S.A.