Dyract®flow

Flowable Compomer Restorative

DENTSPLY
CAULK

DIRECTIONS FOR USE – ENGLISH

For Dental Use Only.

USA: Rx Only.

1. PRODUCT DESCRIPTION

Dyract®flow Flowable Compomer Restorative is a compomer restorative material with flow characteristics which make it ideal for small cavities in anterior and posterior teeth. Dyract®flow Restorative adapts to the cavity wall without the use of hand instruments.

1.1 Delivery forms

Dyract®flow Restorative is available in:

- Small syringes for direct intra-oral application
- 7 shades: 5 Vita® shades, a translucent, and an opaque shade

1.2 Composition

Strontium-alumino-fluoro-silicate glass; Phosphoric acid modified polymerizable monomers; Carboxylic acid modified macromonomers; Reactive diluent; Camphorquinone (CQ) Photoinitiator; Stabilizer; Colorants; Titanium dioxide

1.3 Indications

Dyract®flow Restorative is indicated for the following procedures:

1. Restoration of minimally invasive cavity preparations, pits and fissures, including air abrasion Class I and tunnel preparations.
2. Restoration of minimal, shallow Class V preparations, including incipient Class V abfraction and erosion lesions.
3. As a base/liner under composite, high-density composite and compomer Class I & Class II restorations.
4. Undercut blackout and filling of small defects in indirect restoration preparations.
5. Small margin defect repair in otherwise sound restorations.

1.4 Contraindications

Dyract®flow Restorative is contraindicated for

- Use with patients who have a history of severe allergic reaction to methacrylate resins.

1.5 Compatible adhesives

Dyract®flow Restorative is used following application of a suitable dentin/enamel adhesive and is compatible with all DENTSPLY adhesives designed for use with visible light cured compomer restoratives (see complete Directions for Use of selected adhesive). Use of other dentin/enamel adhesive systems with Dyract®flow Restorative material is at the discretion and sole responsibility of the dental practitioner.

2. GENERAL SAFETY NOTES

Be aware of the following general safety notes and the special safety notes in other chapters of these directions for use.

2.1 Warnings

1. Dyract®flow Restorative contains polymerizable methacrylate monomers which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
2. **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
3. **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs immediately remove material with cotton and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
4. **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

2.2 Precautions

1. This product is intended to be used as specifically outlined in the Directions for Use. Any use of this product inconsistent with the Directions for Use is at the discretion and sole responsibility of the practitioner.
2. Contact with saliva and blood during composite placement may cause failure of the restoration. Use of rubber dam or adequate isolation is recommended.
3. Wear suitable protective eyewear, mask, clothing and gloves. Protective eyewear is recommended for patients.
4. Dyract®flow Restorative should extrude easily. DO NOT USE EXCESSIVE FORCE. Excessive pressure may result in unanticipated extrusion of the material.
5. The Dyract®flow Restorative material syringe should be tightly closed immediately after use. Do not allow dispensing tip to remain as a syringe cap. Replace original cap tightly after each use. Discard and properly dispose dispensing tips after use, as the tips may clog if material is allowed to dry or set inside.

6. DENTSPLY Caulk supplies the appropriate dispensing tip for the Dyract®flow Restorative material syringe. This tip is the only tip that should be utilized for placement of the material.
7. Insufficient data exist to support the use in Class I cavities with isthmus widths beyond central groove areas, or when replacing a functional cusp. Excessive wear or restoration failure may result.

8. Interactions:

- Eugenol-containing dental materials should not be used in conjunction with this product because they may interfere with the hardening and cause softening of the polymeric components.
- The Dyract®flow Restorative is light-cured material. Therefore, it should be protected from ambient light. Proceed immediately once material has been placed.

2.3 Storage Conditions

Dyract®flow Restorative should be kept out of direct sunlight and stored in a well ventilated place at room temperatures between 10°C/50°F and 24°C/75°F. Refrigerated storage is not required, but is acceptable when not in use. Store syringe with original cap only. Allow material to reach room temperature prior to use. Protect from moisture. Do not freeze. Do not use after expiration date.

Humidity can adversely affect the properties of unsealed Dyract®flow Restorative syringes.

Keep syringes sealed in their pouches until use. Under normal ambient conditions, unsealed syringes stay usable for approximately 8 weeks. Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

2.4 Adverse reactions

Product may irritate the eyes and skin. **Eye contact:** irritation and possible corneal damage.

Skin contact: irritation or possible allergic response. Reddish rashes may be seen on the skin.

Mucous membranes: inflammation (see Warnings).

3. STEP-BY-STEP INSTRUCTIONS FOR USE

Direct Restoration, Class I, III, V, small blockouts and restoration repairs

1. Shade Selection: Shade selection should be made prior to the restorative procedure while the teeth are hydrated. Remove any extraneous plaque or surface stain with a rubber cup and pumice or non-fluoridated prophylaxis paste. A Vita Lumin® Vacuum shade guide may be used. The Dyract®flow Restorative shade corresponds to the central part of the respective Vita® tooth.
2. Cavity Preparation:
 - 2.1 Unless repairing an otherwise sound restoration, intended to remain in place, prepare the cavity so that no residual amalgam or restorative material is left.
 - 2.2 In cases where no cavity preparation has been carried out, clean the tooth surface with a rubber cup and pumice or fluoride free prophylaxis paste (e.g. Nupro® Prophylaxis paste).
 - 2.3 Rinse surface with water spray and carefully dry it with air spray. Do not desiccate the tooth substance.
 - 2.4 Use a dental dam or cotton rolls to isolate the cavity from contamination.
3. Placement of Matrix – for optimal proximal contacts proceed as follows:
 - 3.1 Place a matrix (e.g. AutoMatrix® matrix system or Palodent® matrix system) and wedge. Burnishing of the matrix band will improve contact and contour. Pre-wedging/BITine® ring placement is advocated.
 - 3.2 In class II cavities use a deadsoft, thin matrix band.
4. Pulp protection, Tooth Conditioning/Dentin Pretreatment, Adhesive Application: Refer to adhesive manufacturer's directions for pulp protection, tooth conditioning and/or adhesive application. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately to placement of Dyract®flow Restorative material.
5. Dispensing Dyract®flow Flowable Compomer Restorative Material
 - 5.1 Remove cap from the end of the syringe. To assure free flow of material from syringe, express a small amount onto pad, away from the patient field.
 - 5.2 Attach disposable, black dispensing tip to end of the syringe. Turn tip clockwise 1/4 to 1/2 turn to assure that it is fully seated. Tug on tip to be sure that it is locked into the collar of the syringe.
 - 5.3 Dyract®flow Restorative should flow freely with gentle pressure. DO NOT USE EXCESSIVE FORCE. If more than gentle pressure is required, remove from patient field and check for obstruction.
 - 5.4 DISCARD AND PROPERLY DISPOSE DISPENSING TIP IMMEDIATELY AFTER USE. REPLACE ORIGINAL CAP. Do not store syringe with black dispensing tip in place. STORE ONLY WITH ORIGINAL CAP. Note: It is recommended to pull back slightly on the syringe plunger after use to prevent excessive flow of material.
6. Placement of Dyract®flow Restorative Material:

Danger of injury due to excessive force

- Apply slow and steady pressure on the syringe plunger
- Do not use excessive force – syringe rupture may result

Dispense Dyract[®]/flow Restorative directly into preparation site/tooth surface from the syringe tip using slow, steady pressure. Dyract[®]/flow Restorative should extrude easily. DO NOT USE EXCESSIVE FORCE. Avoid lifting the tip while dispensing to minimize air entrapment. Any visible air bubbles should be pierced with a clean, sharp explorer prior to curing. Material may be placed and light cured in increments up to 2mm. (See Curing, Step 7) In deeper preparations it is recommended to place material in 2mm increments, thoroughly light curing each increment. For maximum light transmission and to minimize finishing, avoid overfilling cavity.

7. Curing:

Inadequate polymerization due to insufficient curing

- Check compatibility of curing light
- Check curing cycle
- Check curing output before each procedure

Light cure each area of the restoration surface with a suitable visible light curing unit designed to cure materials containing camphorquinone (CQ) initiator, i.e. spectral output containing 470nm. Minimum light output must be at least 550mW/cm² exposure for at least 20 seconds. For darker/opaque shades (A4 and A3-Opaque) cure for another twenty seconds. DENTSPLY manufactured advanced performance curing units have been shown to cure 2mm increments of most shades of Dyract[®]/flow Restorative material in 10 seconds. Refer to curing light manufacturer's recommendations for compatibility and curing recommendations. The Dyract[®]/flow Restorative should be additionally exposed to the curing unit through the proximal, lingual, and buccal enamel walls following matrix removal for the recommended time.

8. Finishing and Polishing – Start finishing immediately after curing the last increment.

8.1 Contour the restoration using finishing burs or diamonds.

8.2 Use Enhance[®] finishing discs, cups or points or standard aluminium oxide disc series for additional finishing.

8.3 Prior to polishing, check that surface is smooth and free of defects.

8.4 For achieving a very high luster of the restoration, we recommend one of the following methods:

- Use PoGo[®] Polishes.
- Use Enhance[®] polishing foam cups with Prisma[®]•Gloss[™] Regular paste followed by Prisma[®]•Gloss[™] Extrafine polishing paste.

For details see complete directions for use.

STEP-BY-STEP INSTRUCTIONS FOR USE

Cavity Liner

Prior to use as a cavity liner, follow Steps 1 through 5 outlined above for:

1. Shade selection
2. Cavity preparation
3. Matrix placement
4. Pulp protection, Tooth Conditioning/Dentin Pretreatment, Adhesive Application
5. Dispensing (syringe).
6. Placement: Dispense Dyract[®]/flow Restorative directly into preparation site/surfaces. For most cavity lining applications, a thickness of 1mm or less is recommended. With the exception of Class II proximal box (gingival) margins, cavity margins/conditioned enamel are usually closed with selected restorative material. Lining is generally confined to internal conditioned dentin surfaces. Avoid lifting the tip while dispensing to minimize air entrapment. Any visible air bubbles should be pierced with a clean, sharp explorer prior to curing. **Technique Tip:** Use of a clean instrument such as a sharp explorer may be used to evenly spread a thin layer of dispensed Dyract[®]/flow Restorative material over prepared cavity (dentin) surfaces prior to curing.

7. Curing: Light cure each area of the lined surface with a suitable visible light curing unit designed to cure materials containing camphorquinone (CQ) initiator, i.e. spectral output containing 470nm. Minimum light output must be at least 550mW/cm² exposure for at least 40 seconds for liner application.
8. If a thickness of material greater than 1mm is desired, repeat Placement Step 6 and Curing Step 7, placing material in 1mm increments and curing until desired thickness is reached.
9. Proceed immediately to complete the restoration per restorative material manufacturer's Directions for Use. For adhesive restorations, to not disturb or allow contamination of cured liner surface. If contamination occurs, follow adhesive manufacturer's Directions for Use for re-application of adhesive.
Technique Tip: If surfaces remain undisturbed, additional application of adhesive between increments is neither necessary nor recommended.

4. HYGIENE

4.1 Cleaning and disinfection

To prevent Dyract[®]/flow Restorative syringes from exposure to spatter or spray of body fluids, contaminated hands or oral tissues, use of a protective barrier is recommended to avoid syringe contamination. Repeated disinfection may damage label.

5. LOT NUMBER AND EXPIRATION DATE

1. Do not use after expiration date. ISO standard is used: "YYYY/MM"
2. The following numbers should be quoted in all correspondences:
 - Reorder number
 - Lot number on the syringe
 - Expiration date on the syringe

©2008 DENTSPLY International Inc. All Rights Reserved. Printed in U.S.A.

Vita Lumin Vacuum Shade Guide is not a registered trademark of DENTSPLY International Inc.
Vita is a registered trademark of Vita-Zahnfabrik H. Rauter GmbH & Co.

Manufactured by:
DENTSPLY DeTrey GmbH
De-Trey-Str. 1
78467 Konstanz
Germany
Tel.: +49-7531-583-0

Distributed by:
DENTSPLY Caulk
38 West Clarke Avenue
Milford, DE 19963 USA
Tel.: 1-302-422-4511
www.dentsply.com

Distributed by:
DENTSPLY Canada
161 Vinyl Court
Woodbridge, Ontario
L4L 4A3 Canada

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©2008 DENTSPLY International Inc. Tous droits réservés. Imprimé aux U.S.A.