

Get Real.




***ESTHET-X<sup>®</sup> HD***  
*HIGH DEFINITION MICRO MATRIX RESTORATIVE*

## HOW TO ACHIEVE THE MOST REALISTIC RESTORATION.

Start with the perfect blend of strength, handling and exquisite color-matching. Finish with consistently beautiful, natural results. That's how to Get Real. When you take advantage of Esthet•X® HD composite's nano-filler technology and TruMatch® Shade Guide, only you and your patient will know it's a restoration.

### The Esthet•X® HD Composite TruMatch® Shade Guide makes it easy to Get Real.

When you can visualize the finished restoration before you start placing composite, you achieve a highly realistic restoration. Each shade guide tab offers a precise recipe that eliminates the guesswork, saving time and delivering stellar results.

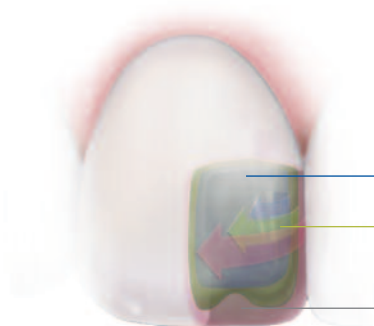


Step 1: Opaque Dentin Shades

Step 2: Regular Body Shades

Step 3: Translucent Enamel Shades

Three different opacities allow you to build realistic restorations from the inside out to replicate natural tooth structure and achieve a high polish in just 15 seconds.

#### GLOSS (GU) AFTER 15 SECONDS OF POLISH Shade A2, Method TM-042-09

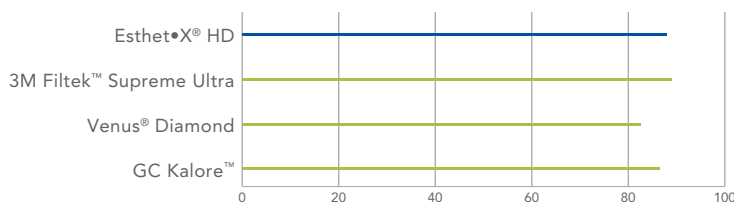

TruMatch® Shade Guide



## Ideal handling: It puts Reality at your fingertips.


Esthet•X® HD composite's optimized handling excels in both anterior and posterior restorations. These benefits apply to single- or multiple-shade restorations, and you enhance reality with Esthet•X® HD composite's high-end polishing capabilities.

## Real technological advances for realistic restorations.

Esthet•X® HD composite provides realistic restorations and has:

- Sculptable handling and workability for realistic results
- Easy and quick polishing for brilliant results
- Outstanding results for all cavity classifications
- High fracture toughness, excellent radiopacity and wear properties that rival natural enamel

### EXCELLENT FRACTURE TOUGHNESS (MPa)

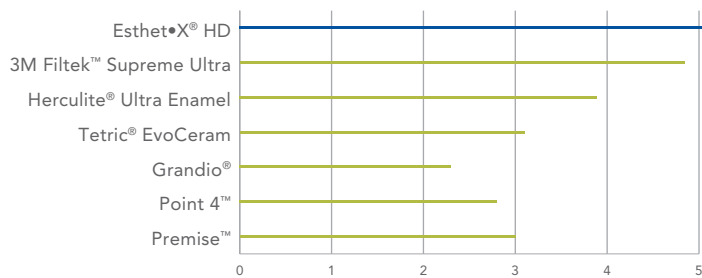

BEFORE



AFTER



Case photos courtesy Dr. Jeff Blank

BEFORE



AFTER



Case photos courtesy Dr. Martin Goldstein

BEFORE



AFTER



Case photos courtesy Dr. Bruce Kleeberger



# Ordering Information

## ESTHET•X® HD COMPULES® TIP

### 630600 COMPLETE KIT

156 Compules® Tips (0.25g each)  
10 each shade: A1, A2, A3, A3.5, A2-O  
5 each shade: W, XL, B1, B2, B3, C1, C2, C3, C5, D2, U, CE, WE, YE  
3 each shade: A4, B5, C4, D3, AE, GE, W-O, A4-O, B2-O, C1-O, C4-O, D3-O  
10 Prime & Bond® NT™ Adhesive Unit Doses  
1 Esthet•X® HD TruMatch® Blended Shade Guide  
1 Syringe Esthet•X®flow – Shade A2  
Enhance® Finishers – 10 Discs, 5 Cups, 5 Points  
PoGo® Polishers – 10 Discs, 5 Cups, 5 Points  
1 Syringe Caulk® 34% Tooth Conditioner Gel (3mL)  
1 Compules® Tips Gun  
Accessories

### 630601 INTRODUCTORY KIT

60 Compules® Tips (0.25g each)  
10 each shade: A1, A2, A3, A3.5, A2-O  
5 each shade: CE, YE  
10 Prime & Bond® NT™ Adhesive Unit Doses  
1 Esthet•X® HD TruMatch® Blended Shade Guide  
1 Syringe Esthet•X®flow – Shade A2  
Enhance® Finishers Sample  
PoGo® Polishers Sample  
1 Syringe Caulk® 34% Tooth Conditioner Gel (3mL)  
1 Compules® Tips Gun  
Accessories

### 630604 OPERATORY KIT

20 Compules® Tips (0.25g each)  
5 each shade: A1, A2, A3, A2-O, YE  
Enhance® Finishers Sample  
PoGo® Polishers Sample  
1 Compules® Tips Gun  
Esthet•X® HD Shade Tab – A2

#### REFILL PACKAGES

##### Regular Body Shades (50 each of one shade)

630617B A1  
630618B A2  
630619B A3

##### Regular Body Shades (20 each of one shade)

630615 White - W  
630617 A1  
630618 A2  
630619 A3  
630620 A3.5  
630622 B1  
630623 B2  
630627 C2  
630633 Universal - U

##### Regular Body Shades (10 each of one shade)

630616 XL  
630621 A4  
630624 B3  
630625 B5 - DY  
630626 C1  
630628 C3  
630629 C4  
630630 C5 - XGB  
630631 D2  
630632 D3

##### Translucent Enamel Shades (10 each of one shade)

630610 Clear Enamel - CE  
630611 White Enamel - WE  
630612 Yellow Enamel - YE  
630613 Amber Enamel - AE  
630614 Gray Enamel - GE

##### Opaque Dentin Shades (10 each of one shade)

630634 White Opaque - W-O  
630635 A2-O  
630636 A4-O  
630637 B2-O  
630638 C1-O  
630639 C4-O  
630640 D3-O

## ESTHET•X® HD EASY-TWIST SYRINGE

### 630711 COMPLETE KIT

31 Syringes (3g each) – 1 of each shade:  
A1, A2, A3, A3.5, A4, B1, B2, B3, B5, C1, C2, C3, C4, C5, D2, D3, U, YE, AE, GE, CE, WE, W, XL, W-O, A2-O, A4-O, B2-O, C1-O, C4-O, D3-O

### 630714 INTRODUCTORY KIT

5 Syringes (3g each)  
1 of each shade: CE, A2, A3, A3.5, A2-O  
5 Prime & Bond® NT™ Adhesive Unit Doses  
Prime & Bond® NT™ Accessories  
1 Syringe Caulk® 34% Tooth Conditioner Gel  
Enhance® Finisher Sample  
Palodent® System Sample

#### REFILL PACKAGES (1 syringe 3g)

##### Regular Body Shades

630655 White - W  
630656 XL  
630657 A1  
630658 A2  
630659 A3  
630660 A3.5  
630661 A4  
630662 B1  
630663 B2  
630664 B3  
630665 B5 - DY  
630666 C1  
630667 C2  
630668 C3  
630669 C4  
630670 C5 - XGB  
630671 D2  
630672 D3  
630673 Universal - U

##### Translucent Enamel Shades

630650 Clear Enamel - CE  
630651 White Enamel - WE  
630652 Yellow Enamel - YE  
630653 Amber Enamel - AE  
630654 Gray Enamel - GE

##### Opaque Dentin Shades

630674 White Opaque - W-O  
630675 A2-O  
630676 A4-O  
630677 B2-O  
630678 C1-O  
630679 C4-O  
630680 D3-O

## ACCESSORIES

### 635105 CAULK COMPULES® TIP GUN

### 630603 ESTHET•X® HD TRUMATCH® SHADE GUIDE

## ESTHET•X® FLOW COMPULA® TIP

### 648101 INTRODUCTORY KIT

24 Compula® Tips (0.25g each)  
3 of each shade: A1, A2, A3, A3.5  
2 of each shade: A4, B1, U, BW, C4, A2-O  
1 Compules® Gun  
1 Syringe Caulk® 34% Tooth Conditioner Gel (3mL)  
20 Flocked Applicator Tips  
Enhance® Finishers Sample  
PoGo® Polishers Sample  
Esthet•X® HD Compules® Tips Sample  
20 Prime & Bond® NT™ Adhesive Unit Doses

#### REFILL PACKAGES

##### 20 Compula® Tips (0.25g each)

648110 A1  
648111 A2  
648112 A3  
648116 A3.5  
648113 A4  
648114 B1  
648118 C4  
648115 U/LYG (Universal)  
648117 BW  
648119 A2-O

## ESTHET•X® FLOW SYRINGE

### 648001 INTRODUCTORY KIT

7 Syringes (1.3g each) – 1 of each shade:  
A1, A2, A3, A3.5, A4, B1, U/LYG Universal  
30 Gray Syringe Tips  
1 Syringe Dyract®flow – Shade B1 (1.3g)  
6 Black Syringe Tips for Dyract®flow  
20 Prime & Bond® NT™ Adhesive Unit Doses  
20 Microbrushes  
1 Syringe Caulk® 34% Tooth Conditioner Gel (3mL)  
PoGo® Polishers Sample  
Esthet•X® HD Compules® Tip Sample

#### REFILL PACKAGES

##### 2 Syringes (1.3g each)

##### 30 Gray Syringe Tips

648020 A1  
648021 A2  
648022 A3  
648026 A3.5  
648023 A4  
648024 B1  
648028 C4  
648025 U/LYG (Universal)  
648027 BW  
648029 A2-O

### 648003 SYRINGE TIP REFILL

60 Syringe Tips

**DENTSPLY**  
CAULK

38 West Clarke Avenue  
Milford DE 19963  
1.800.LD.CAULK (532.2855)  
www.caulk.com • www.dentsply.com