

Choose your obturation solutions



ProTaper Next®
paper points



Conform Fit™ gutta-percha
points for Protaper Next®



GuttaCore® for
ProTaper Next®

INL/ BEN PTNO TIP 000 / 01/2017
- updated 11/2017



Maillefer Instruments Holding Sàrl
Chemin du Verger 3 - 1338 Ballaigues - Switzerland
Phone: +41 (0)21 843 92 92 - endo@dentsplysirona.com
www.dentsplysirona.com

 **Dentsply
Sirona**
Endodontics

ProTaper Next®

Flexible performance



+
**WE
KNOW
ENDO.**

 **Dentsply
Sirona**
Endodontics



GLIDE PATH



PG

ProGlider®

SHAPING



X1

X2

X3

X4

X5

ProTaper Next®

OPTIONAL

ProGlider® Glide Path

PG



016

2%*

ProTaper Next® Shaping

X1



017

4%*

X2



025

6%*

X3



030

7%*

X4



040

6%*

X5



050

6%*

Motor Settings:

300 rpm / 2 - 5.2 Ncm

*variable taper