Ceramco® II & Ceramco® II Silver
Crown and Bridge Porcelain System
Instructions

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
Distributed by:
DENTSPLY Canada
161 Vinyl Ct.
Woodbridge, Ontario
L4L 4A3
905-851-6060

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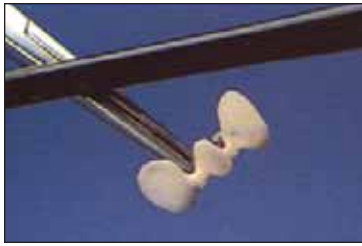Ceramco® II & Ceramco® II Silver Porcelain Systems Instructions

ENGLISH

INTRODUCTION

Ceramco II and Ceramco II Silver porcelains provide the esthetic and production qualities demanded of a premium restorative material. To achieve optimum results, observe the guidelines and instructions in this manual.

Ceramco II Silver porcelains have been engineered to provide the exceptional handling, thermal stability and esthetic properties of original Ceramco II porcelains. The system is inherently resistant to discoloration by silver-containing alloys, eliminating the need for special liquids or techniques when used with silver-containing alloys.

Indications

The Ceramco II and Ceramco II Silver porcelains may be used for porcelain fused to metal, veneer, porcelain jacket, and inlay/onlay fixed prosthodontic restorations.

Contraindications

Only the indications listed above are suitable.

Warnings

Consult applicable DENTSPLY MSDS (Material Safety Data Sheets) for safe handling and usage. Use only with recommended Ceramco II & Ceramco II Silver system components. Use protective equipment to guard against dust when grinding.

Precautions

Refer to the special notes throughout the instructions for best practices for using Ceramco II & Ceramco II Silver materials. Other porcelains are not thermally compatible and will not have the correct dilatometric characteristics, and therefore should not be used. While initial results with some materials may appear acceptable, internal stress can compromise long term success.

Porcelain mixed with liquid should not be stored for more than two days. The use of humidior-type trays for long-term storage is not recommended. Chalky porcelain or white spots can result from wet storage of porcelain.

To assure the integrity of the system, only the recommended materials should be used.

Adverse Reactions

The Ceramco II & Ceramco II Silver system contains components which may cause skin dryness, sensitization (allergic contact dermatitis or other allergic reactions in susceptible persons).

Instructions

Handling

- Ceramco II and Ceramco II Silver porcelains both meet rigid standards for fine-grain porcelains. These standards provide the user with an easy-to-handle porcelain that polishes to a high gloss.
- Condense the porcelain by blotting with a tissue; or by lightly tapping the model, then blotting away the surface moisture with a tissue. Do not use heavy vibration to condense.

Thermal Stability and Expansion

Both Ceramco II and Ceramco II Silver porcelains conform to the same ISO standard for thermal expansion and have the same excellent thermal stability.

Firing Temperature

Ceramco II Silver porcelains fire about 20°C (36°F) higher than Ceramco II porcelains. Ceramco II Silver contains a more refractory frit, necessitating the higher temperatures.

Modeling Liquids

- Caution should be used in selecting modeling liquids as some contain ingredients that cause discoloration with silver-containing alloys.
- Ceramco Universal Sculpting Liquid is recommended for its ability to improve the handling and carving of porcelain and to reduce slumping during build-up. This single liquid can be used with opaque, body and incisal porcelains.
- If you are currently using Ceramco Modeling Liquids, you may wish to continue. The Ceramco Modeling Liquids contain anti-greening agents useful for Ceramco II porcelains and are desirable for improved handling properties.

Liquid for Overglaze

Use Ceramco FineGrain™ Stain Liquid labeled "contains diethylene glycol." Other stain liquids may result in discoloration with silver-containing alloys.

Combining Systems and Use with Ultra-Pake® Porcelain

- Ceramco II Silver and Ceramco II porcelains may be combined in the same restoration. **However, the firing schedules for Ceramco II Silver porcelain must be used.**
- To ensure a complete non-greening system on silver-containing alloys, only Ceramco II Silver or Ultra-Pake porcelains should be used.

Note: Extensive testing has shown that successful restorations are achieved by combining the Ultra-Pake opaquing system with the Ceramco II Silver porcelains on silver alloys.

Furnace Purging

Use of non-greening porcelains is not a substitute for established maintenance procedures. Routine purging of the furnace should be performed to remove excess contaminants from the muffle. The frequency is dependent on usage. As a guideline, weekly purging with carbon pellets is recommended when silver alloys are used.

Color Tagging Agents

Colorizer™ pens, food color, felt-tip pens and other color tagging systems for porcelain build-up should not be used with silver-containing alloys as they may cause slight discoloration.

THE CERAMCO II & CERAMCO II SILVER PORCELAIN SYSTEM COMPONENTS

The Ceramco II line is available in "G" (German, keyed to the Vita Lumin' shade guide) and "B" (Bioform, keyed to the Bioform®, Trubyte® and Bioform Color-Ordered™ shade guides) shades with a full complement of opaque, body, incisal, modifier and specialty powders. The Ceramco II Silver line is available in "G" (German, keyed to the Vita Lumin' shade guide). This complete line provides the ceramist maximum versatility, as well as ease and consistent performance.

The following porcelains are available in both Ceramco II and Ceramco II Silver formulations:

Paint-O-Pake Porcelains	16 High Value Shades 16 Standard Shades 12 Opaque Modifiers	
Opaceous Body Porcelains*	16 Body Shades 4 Masking Shades 5 Modifiers	
Body Porcelains	16 Body Shades 5 Chroma Rich Shades 12 Body Modifiers	
Incisal Porcelains	8 Regular Incisal Porcelains 10 Opal Incisal Porcelains* Transparent Porcelain Translucent Porcelain Porcelain Overglaze 3 Tissue Tint Porcelains	
Add-On Porcelains*	4 Shades 4 Tissue Tint Shades	

Other products from Ceramco you can use in conjunction with Ceramco II or Ceramco II Silver to achieve the optimum restoration:

FinalTouch™ Low-Fusing*	3 Body Shades 3 Modifiers	3 Incisal Shades 9 Stains
Chromaline™ Chroma Enhancers*	16 Modifiers	2 Mamelons
FineGrain™ Stains*	21 Stains	1 Low-temp glaze

Separate instructions are available for the components identified with an asterisk() as well as for the Ultra-Pake system.

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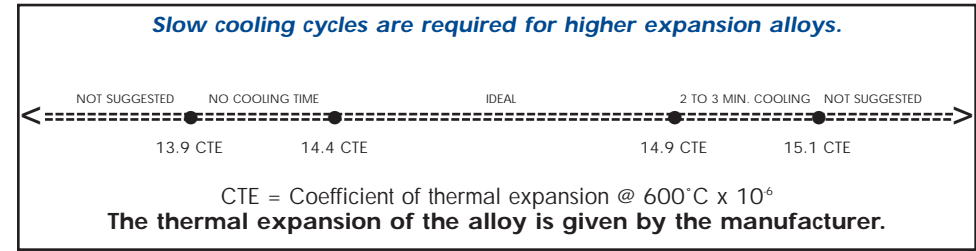
ALLOY SELECTION

Ceramco II porcelains are compatible with most alloys, although some slight variations in technique are suggested depending on the alloy.

FRAMEWORK DESIGN AND ALLOY PROCESSING

- Design the wax patterns for copings and frameworks to support an even thickness of Ceramco II or Ceramco II Silver porcelain whenever possible. Total porcelain thickness, including opaque, body, and incisal layers, should not be less than 1.0 mm or greater than 2.5 mm in any area.
- Block out sharp angles on the preparation prior to the wax-up steps. Round off sharp angles on the coping or framework during the wax-up or alloy surface finishing steps.
- Design frameworks, which have multiple pontics or large single pontics, to have an exposed band of metal on the lingual side. This lingual band will support the porcelain and reduce checking caused by thermal expansion differences between the porcelain and the metal.
- Attach a piece of 20-gauge wire, wax to the lingual collar during the wax-up step. This provides a safe holding area that protects delicate margins and can improve the ease of manipulating the framework during the porcelain application and firing process.

PAINT-O-PAKE® PORCELAINS

The purpose of the opaque porcelain is to mask the metal and provide the core color of the ceramic restoration. Ceramco offers a number of opaquing systems to provide the dental laboratory with a wide variety of options.

The following instructions are applicable for:

Ceramco II Paint-O-Pake Porcelains
Ceramco II Hi-Value Paint-O-Pake Porcelains
Ceramco II Silver Paint-O-Pake Porcelains
Ceramco II Silver Hi-Value Paint-O-Pake Porcelains
Universal Paint-O-Pake Modifier Porcelains

Note: Detailed directions for the use of Ultra-Pake opaques with Ceramco II and Ceramco II Silver porcelains are available upon request.

General Guidelines

- Mix the opaque powder with Ceramco Opaque Modeling Liquid, Universal Sculpting Liquid, or distilled water to a creamy consistency that allows the material to be applied in a thin, even layer. Ceramco Opaque Modeling Liquid improves the handling characteristics of the opaque porcelain and inhibits discoloration of the porcelain on silver-based alloys. Universal Sculpting Liquid can be used with Ceramco II Silver porcelains or when there is no concern for silver discoloration with Ceramco II porcelains.

Note: Material should be remixed with distilled water only. Do not use excessive vibration or allow thick layers of opaque porcelain to accumulate in the occlusal, interproximal or margin areas. Thick layers of opaque porcelain will tear or fissure during the firing process.

First Opaque Porcelain Application

1. Holding the framework with a hemostat or locking tweezers, apply the first opaque layer thinly and evenly with a brush or instrument. Tap the hemostat very lightly to condense and smooth the opaque porcelain surface.

Photo 1 First Opaque Porcelain Application

2. Dry the opaque porcelain with warm, forced-air equipment or by holding the casework close to, but not in, the entrance of the furnace. Do not use extreme heat to dry the opaque layer. Excessive heat causes blisters, peeling, or tearing of the opaque porcelain.
3. Fire the first opaque layer. (Consult the firing schedules in Table 7 for proper firing.) Use the visual indicators described in the firing guidelines in Table 7. Change the firing schedule to achieve these visual indicators. If the opaque layer appears underfired (dull) or overfired (glazed), remove the fired opaque by grinding or blasting; then reapply the opaque layer and fire using the suggested firing adjustments.

Second Opaque Porcelain Application

1. Remoisten the dried opaque porcelain mixture with distilled water only. The consistency for the second layer of opaque porcelain should be slightly more viscous than the initial mix.
2. Apply the second opaque porcelain layer using the same technique as the first layer. Make sure that this layer is thick enough to finish masking the metal substructure. A fired layer of 0.2 mm of opaque porcelain is required to completely mask dark metal surfaces.
3. Fire the second opaque porcelain layer. The surface should have a slight sheen. Check the fired work to make sure it has the correct visual indicator. Make firing adjustments as needed (See Table 7).

Photo 2 Second Opaque Porcelain Application

Spray-Opaque Technique

- To achieve the proper shade with the thinner opaque porcelain layer created by the spray-opaque technique, Ceramco II or Ceramco II Silver Hi-Value opaque porcelains are recommended.
- Follow the spray-opaque equipment manufacturer's recommendations for air pressure and liquids for mixing.

Caution: Some liquids can cause discoloration with silver-containing alloys.

- For the best results, apply two layers of the opaque porcelain and fire twice as described in the traditional application. Use the traditional application technique on a sample coping to determine the proper firing schedule for the individual furnace. Spray-opaque creates a surface texture which makes the visual indicator difficult to interpret.

PORCELAIN MODIFIERS

Note: Do not use intense mixtures until you are familiar with the effects of the porcelain modifiers.

- Ceramco offers 12 opaque modifiers and 12 body modifiers. Within each group, there are 6 chroma modifiers and 6 hue modifiers.

Chroma Modifiers: (See Tables 1 & 2)

- Each opaque porcelain chroma modifier has a corresponding body porcelain modifier as follows:

Red-Brown
Red-Yellow
Gray-Brown
High Value Red-Brown
High Value Red-Yellow
Neutral

The chroma modifier porcelains are concentrated colors. Use these modifiers to increase or decrease the dominant hue of a shade. Red-brown, red-yellow and gray-brown also decrease value while increasing chroma.

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- For small chroma shifts, such as achieving a shade that is between A1 and A2, use the following approximate proportions:

Opaque: 1 part opaque modifier porcelain to 12 parts opaque porcelain.

Body: 1 part body modifier porcelain to 9 parts body porcelain.

- Increase the proportion of the modifier porcelain to achieve stronger effects for cervical or occlusal areas.

Hue Modifiers: (See Tables 3 & 4)

- Hue modifiers intensify specific colors for special effects or custom shades. Hue modifiers are also used to compensate for specific color problems on cases.

Opaque Hue Modifiers:

Yellow, Pink, White, Orange, Violet, Brown

- Use violet, white and brown opaque modifier porcelain in the same proportions as the chroma modifiers: 1 part opaque modifier porcelain to 12 parts opaque porcelain. Yellow, pink and orange are not as highly concentrated colors. For these colors, a proportion of 1 part modifier to 8 parts opaque porcelain yields a moderate effect.

Body Hue Modifiers:

Yellow, Pink, White, Orange, Blue, Gray

- Blue, white and gray body porcelain modifiers are generally used in the same proportions as the chroma modifiers: 1 part modifier porcelain to 9 parts body porcelain. Yellow, pink and orange are not as intense. For these colors, a proportion of 1 part modifier to 5 parts body porcelain gives a moderate effect.

Applying Porcelain Modifiers

1. Mix the porcelain modifiers with the corresponding opaque or body porcelain, or use them undiluted for strong effects. The application, condensation, drying, and firing of modified porcelains are the same as the basic body porcelain technique. Separate bakes are not necessary.

2. Apply the opaque modifiers at the same time as the second layer of opaque porcelain.

3. Apply the modified opaque mixture to the desired areas; then apply the unmodified opaque to the remaining surfaces.

4. Use a fine-point brush to feather the modified opaque mixture into the unmodified opaque.

5. Tap very lightly to smooth the surface.

Note: Vigorous tapping or vibration shifts the modified porcelain to areas where it is not wanted.

6. Modifications which require fine line control may be applied and fired separately.

Photo 3 Applying Opaque Porcelain Modifiers at the Cervical and Interproximal areas.

7. Body modifiers may be applied by several techniques: as an underlying layer, used for the entire build-up; or placed in random locations or placed in specific locations, such as the central fossa of a posterior crown.

Note: The ChromaLine™ porcelains are available separately. These highly chromatinized modifiers (70% more chroma than standard body) are keyed to the Vita shades and can be used to enhance the color of other porcelains in the fabrication of porcelain fixed prosthodontic restorations. See separate instruction booklet for ChromaLine porcelain modifiers.

BODY, OPAQUE BODY AND INCISAL PORCELAINS

The following techniques apply to all Ceramco II & Ceramco II Silver opaque body, body, body modifier and incisal porcelains. Detailed guidelines for the opaque body porcelains and opal incisal porcelains are provided in the specific instructions for those products.

Note: The Ceramco II and Ceramco II Silver Opaque Body Porcelains address the problem of limited space, and simplify internal color modifications and overall technique. Separate directions for Opaque Body Porcelains are available with kits or by request from Ceramco.

General Guidelines

- Mix the porcelains with the Universal Sculpting Liquid, Body Modeling Liquid or distilled water to a paste-like consistency. Material should be remixed with distilled water only.
- The body porcelain may be built to full contour and then cut back for the incisal application, or slightly under contoured in some areas to allow for the immediate application of the incisal porcelain.
- With either technique, the total porcelain build-up should be slightly larger than the expected final contour to compensate for normal firing shrinkage.
- More shrinkage occurs along the incisal-cervical axis or the occlusal-cervical axis than along the labia-lingual or mesio-distal dimensions. Therefore, the most compensation for shrinkage should be in the vertical height of the crown.

Note: *Do not overbuild. Excessive build-up of the porcelain promotes interproximal tearing on bridges and splints, and makes shade control difficult. By contrast, controlled and detailed build-ups save grinding time and labor and improve color control.*

Body Porcelain Application

1. Apply the body porcelain with a Ceramco ceramist brush. Apply the body porcelain in small increments to avoid trapping air between layers.

Note: *Blot excess moisture with absorbent tissue paper as you build to prevent slumping of the mass. The blotting tissue should be placed on the surface of the build-up which is the farthest distance from the area where the porcelain is being applied. This technique improves the condensation of the porcelain mass and maintains a balanced moisture level throughout the build-up.*

2. Build the body or opaceous body porcelain around the circumference of the coping or framework. Body porcelain which has been modified to have more chroma may be placed on the occlusal or cervical surfaces.

Photo 4 & 5 Body Porcelain Application

Multiple Unit Technique

1. To improve condensation of the porcelain in the hard-to-reach areas on bridges, apply and condense the porcelain at the cervical and interproximal areas before seating the framework on the model.

Note: *Do not allow the porcelain in these areas to dry completely before the rest of the build-up is started.*

2. On bridges where porcelain to tissue contact is required, isolate the tissue area of the stone model with DENTSPLY Prosthetics Die Sealer and lubricate with DENTSPLY Prosthetics Die Release. Alternatively, soak the tissue contact area with water and adapt a piece of single-ply tissue paper or rice paper to the model with a wet brush. Apply body porcelain to the saddle area of the pontic and seat the framework on the model. If the bridge is not seated completely, tap the model gently while lightly pressing down on the unit until the framework is completely and correctly seated. Blot and tap again to remove excess moisture before proceeding with the rest of the build-up.
3. When the porcelain build-up is complete, cut between units at the proximal junction with a thin, sharp blade. This technique allows each unit to shrink to its own center. It also prevents interproximal tearing and voids in these areas.

Incisal Porcelain Application

Note: *Opal Incisal Porcelains... Ceramco II and Ceramco II Silver opal incisal porcelains closely imitate the transmission and reflection of light observed in natural teeth. These opal incisal porcelains scatter more light, reducing grayness in the restoration.*

Ceramco II and Ceramco II Silver opal incisal porcelains may be used alone or blended with conventional incisal porcelains. Separate instructions are available with kits or upon request from Dentsply Ceramco.

- The incisal porcelains may be used alone, mixed with each other, or blended with body porcelain, transparent porcelain or translucent porcelain. Table 5, Suggested Combinations of Incisal Porcelains, lists the

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Ceramco incisal porcelains recommended for different shade prescriptions, while Table 6, Ceramco Incisal Porcelains, describes the eight incisal porcelains, translucent and transparent porcelains, and lists some suggestions for custom shades and special effects.

Note: *Ceramco II and Ceramco II Silver body porcelains are very translucent. Except for special effects, do not use thick layers of incisal porcelains. Excessive amounts of incisal porcelain on a restoration may result in final shades which are low in chroma and appear gray.*

Cutback Technique

1. After building the body porcelain to full contour, cut away the incisal or occlusal tips of the body porcelain build-up with a thin blade. Make tapering beveled cuts in the labial of buccal surfaces. The length and depth of the cut and the sharpness of the angle of the cut will affect the degree of translucency and contrast between incisal and body colors.

Note: *Do not make cutbacks too deep. Final shades will appear low in chroma or too gray if the incisal layer is too thick.*

2. The incisal porcelain or the blended incisal mixture is then applied over this area. Using several incisal porcelains or layering one incisal porcelain next to another creates a more natural color.

Integrated Build-Up Technique

- With the integrated build-up technique, allowance for the incisal porcelain layer is taken into consideration during the body application.
- The desired incisal porcelain or incisal blend is applied and condensed as part of the complete build-up.

Discrete Vertical Layering

- Layer the selected incisal porcelains in narrow vertical columns across the facial-incisal area to simulate the incisal rod formations found in natural teeth.
- Use incisal porcelains of different colors or

translucency levels arranged in discrete parallel columns to create contrast.

- For example, pink incisal, white incisal, and transparent porcelains can be applied in narrow rods in a repeated motif across the incisal third of an anterior build-up.
- A thin layer of translucent or transparent porcelain may be applied over the area and lightly brushed toward the cervical area to complete the effect.

Blended Incisal Porcelain

- The selected incisal porcelain may be blended with the body porcelain shade to provide intermediate levels of translucency or to create a gradual, subtle shift from dentin color to enamel translucency at the incisal blend area of a build-up.
- A starting ratio could be 2:1 or 3:1 parts incisal porcelain to parts body porcelain.

Photo 6 Incisal Application

Completing the Build-Up

1. Use a large, soft brush to smooth the incisal application over the body porcelain build-up. However, it is not necessary to sweep the incisal porcelain down over the entire surface unless a custom shade requires extra translucency in the cervical area.
2. When the porcelain build-up is complete, remove the case from the model. If the mesial and distal contact areas are dry, lightly moisten them with a damp brush; then add porcelain to the contact areas. This porcelain addition should be carefully condensed to ensure that no color shift occurs between the contact area and the basic build-up.

Drying the Build-Up

1. Dry the build-up on firing pegs away from direct heat for at least five minutes before bringing it close to the open furnace entrance. Increase the drying time to ten minutes for bridges or splints.

2. Fire the porcelain, using the porcelain firing schedule as a guideline. Properly fired Ceramco porcelain will exhibit a shiny but granular surface texture (See Table 7).

Photo 7
First Body and Incisal Application

Initial Contouring

- After firing, grind the excess porcelain that prevents correct seating on the model. Use fine-grit, non-contaminating stones, discs, or carbide and diamond burs.
- Grind the case to achieve the proper occlusion, carving and accentuating the correct anatomical contours and details.
- If no porcelain additions are needed, the case may be glazed after thorough cleaning.
- If additions are required, clean the case by lightly blasting with non-recycled 50 micron aluminum oxide; then with a steam cleaner or ultrasonic bath using distilled water.

Note: Caution must be exercised to prevent contamination of the grinding medium with silver-containing alloys to avoid discoloration in subsequent firing.

Second Application of Body and Incisal Porcelain

- Apply the additional porcelain using the same steps and techniques described for the initial application.
- Care should be taken to match the layering of the incisal porcelain over the body porcelain.
- Second applications of Ceramco II and Ceramco II Silver porcelains require the same care in drying and firing as the first layer.

Final Contouring

After all contouring and anatomical adjustments have been ground, clean the case thoroughly with distilled water in steam or ultrasonic cleaning equipment.

Stain and Glazing

- Make sure the case is scrupulously clean and dry before attempting staining, applied or natural glaze steps.
- Use a natural glaze cycle or apply Ceramco II Low-Temp Glaze or Ceramco II Silver Overglaze. (Consult the firing schedules in Table 7.)
- Ceramco FineGrain™ stains work well with either technique and provide the perfect color complement to Ceramco porcelains. For details, consult the Ceramco FineGrain™ stain manual.
- If desired, polish the porcelain before or after glazing steps. Porcelain polishing wheels, diamond polishing pastes, felt wheels, or very fine-grit pumice compounds can be used for this technique.

Photo 8 & 9
Complete single unit crown and 3 unit bridge.

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TABLE 1 Opaque and Body Porcelain Chroma Modifiers

Modifier	Effect
Red-Brown	Increases chroma while decreasing value
Red-Yellow	Increases chroma while decreasing value
Gray-Brown	Increases chroma while decreasing value
High-Value Red-Brown	Increases chroma
High-Value Red-Yellow	Increases chroma
Neutral	Decreases chroma; increases value if used in large proportion

TABLE 2 Related Shade Guide Designations for Chroma Modifiers

Modifier	Suggested For Shade Designations
Red-Brown	A1, A2, A3, A3.5, A4, D2, D4
Red-Yellow	B1, B2, B3, B4, D3
Gray-Brown	C1, C2, C3, C4
High-Value Red-Brown	A1, A2, A3, A3.5, A4, D2, D4
High-Value Red-Yellow	B1, B2, B3, B4, D3
Neutral	All

TABLE 3 Opaque Hue Modifiers

Color	Suggested Uses
Yellow	<ul style="list-style-type: none"> Increase pure yellow hue in overall shade. Accentuate dentinal areas for shades which have a dominant yellow hue.
Pink	<ul style="list-style-type: none"> Increase pink chroma.
White	<ul style="list-style-type: none"> Increase value, decrease chroma. Create internal decalcification areas. Create internal horizontal or vertical striations.
Orange	<ul style="list-style-type: none"> Create interproximal, cervical or occlusal color accents. Increase red-yellow tones with a high value.
Violet	<ul style="list-style-type: none"> Compensate for "headlight effects" on certain types of anterior restorations. Subtly reduce a dominant yellow hue. Internalize incisal effects.
Brown	<ul style="list-style-type: none"> Create "root" color on cases where gingival recession required characterized anatomy and color. Intensify occlusal or interproximal accents. Reduce brightness without losing "warmth."

TABLE 4 Body Hue Modifiers

Color	Suggested Uses
Yellow	<ul style="list-style-type: none"> Create random dental color accents. Increase dominant yellow hue.
Pink	<ul style="list-style-type: none"> Accent mamelon effects. Create subtle increase in brightness. Create incisal effects or accents.
White	<ul style="list-style-type: none"> Increase value. Dilute chroma without increasing translucency. Create decalcification, or horizontal and vertical striations. Achieve posterior occlusal effects.
Orange	<ul style="list-style-type: none"> Create random dental color accents, and interproximal, cervical and occlusal color accents. Create incisal effects. Achieve subtle increase in dominant red-yellow hue.
Blue	<ul style="list-style-type: none"> Create illusion of translucency. Create incisal effects or accents. Achieve subtle decrease of value in shades with a dominant red-yellow hue.
Gray	<ul style="list-style-type: none"> Decrease value, reduce brightness. Create interproximal accents. Achieve incisal effects.

TABLE 5
Suggested Combinations of Ceramco Body and Incisal Porcelains

Body Shade	Suggested Incisal	Accent Incisal
A1	Extra Light	Light, Pink
A2	Extra Light	Light, Pink
A3	Light	Orange, Medium
A3.5	Medium	Orange
A4	Medium	Orange, Dark
B1	Extra Light	Light, White
B2	Extra Light	Light, White
B3	Light	Medium, Yellow
B4	Light	Medium, Yellow
C1	Extra Light	Light, White
C2	Light	Orange, Medium
C3	Medium	Orange
C4	Dark	Medium, Orange
D2	White	Pink, Light
D3	Extra Light	Pink, Light
D4	Light	Orange, Pink

TABLE 6
Ceramco Incisal Porcelains

Note: Opal incisals are also available and are detailed in a specific instruction manual.

Extra Light: A very translucent incisal porcelain used for light, bright shades. May be applied to contact areas to allow color transmission from adjacent teeth, or used in incisal layering to create incisal rod formations.

Light: A translucent, very light-gray incisal porcelain used for light shades, contact areas and to accent incisal rod formations.

White: A bright, high value incisal porcelain with less translucency than the other incisals. May be used as an accent in vertical layering of incisal porcelains or for posterior restorations where gray cusps are undesirable.

Pink: A translucent, low-chroma pink incisal porcelain used for light shades, especially those shades in the red-brown or red-gray incisal category. The subtle pink tones can be used in vertical layering to accent incisal rod formations.

Medium: A translucent, light-gray incisal porcelain used for medium chroma shades. May also be used on lighter shades to increase the illusion of translucency when facial or incisal space is limited.

Orange: A translucent, light orange-gray incisal porcelain used for medium or deep chroma shades. Enhances the red-yellow tones of some shades, and can be used as an accent color in vertical incisal layering techniques.

Yellow: A translucent, low-chroma yellow incisal porcelain that increases the brightness of an incisal blend without losing translucency. Used to enhance shades with a dominant yellow-red hue.

Dark: A translucent, dark-gray incisal porcelain for deep chroma shades. Can be used to accent incisal rod formations in vertical incisal layering techniques, or create dark horizontal barring effects or light amalgam staining effects.

Translucent: A translucent porcelain with no shading oxides. Ideal for overlays on body and incisal porcelains, particularly when layering of color has been used.

Transparent: A clear porcelain used to create clear incisal edges, accent mamelon formation or vertical incisal layering techniques, or as a facial or occlusal overlay.

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TABLE 7 - Firing Guidelines for Products

Note: You may prefer to alter these programs to meet individual requirements or circumstances. Visual results are the most important criteria for judging the adjustments of a firing program to your individual needs.

Firing Programs in °F		Time (min)			Time (min)			Vacuum		Set Temp		Temperature			
Prog #	Program Description	Dry	Pre Heat	Vac Hold	Hi Temp Hold	Hold	Cool	Set Point (in.Hg)	Idle	Hi Temp	Vac Start	Vac Stop	Heat Rate (°F/min)	Night	
01	Ultra Pake®	3	3	0	1	0	0	29	932	1787	932	1787	306	200	
02	Paint-O-Pake®	3	3	0	0	0	0	29	1202	1778	1202	1742	126	200	
04	Ceramco® II Silver Body	5	5	0	0	0	0	29	1202	1760	1202	1724	99	200	
05	Ceramco® II Body	5	5	0	0	0	0	29	1202	1724	1202	1688	126	200	
06	CII Silver Natural Glaze	3	3	0	1	0	0	0	1202	1760			99	200	
07	CII Natural Glaze	3	3	0	1	0	0	0	1202	1724			126	200	
08	CII Silver Add-On	5	5	0	0	0	0	29	1202	1724	1202	1634	126	200	
09	CII Add-On	5	5	0	0	0	0	29	1202	1679	1202	1589	126	200	
10	CII Silver Overglaze	3	3	0	1	0	0	0	1202	1715			126	200	
11	Low Temp. Glaze	3	3	0	1	0	0	0	1202	1679			126	200	

Firing Programs in °C		Time (min)			Time (min)			Vacuum		Set Temp		Temperature			
Prog #	Program Description	Dry	Pre Heat	Vac Hold	Hi Temp Hold	Hold	Cool	Set Point (in.Hg)	Idle	Hi Temp	Vac Start	Vac Stop	Heat Rate (°C/min)	Night	
01	Ultra Pake®	3	3	0	1	0	0	29	500	975	500	975	200	95	
02	Paint-O-Pake®	3	3	0	0	0	0	29	650	970	650	950	70	95	
04	Ceramco® II Silver Body	5	5	0	0	0	0	29	650	960	650	940	55	95	
05	Ceramco® II Body	5	5	0	0	0	0	29	650	940	650	920	70	95	
06	CII Silver Natural Glaze	3	3	0	1	0	0	0	650	960			55	95	
07	CII Natural Glaze	3	3	0	1	0	0	0	650	940			70	95	
08	CII Silver Add-On	5	5	0	0	0	0	29	650	940	650	890	70	95	
09	CII Add-On	5	5	0	0	0	0	29	650	915	650	865	70	95	
10	CII Silver Overglaze	3	3	0	1	0	0	0	650	935			70	95	
11	Low Temp. Glaze	3	3	0	1	0	0	0	650	915			70	95	

The suggested programs for Ceramco products were developed as guidelines with the Centurion™ QEX furnace. **These programs differ from previously published guidelines. Where applicable, programs were unified and simplified.** The new firing schedules were thoroughly tested and found to yield reliable, esthetic results.

Visual Indicators:

Personal preferences, techniques and case requirements differ and recommended or preprogrammed firing schedules are only guidelines. **Visual results are the most important criteria** for judging the adjustment of a firing program to your individual needs. The following visual indicators are presented as a guide:

Firing Step	Indicator (desired appearance)
Ultra Pake®	Rough, reflective surface
First Powder Opaque	Slight sheen/not glazed
Second Powder Opaque	Dull sheen, slight matte finish
Body/Incisal	Shiny appearance, neither milk nor glazed
Add-On Porcelain	Smooth, satiny gloss*
Natural Glaze	Satiny gloss*
Low Temp Glaze/Overglaze	High gloss*

*Degree of gloss may be higher or lower based upon individual preferences.

If the results achieved with a program do not conform to your needs, the following guidelines will help you in making adjustments:

Program adjustment guidelines:

Result	Possible Cause	Solutions
White spots/ chalkiness	First bake body temp too low	Increase high temp ¹ Decrease heat rate ² Increase hold time ³
	Dried too rapidly	Increase dry time ⁴
	Insufficient vacuum	Increase vac stop temp ⁵ Check hoses for leaks
Value too low	Overfired body	Decrease high temp ⁶ Increase heat rate ⁷
Chroma too low	Underfired body	Increase high temp ¹ Decrease heat rate ²
	Body dried too rapidly	Increase dry time prior to starting program ⁴
Chroma too high	Overfired body	Decrease high temp ⁶

Footnotes:

1. If the discrepancy with visual indicator is extreme, increase the high temp 15°C (27°F). If the discrepancy is small, increase the high temp by 5°C (9°F).
2. Decrease the heat rate by 15°C/min (27°F/min).
3. Increase the hold time in 0.5 min increments.
4. Increase the dry time in 2 min increments.
5. Increase the vac stop temp by 10°C (18°F)
6. If the discrepancy with the visual indicator is extreme, decrease the high temp 15°C (27°F). If the discrepancy is small decrease the high temp by 5°C (9°F).
7. Increase the heat rate in 5°C/min (9°F/min) increments.

¹ Vita Lumin is a registered trademark of Vita Zahnfabrik A.G.

If you have further questions or need assistance, please call our technical staff at 800-487-0100. (Outside the USA and Canada, please call your distributor.)

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