SureSmile® Aligner and Retainer

Doctor Instructions For Use

These are custom-made aligners and/or retainers for an individual patient.

- ENG Instructions For Use
- DEU Gebrauchsanweisung
- DNK Brugsanvisning
- ESP Instrucciones de Uso
- FRA Mode d'emploi
- ITA Istruzioni d'uso
- NLD Gebruiksaanwijzing
- NOR Bruksanvisning
- PRT Instruções de utilização
- SWE Bruksanvisning

suresmile.com

For Professional Use Only – Rx Only

Device Description

SureSmile® Aligners are fabricated from clear thin thermoformed plastic in a sequential series to reposition the teeth progressively.

SureSmile® Retainers are fabricated from clear thin thermoformed plastic designed to hold teeth in place.

Note: Product may not be available for sale in your country.

Indications for Use

SureSmile® Aligners are indicated for the treatment of tooth malocclusion in patients with permanent dentition.

Intended Use/Performance Characteristics

SureSmile® Aligners are intended to align teeth into position and treatment of tooth malocclusion in patients with permanent dentition.

SureSmile® Retainers are intended for use in retention and the prevention of relapse at the completion of orthodontic treatment.

Clinical Benefits

Clinical benefits of orthodontic treatment include overall improvement of tooth health and correction of malocclusion.

Contraindications

SureSmile® Aligners and Retainers are contraindicated for use in patients with active periodontal disease.

Warnings

- In rare instances, some patients may be allergic to plastic. Patients with a history of allergic reactions to plastics should not use this product.
- Do not use damaged or broken aligners or retainers, as their parts can be accidentally swallowed or aspirated, and may be harmful.
- Report serious incidents that have occurred in relation to the device to device manufacturer and the Competent Authority in Member State of residence, or other applicable regulatory agency.

Precautions for Doctors

- Do not sterilize SureSmile® Aligners or Retainers.
- Treatment with SureSmile® Aligners is an orthodontic therapy and you should consider all potential side effects associated with such therapy.
- Orthodontic treatment may shorten the teeth roots. This may reduce the longevity of the teeth.
- The health of the bone and gums supporting the teeth may be adversely affected.
- When evaluating the use of interproximal reduction to create space for tooth movement, you should consider all associated risks.
- Joint pain, headaches, or ear problems may result due to temporomandibular joint issues. This rarely occurs.

- Certain medical conditions or medications can affect orthodontic treatment outcomes.
- Missing, erupting or unusually shaped teeth may affect treatment outcomes.
- You may need to adjust the patient's bite at the end of treatment.
- Existing dental restorations such as crowns or bridges, may require recementation or, in some cases, replacement.
- Aligners are not effective for moving dental implants.
- To help prevent supra-eruption, the aligner should cover all teeth.
- You may need to use skeletal expanders, bonded auxiliaries, elastics and other fixed and/or removable orthodontic appliances for more complicated treatment plans where aligners alone may not be adequate to achieve the desired outcome.
- Supplemental treatment may be required for severe open bite/overjet or skeletally narrow jaw.
- To correct severe crowding or jaw imbalances, oral surgery could be required. If so, consider risks associated with surgery before starting aligner treatment.
- You may need to bond attachments temporarily to one or more teeth during treatment to facilitate aligner retention or tooth movement.
- Replace attachments if they become loose.
- Remove attachments at the end of treatment.

Precautions for Patients

- The patient may experience tenderness or temporary discomfort at initial aligner or retainer placement and after switching to each new aligner.
- The aligner and retainer may initially affect speech and may result in a lisp, usually disappearing within two weeks.
- The patient may experience a temporary increase in salivation or dryness of mouth.
- The aligners and retainer may scratch or irritate the patient's lips, cheeks or gums.
- The patient should not wear aligners when eating food, and when drinking all liquids except cold water. The patient must not chew gum, candy or ice while wearing aligners or retainers.
- If patients do not brush and floss correctly during treatment, or if they consume sugary foods or beverages while wearing aligners, they may be subject to tooth decay, periodontal disease and permanent markings from decalcification and staining.
- Lack of compliance in wearing the aligners and retainers for the prescribed number of hours per day or not using them as directed can increase treatment time and affect desired results.
- The patient's teeth may shift position after aligner treatment. To help prevent this, the patient should wear retainers.

Adverse Reactions

Allergic reactions, to plastics.

How-To Instructions

- Rinse new aligners or retainers with cold water before inserting in the patient's mouth.
- When an extraction is part of treatment, retain the space until the area heals. Add pontics to the aligners, if applicable.
- Seat aligners or retainers on the patient's teeth and check for soft tissue impingement. Trim the aligners or retainers as needed. Polish them after trimming.
- Complete any attachment placement and perform any necessary interproximal reduction (IPR) procedures according to the treatment plan.
- Review the SureSmile® Aligner and Retainer Patient Instructions with the patient and go over the following points:
 - Each aligner and retainer is engraved with the first four letters of the patient's name, a "U" for upper teeth or an "L" for lower teeth, followed by the stage number or RET for retainers.
 - Aligners should be worn 20 to 22 hours daily. Each set of aligners should be replaced after approximately 2 weeks of use.
- Dispense the next sets of aligners. Make sure you distribute enough aligners to last until the next appointment.
- At each appointment, check teeth for tight interproximal contacts at the appropriate aligner stage and check the fit of the aligners.
- If the patient needs additional aligners, you may need to take a new scan.
- Recommend that the patient wears retainers after treatment.

Daily Care and Maintenance of Aligners and Retainers

- Patients should clean their aligners and retainers before each insertion with a dental appliance cleaner.
- Patients should NOT use toothpaste or soak aligners or retainers in mouthwash, as this can scratch and dull the plastic, adversely affecting the product's clarity.

Proper Oral Hygiene

- Patients should brush and floss their teeth after each meal or snack before reinserting their aligners or retainers.
- We recommend regular dental checkups and cleaning for the continued health of a patient's teeth and gums.

If you have additional questions after reading these instructions, please contact Customer Service.

Symbol Glossary

Symbol	Title of Symbol	Description of Symbol
	Consult Instructions for Use	Indicates the need for the user to consult the instructions for use.
	Caution	Indicates the need for the user to consult the Instructions for Use for important cautionary information such as warnings and precautions.
	Manufacturer	Indicates the medical device manufacturer.
	Authorized Representative in the European Community	Indicates the authorized representative in the European Community
	Prescription Use	Caution - US Federal law restricts this device to sale by or on an order of a dentist.
	Single patient - multiple use	When a medical device is intended by the manufacturer for reuse on a single patient.
	Medical device	Indicates that the product is a medical device
	Serial number	Indicates the manufacturer's serial number so that a specific medical device can be identified.

 **Manufacturer:**
OraMetrix
 2350 Campbell Creek Blvd.
 Suite 400
 Richardson, Texas 75082 USA

Authorized EU Representative
OraMetrix GmbH
 Rungestr. 19
 10179 Berlin Germany

Australian Sponsor
Dentsply Sirona Pty Ltd
 11-21 Gilby Road
 Mount Waverley VIC 3149
 Australia
 1300 55 29 29

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©2021 Dentsply Sirona