# SureSmile® Aligner and Retainer

## Patient Instructions For Use

Your SureSmile® Aligners and Retainers were custom made for you based on your doctor's prescription. Please read these directions carefully before use.

|     |                       |     |                          |
|-----|-----------------------|-----|--------------------------|
| ENG | Instructions For Use  | JPN | アライナーおよびリテーナー            |
| DEU | Gebrauchsanweisung    | KOR | 사용 설명서                   |
| DNK | Brugsanvisning        | NLD | Gebruiksaanwijzing       |
| ESP | Instrucciones de Uso  | NOR | Bruksanvisning           |
| FIN | Käyttöohjeet          | POL | Instrukcja obsługi       |
| FRA | Mode d'emploi         | PRT | Instruções de utilização |
| ISL | Notkunarleiðbeiningar | RUS | Инструкция по применению |
| ITA | Istruzioni per l'uso  | SWE | Bruksanvisning           |

## Device Description

SureSmile® Aligners are fabricated from clear thin thermoformed plastic in a sequential series to reposition the teeth progressively.

SureSmile® Retainers are fabricated from clear thin thermoformed plastic designed to hold teeth in place.

Note: Product may not be available for sale in your country.

## Indications for Use

SureSmile® Aligners are indicated for the treatment of tooth malocclusion in patients with permanent dentition.

## Intended Use/Performance Characteristics

SureSmile® Aligners are intended to align teeth into position for the treatment of tooth malocclusion in patients with permanent dentition as prescribed by your doctor.

SureSmile® Retainers are intended for use in retention and the prevention of relapse at the completion of orthodontic treatment.

## Clinical Benefits

Clinical benefits of orthodontic treatment include overall improvement of tooth health and correction of malocclusion.

## Contraindications

SureSmile® Aligners and Retainers are contraindicated for use in patients with active periodontal disease.

## Warnings

- In rare instances, some patients may be allergic to plastic. If you have a history of allergic reactions to plastics you should not use this product. If an allergic reaction occurs while wearing your aligners or retainers, discontinue use and consult your doctor immediately.
- Do not use damaged or broken aligners or retainers, as their parts can be accidentally swallowed or aspirated, and may be harmful.
- If your aligners, retainers or an attachment is lost or broken, immediately contact your doctor.
- Call your doctor immediately if you lose or break your retainers. Too much time without retainers will allow your teeth to move.
- Report serious incidents that have occurred in relation to the device to device manufacturer and the Competent Authority in Member State of residence, or other applicable regulatory agency.

## Precautions

- Do not chew gum, candy, or ice while wearing the device.
- You may experience tenderness or temporary discomfort at initial aligner or retainer placement and after switching to each new aligner.
- The aligner and retainer may initially affect speech and may result in a lisp, usually disappearing within two weeks.
- You may experience a temporary increase in salivation or dryness of mouth.

- The aligners and retainers may scratch or irritate your lips, cheeks or gums. If this occurs, contact your doctor.
- Lack of compliance in wearing the aligners and retainers for the prescribed number of hours per day or not using them as directed can increase treatment time and affect desired results.
- Your teeth may shift position after aligner treatment. To help prevent this, you should wear retainers.
- If your aligner and retainer are not fitting well in your mouth, it may need to be adjusted. Contact your doctor.

## Adverse Reactions

Allergic reactions, to plastics.

## How-To Instructions


### Cleaning Your Aligners/Retainers

Clean your aligners and retainers before you insert them.

- Use the dental appliance cleaner recommended by your doctor. Be sure to follow the instructions of the dental appliance cleaner. Retainer Brite® products are recommended to clean your SureSmile® aligners and retainers.
- You may use a soft bristle toothbrush and water. You should NOT use toothpaste or soak your aligners and retainers in mouthwash as this can scratch or dull the plastic, adversely affecting the clarity. Do NOT use denture cleaners.
- Rinse each aligner and retainer thoroughly with water after each cleaning. Shake off excess water and then store your aligners and retainers in a protective case.

### Inserting Your Aligners and Retainers

- Wear your aligners per your doctor's instructions, usually 20-22 hours daily (includes sleeping).
- Wear your retainers as indicated by your doctor. Not wearing your retainers as prescribed may allow your teeth to move back into the old position.
- Handle your aligners and retainers with care. Handle ONE aligner or retainer at a time when inserting, removing or cleaning.
- Wash your hands thoroughly with soap and water before handling your aligners and retainers.
- Clean your aligners and retainers before you insert them.
- Make sure you have the correct stage aligner. Each aligner and retainer is engraved with the first four letters of your name, a "U" for upper teeth or an "L" for lower teeth, followed by the stage number or RET for retainers.
- Insert only one aligner or retainer at a time. Trying to seat two at once, or using excessive pressure, may damage the device.
- You may insert either the upper or the lower aligner or retainer first. When inserting each aligner or retainer, gently push it over your front teeth. Next, apply equal pressure with your fingertips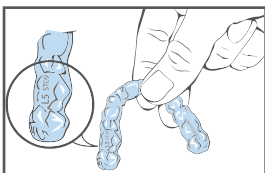


to the tops of your left and right molars (big teeth at the back) until the aligner or retainer snaps into place.

- If necessary to fully seat your aligners and retainers, insert a Chewies™ Tray Seater into your mouth and position it between your upper and lower arches.
- Gently bite down on the Chewies™ Tray Seater and work your aligner or retainer into place until fully seated.
- When not using Chewies™, do not bite your aligners or retainers into position as this may damage them.
- Minor discrepancies between the new aligner and the current tooth position are normal since the teeth need time to conform to the new aligner position. If you experience sharp pain or significant discomfort, remove the aligners and contact your doctor immediately. For retainers, if you experience sharp pain or significant discomfort, remove the retainer and contact your doctor immediately.

## Removing Your Aligners and Retainers


- Remove your aligners and retainers when eating all snacks and meals, and when drinking all liquids except for cool water.

- For removal, on one side of your mouth, use your fingertip on the inside of your back molar and slowly pull the aligner or retainer from your molars.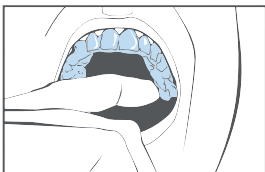


- Repeat this process on the other side of your mouth before trying to remove the aligner or retainer completely.

- If resistance is felt, it may be necessary to repeat the previous steps in several locations along your arch. Do not bend, twist or use excessive force to remove your aligner or retainer as this may damage it.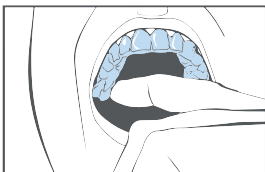

- Once the aligner or retainer is disengaged from the molars on both sides of your mouth, you should be able to work your way slowly forward, gently prying the aligner or retainer away from your teeth with your fingertips.

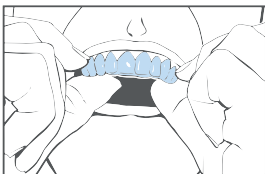### **Remember!**

- Be careful when removing your aligners, especially if there are multiple attachments.
- DO NOT use sharp objects to remove your aligners or retainers.
- Consult with your doctor if your aligners or retainers are extremely difficult to remove.
- To help prevent damage to your aligners or retainers, avoid unnecessary removal.

## Storing and Changing Your Aligners and Retainers

- To help protect your aligners and retainers from loss or damage, store them in a case when they are not in your mouth.
- When it is time for your next aligner stage, clean and place the aligners that you have just finished wearing in a safe location.
  - If your “current stage” aligner is lost or broken, your doctor may recommend that you temporarily use your previous aligners while a replacement for the “current stage” is made.
- Do not store aligners and retainers near a heat source. This may cause them to distort, resulting in improper fit in your mouth.

## Oral Hygiene

During treatment it is important to maintain good oral hygiene.

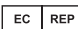
- Trapping food in your aligners and retainers during treatment can lead to tooth decay.
- You should brush and floss your teeth before inserting your aligners or retainers.
- We recommend regular dental checkups and cleaning for the continued health of your teeth and gums.

## Safe Disposal

Used aligners and retainers may be discarded in regular household trash, as you would a toothbrush or used dental floss.

**If you have additional questions after reading these directions, please consult your doctor.**

## Symbol Glossary

| Symbol  | Title of Symbol                                     | Description of Symbol  |
|---|---|--|
|  | Consult Instructions for Use                        | Indicates the need for the user to consult the instructions for use.   |
|  | Caution   | Indicates the need for the user to consult the Instructions for Use for important cautionary information such as warnings and precautions. |
|  | Manufacturer  | Indicates the medical device manufacturer.   |
|  | Authorized Representative in the European Community | Indicates the authorized representative in the European Community  |
|  | Prescription Use                                    | Caution - US Federal law restricts this device to sale by or on an order of a dentist.   |
|  | Single patient - multiple use                       | When a medical device is intended by the manufacturer for reuse on a single patient.   |
|  | Medical device                                      | Indicates that the product is a medical device   |
|  | Serial number                                       | Indicates the manufacturer's serial number so that a specific medical device can be identified.  |

 Manufacturer:  
**OraMetrix**  
 2350 Campbell Creek Blvd.  
 Suite 400  
 Richardson, Texas 75082 USA

**Authorized EU Representative**  
**OraMetrix GmbH**  
 Rungestr. 19  
 10179 Berlin Germany

**Australian Sponsor**  
**Dentsply Sirona Pty Ltd**  
 11-21 Gilby Road  
 Mount Waverley VIC 3149  
 Australia  
 1300 55 29 29

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©2021 Dentsply Sirona