Service Pack CEREC Ortho SW 1.1.2

Update Description (EN)

Installation notes:

- Perform a backup of all patient data before starting the installation process.
- Do not perform an installation on several CEREC units or computers at the same time if these are within the same network.
- Before starting the installation process, please check the patient database directory in the software (Configuration/Settings/Patient Database). The path should always point to a directory, never to the root of a drive, i. e. never to “D:\”, but rather to “D:\Data”.
- If you are running a CEREC Premium SW 4.4.x and an inLab SW 15.1 software on your PC, please make sure that both products are installed in different directories. This can be achieved by performing custom installations of the products.

Changes:

- New EXPORT button for MTM Clear Aligners and Dentsply Ortho Lab (USA)
- Quality control agreement: With the user's permission, anonymous usage data is logged to help us improve workflows as well as software performance and quality.
- The “Border Line Warning” no longer appears.
- Fix for the double-arrow bug in ACQUISITION which allowed the bite registration to be skipped
- Reduced export file size for model exports supporting color (e.g. Dolphin Imaging)
- Fix for double-arrow bug in EXPORT which occurred when scans were requested from Dolphin
- Additional languages French, Italian, Japanese, Chinese (Simplified & Taiwan), Portuguese (BR), Russian, Korean
- Adjusted internal data format for ClearCorrect exports to expedite the modeling process before treatment planning
- Connect Center is installed with the software.

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