

CEREC SW 5.1.2

UPDATE DESCRIPTION MATERIAL PACK

The update “Material Pack CEREC SW 5.1.2” requires a pre-installed version CEREC SW 5.1.2.

Integration of new materials (compared to CEREC SW 5.1.2):


Material releases for CEREC Primemill
Ivoclar Vivadent Telio CAD
Ivoclar Vivadent Telio CAD Abutment
Merz artBloc Temp
Vita CAD Temp monoColor
Vita CAD Temp multiColor
Vita CAD Temp IS

Please observe regional restrictions regarding TiBase and materials.
Refer to the document, “Combinations of titanium bases and materials in the CEREC software 5.1.2” at my.cerec.com.

Installation & License

■ Automatic update:

A pop-up message informing about the new update shows up. Start the update process directly by pressing “Download” or start the process later. If you choose to start the update process later, you must go to the Windows taskbar. Open the Dentsply Sirona AutoUpdate Center by clicking on the corresponding icon.

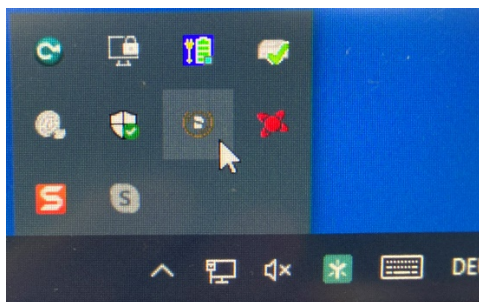

The CEREC SW 5.1.2 Material Pack is listed in the update list of the Dentsply Sirona AutoUpdate Center. The download of the service pack can be started and paused and continued if necessary. After the download is complete, the installation of the Material Pack can be started.

- Perform a data backup prior to installation.
- Before beginning the installation, check in the database settings (CEREC SW: Configuration\Settings\Patient Database), that the patient database is saved in its own directory – not in the top level of a drive (e.g., “D:\Data” instead of “D:\”).

- Do not perform installation on multiple CEREC units or workstations at the same time if they are connected via a network.
- The use of the correct graphic card driver is necessary for ensuring an optimal functionality of your CEREC Software 5.1.2. At the startup of your software, it will automatically check whether your driver is correct. Should you receive a notification, please contact customer service.
- No additional license is needed to update from a pre-installed CEREC SW 5.1.x to CEREC SW 5.1.2 Material Pack.

All CEREC SW downloads can be found on:

<https://www.dentsplysirona.com/en/explore/cerec/downloads.html>

my.cerec.com