

# inLab Profire sinter furnace

## Procedure for installing new sinter programs:

### Note:

The Update-File for sinter programs is suitable for all furnace versions of the inLab Profire series. Before installing the update, we recommend creating backup file of the furnace data on a USB stick. Please follow the instructions in the user manual /Section 5.6 "Data transfer via USB stick" and select the option "Backup data".



You have downloaded an Update-File for new sinter programs for the inLab Profire sinter furnace from the Internet site <https://www.dentsplysirona.com/inlab-downloads>.

### The update provides the following:

- Two sinter programs for the material e.max ZirCAD from Ivoclar Vivadent

### To install the new sinter programs, please proceed as follows:

Copy the files **heating\_files** contained in the folder "LAB-software-download-inLab-profire-ivoclar.zip" to the root directory of a USB stick.

### Note:

Please make sure that the USB stick (at least USB 2.0) is formatted in FAT32, that there are no other files or folders on the USB stick and that the stick is not password protected.

Follow the instructions in the user manual/chapter 5.6 "Data transfer via USB stick" and select the option "Load my programs".

When the installation is complete, a message will appear indicating that the sinter programs has been successfully installed. Please restart the inLab Profire by using the power switch on the back of the furnace. Remove the USB stick and wait approx. 60 seconds before switching on the furnace again until the green LED on the front panel is no longer illuminated.

Please check if the new sinter programs were installed in the fixed program area of the furnace. To do this, tap the "Programs" icon in the main menu, then tap on "Programs" and then tap on "Fixed programs". Now the new sinter programs should be displayed.

If you have any questions, please contact your dealer or the Dentsply Sirona Customer Service.

Date: April 2021