


# ProTaper Gold<sup>®</sup> Rotary files

Efficient, Flexible, Brilliant

+

**STERILE**

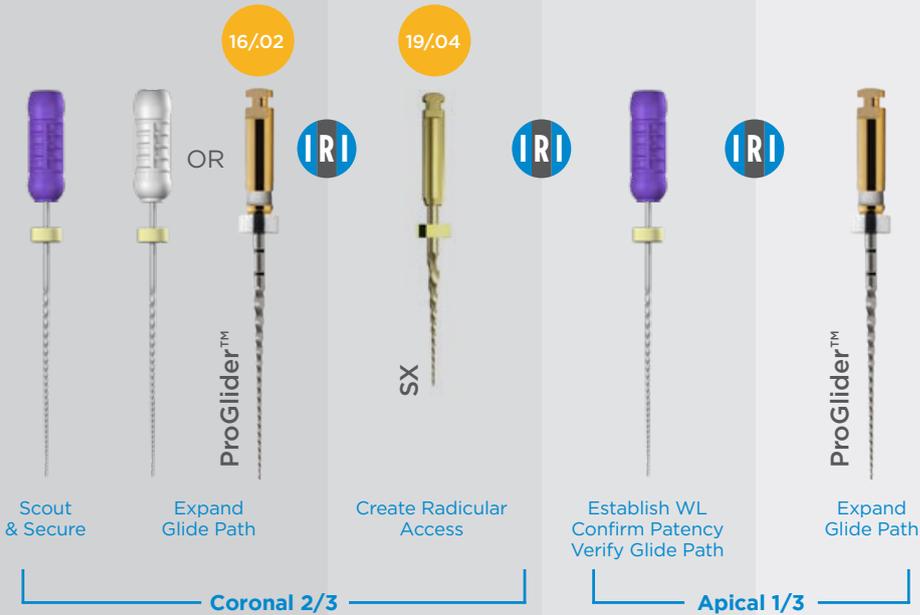

## TREATMENT TECHNIQUE GLIDE PATH MANAGEMENT

- Use ProLube<sup>®</sup> Root Canal Conditioner and small hand files when manually scouting and securing canals
- Use NaOCl when mechanically pre-shaping or shaping canals
- Use ProGlider™ Rotary Glide Path File & ProTaper Gold SX files in one or more passes to expand the glide path
- Use ProTaper Gold SX to relocate the coronal aspect of a canal away from an external root concavity, eliminate triangles of dentin, or produce more shape as desired
- Irrigate, recapitulate with a 10 file and re-irrigate after removing each rotary file
- Frequently clean and inspect the flutes of the rotary files

 **300 RPM / Torque 150-520 g.cm**



 **Dentsply  
Sirona**

## SHAPERS

18/02      20/04



S1

S2

Brush, Follow  
& Shape to Length

## FINISHERS

20/07      25/08



F1

F2

Follow, Brush  
& Finish to Length

Expand Final Shape as Desired

30/09      40/06      50/05



F3

F4

F5

Inspect Apical Flutes for Dentin to Verify Shape

## TREATMENT TECHNIQUE SHAPING AND FINISHING SEQUENCE

- Use ProTaper Gold files in regions of the canal that have a confirmed, smooth and reproducible glide path
- Use ProTaper Gold files in a deliberate brushing motion, on the outstroke, to more optimally prepare canals that exhibit irregular cross-sections and to facilitate apical file progression
- Use any ProTaper Gold file in one or more passes to safely achieve length

Proceed to 3D Disinfection  
and Obturation

Manufactured By:

DENTSPLY International, Inc. 608 Rolling Hills Dr. Johnson City, TN 37604

www.dentsplysirona.com

Phone: 1-800-662-1202



 **Dentsply  
Sirona**