

# CEREC SW 5.2.11

## Update Description (EN)

Dentsply Sirona is pleased to announce the new update CEREC SW 5.2.11. The installation of the "CEREC SW 5.2.11" update does not require a previously installed version. The automatic update to "CEREC SW 5.2.11" is only offered if a CEREC SW 5.2.x is installed.

### Changes made to previous version 5.2.10:

New CEREC Primemill Firmware version 2.11.8

- New water pump sequence in Routine Actions
- Bug fix for problem when setting static IP address

New CEREC SpeedFire firmware version

- Introduction of a sintering mode (fast/esthetic), previously only available for CEREC MTL Zirconia
- Adjustments to the user interface
- Minor bugfixes

New materials

- **Kuraray Noritake** – new block material: KATANA Avencia 2 ML
- **Kuraray Noritake** – new block material: KATANA Avencia 2 HT
- **Kuraray Noritake** – new block material: KATANA Avencia 2 OE (Japan only)
- **Kuraray Noritake** – new block material: KATANA Avencia P (Japan only)
- **SHOFU** – new block material: BLOCK PEEK (Japan only)

Updates for existing materials:

- **Dentsply Sirona** – CEREC MTL Zirconia: new shades & data matrix codes
- **Dentsply Sirona** – CEREC Cercon 4D Abutment: available for USA region
- **VITA** – VITABLOCS Mark II & TriLuxe forte: new shades

Material SpeedFire process updates:

- **Dentsply Sirona** CEREC MTL Zirconia
  - new sintering processes for CEREC MTL Zirconia for bridge indication
  - new esthetic process for CEREC MTL Zirconia
- **Ivoclar** – Faster IPS e.max CAD crystallization


SW Changes:

- Bugfixes for additional software stability.
- Bugfixes to support operating systems and graphics card drivers.

## Installation & License

- Automatic update:

A pop-up message informing about the new update shows up. Start the update process directly by pressing “Download” or start the process later. If you choose to start the update process later, you must go to the Windows taskbar. Open the Dentsply Sirona AutoUpdate Center by clicking on the corresponding icon.

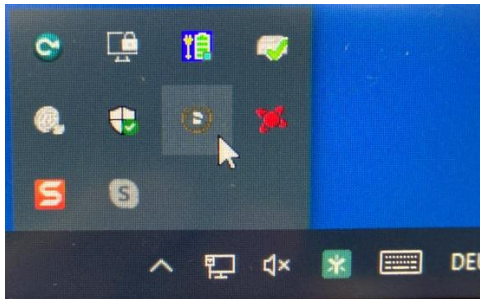

The CEREC SW 5.2.11 Service Pack is listed in the update list of the Dentsply Sirona AutoUpdate Center. The download of the service pack can be started and paused and continued if necessary. After the download is complete, the installation of CEREC SW 5.2.11 can be started.

- Perform a data backup prior to installation.
- Before beginning the installation, check in the database settings (CEREC SW: Configuration\Settings\Patient Database), that the patient database is saved in its own directory – not in the top level of a drive (e.g., “D:\Data” instead of “D:\”).
- Do not perform installation on multiple CEREC units or workstations at the same time if they are connected via a network.
- The use of the correct graphic card driver is necessary for ensuring an optimal functionality of your CEREC Software 5.2.11. At the startup of your software, it will automatically check whether your driver is correct. Should you receive a notification, please contact customer service.
- No additional license is needed to update from a pre-installed CEREC SW 5.2.x to CEREC SW 5.2.11.