

Connect Software 5.2.8

Update Description (EN)



Dentsply Sirona is pleased to announce the new Connect SW 5.2.8 update. The Software will be available from November 13th, 2024 for all Connect 5.x users.

Changes made to previous version 5.2.7:


New Order Workflow: Connect SW 5.2.8 connected with DS Core

Connect Software 5.2.8 integrates with the DS Core cloud platform This enables a seamless data transfer and improved workflow flexibility with DS Core and digital impression acquisition with on premise Connect Software on AC or Laptop hardware:


- Create / open patient in DS Core
- Create treatment in DS Core
- Start DI Scan in DS Core and select the intraoral Scanner

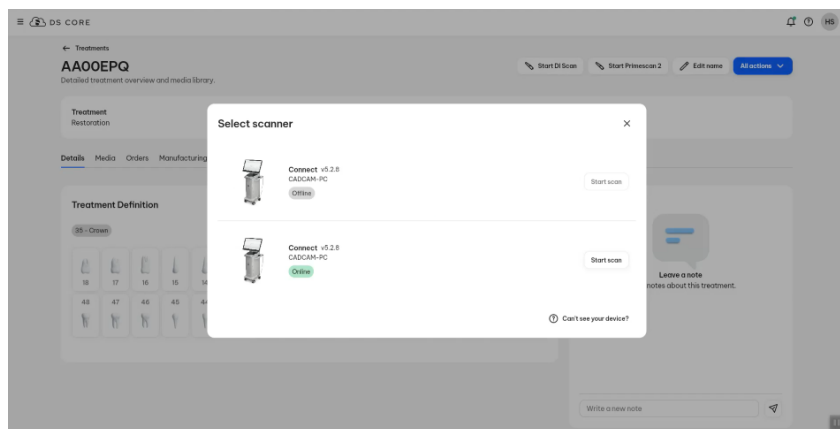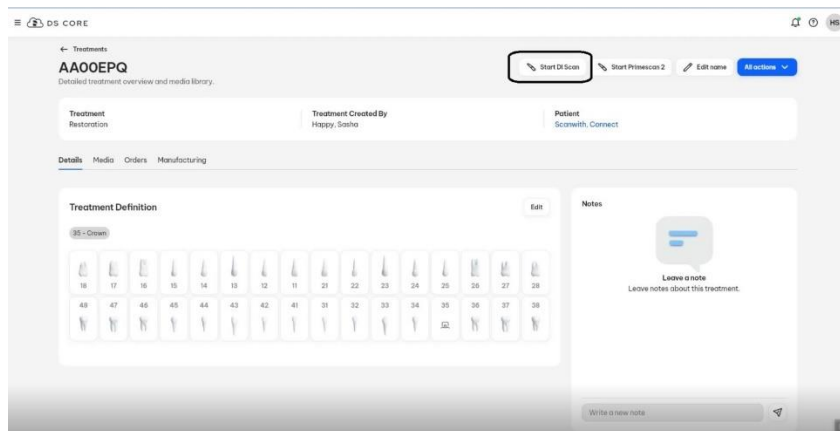


- Case is automatically opened in Connect Software

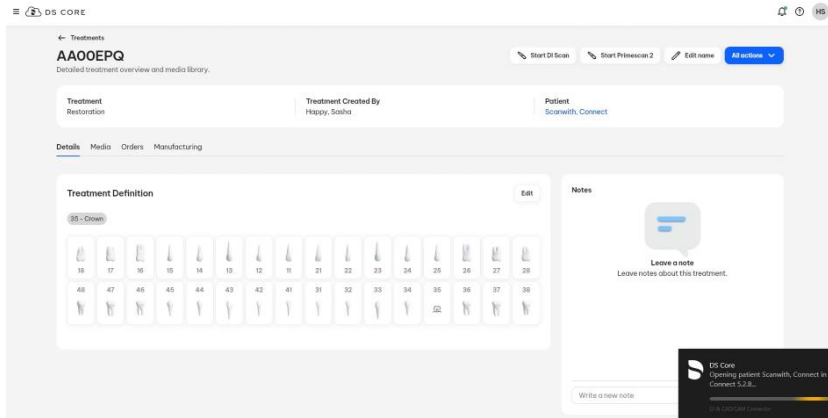

- Start scanning

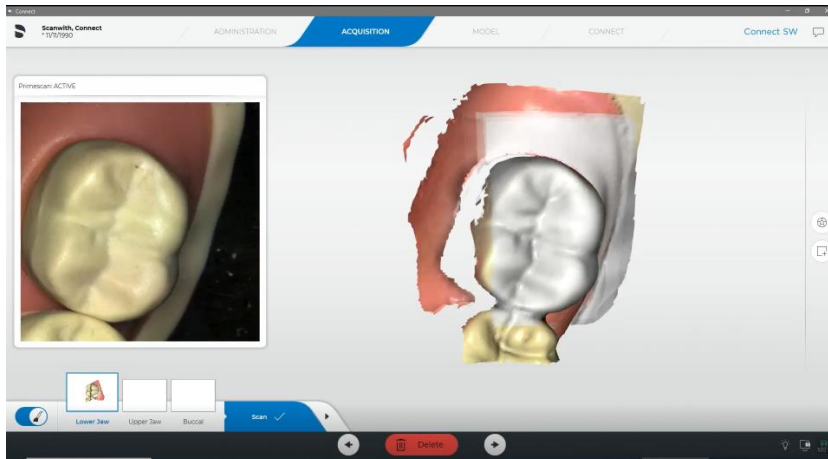

- Uploading scan to DS Core Viewer

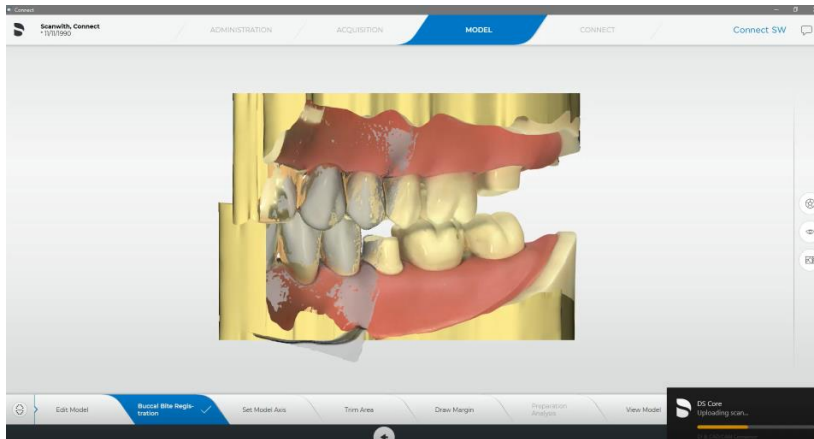

- Use scan model and treatment info to enter DS Core order process




Further changes made to previous version 5.2.7:

- **Manually triggered DS Core upload**
 - Upload of models with automatically calculated or adjusted preparation margin and treatment information
- **Updates for material and TiBase library (Straumann new Gingiva heights for Variobase BLX, ATTX & XiVE for e.max CAD in CE)**
- **Footswitch info only displayed with AC hardware**

Installation notes:

Automatic update: The update notification for Connect SW 5.2 will pop-up automatically. Start the update process directly by pressing “Download” or start the process later. If you choose to start the update process later, you must go to the Windows taskbar.

- Open the Dentsply Sirona AutoUpdate Center by clicking on the corresponding icon: 
- The Connect SW 5.2.7 is listed in the update list of the Dentsply Sirona AutoUpdate Center. After the download is complete, the installation of Connect SW 5.2.7 can be started.
- Perform a backup of all patient data before starting the installation process
- Do not perform an installation on more than one acquisition unit or computer at the same time if these are within the same network
- Before starting the installation process, please check the patient database directory in the software
- (Configuration/Settings/Patient Database). The path should always point to a directory, never to the root of a drive, i. e. never to “D:\”, but rather to “D:\Data”
- The use of the correct graphic card driver is necessary for ensuring an optimal functionality of your Connect Software 5.2.7. At the startup of your software, it will automatically check whether your driver is correct. Should you receive a notification, please contact customer service.

Hardware support note:

Connect SW 5.2.7 supports Omnicam (with guided scanning) and Primescan (no guided scan required) scanners but requires the newer AC models with touchscreen and Windows 10 to run. Pre-2019 Omnicam units (without touchscreen) will require a Windows 10 update in order to use this software. For more information on upgrading these devices, please contact your dealer representative

All Connect SW downloads can be found here:

- <https://www.dentsplysirona.com/en/explore/cerec/downloads.html>
- <https://www.dentsplysirona.com/en/explore/digital-impression/downloads.html>
- <https://www.dentsplysirona.com/en/discover/discover-by-brand/connect-software.html>
- <https://www.dentsplysirona.com/en/explore/3d-printing/primeprint/software-downloads.html>
- <https://my.cerec.com/en/products/connect-sw-5-2.html>