WaveOne® Gold

EN	Instructions For Use	2
FR	Instructions d'utilisation	7
IT	Istruzioni per l'uso	13
DE	Gebrauchsanweisung	19
ES	Instrucciones de uso	25
PT	Instruções de Utilização	31
NL	Gebruiksaanwijzing	37
SV	Användarinstruktioner	43
DA	Brugsanvisning	48
NO	Brukerinstruksjoner	54
LT	Naudojimo instrukcija	59
LV	Lietošanas norādījumi	64
ET	Kasutusjuhised	69
CS	Návod k použití	74
SK	Návod na použitie	79
HU	Használati utasítás	85
RO	Instrucțiuni de utilizare	91
SR	Uputstva za upotrebu	97
BS	Uputstva za upotrebu	102
CNR	Uputstva za upotrebu	107
SQ	Udhëzimet për përdorim	112
EL	Οδηγίες χρήσης	118
TR	Kullanım Kılavuzu	124
PT-BR	Instruções de uso	129
KO	사용 지침	135

WaveOne® Gold

EN

*FOR DENTAL USE ONLY
FOR PROFESSIONAL USE ONLY
SINGLE USE - REPROCESSING PROHIBITED
STERILE – STERILIZED BY RADIATION*

INSTRUCTIONS FOR USE

SINGLE USE STERILE ENDODONTIC INSTRUMENTS

0) INDICATIONS FOR USE

- The products are indicated for treatment of endodontic disease.
- Intended purpose: Engine-driven instrument intended for root canal preparation (glide path, shaping and debridement of the root canal).
- Endodontic instruments are to be used only in a clinical or hospital environment, following good dental practice, by qualified dental professionals such as general practitioners as well as Endo specialists (Endodontist) and Dental Assistants.
- Instruments shall be used in combination with an endodontic motor (manually in severe curvatures).

1) CONTRAINDICATIONS

None.

2) CONTENT, COMPOSITION AND COMPATIBLE DEVICES

WaveOne® Gold instruments are made of three main components: the working part made of a nickel-titanium alloy (M-Wire®), a colored silicone stopper and a plated brass shank with an ABS plastic ring.

The files are manufactured with a process that results in a file that has a colored appearance.

Due to this proprietary processing, the files may appear slightly curved. This is not a manufacturing defect. While the file can be easily straightened using only your fingers, it is not necessary to straighten the file prior to use. Once inside the canal, the file will follow the root canal anatomy.

The WaveOne® Gold range comprises following endodontic instruments:

Code z-quantity xx-length y-file size	Product	ISO XXX YYYv XXX-ISO size YYY-taper size v-variable taper	Available lengths	Packaging and content
BST W1GG zxx 015	WaveOne® Gold Glider files	GLIDER: ISO 015 002v	21/25/31 mm	Blister with 3 instruments (sterile) Blister with 6 instruments (sterile)
A 075z 2xx G0y	WaveOne® Gold shaping files	SMALL: ISO 020 007v PRIMARY: ISO 025 007v MEDIUM: ISO 035 006v LARGE: ISO 045 005v	21/25/31 mm	Blister with 3 instruments (sterile) Blister with 6 instruments (sterile)
A 0754 2xx G9A 03	WaveOne® Gold Assortment	SMALL PRIMARY MEDIUM LARGE	21/25/31 mm	Blister with 1 instrument of each ISO (sterile)
BST W1GS 325 AST	WaveOne® Gold Sequence	SMALL GLIDER	25 mm	Blister with 1 instrument of each ISO (sterile): PRIMARY ISO 020 007v GLIDER ISO 015 002 K-File 010

- Use in a reciprocating motion with a designated drive system having the approved WaveOne®/WaveOne® Gold settings.
- Lubricants such as NaOCl, EDTA, ProLube, Glyde™ shall be used.
- The use of radiographs in combination with an apex locator and a tool for adjusting the silicon stopper to the correct working length is the appropriate method of working length determination.

3) WARNINGS

- Strictly follow this Instruction for Use to minimize following risks to the device, the patient and/or the user:
 - Breakage of instrument.
 - Cross-contamination.
 - Heat generation due to insufficient lubrication and irrigation.
 - Swallowing of working part of the instrument.
 - Toxic or allergic reactions caused by processing residues.
- Similar to all machine-driven root canal instruments, WaveOne® Gold instruments should not be used in a root canal with an abrupt apical curvature due to heightened risk of separation. In this case, pre-curved hand files should be used in the apical region.

4) PRECAUTIONS

- Safety and effectiveness of use have not been established in pregnant or breastfeeding women or in children.
- Use a rubber dam system during the endodontic procedure.

- For your own safety, wear personal protective equipment (gloves, glasses, mask).
- Inspect the packaging before use and do not use the instruments if the packaging is damaged.
- Do not use the instruments after expiration date.
- Check the instrument before each use for signs of defects such as deformations (bent, unwound), breakage, corrosion, damaged cutting edges, loss of color coding or marking. With these indications the devices are not able to fulfil the intended use with the required safety level, instruments should be discarded.
- Before using any instrument, make sure it is well connected to the contra-angle head.
- Check instrument and clean working part frequently during instrumentation, inspecting for signs of distortion, elongation or wear, such as uneven flutes, dull spots. With these indications that the devices are not able to fulfil the intended use with the required safety level, instruments should be discarded.
- Instruments should not be fully immersed in a sodium hypochlorite solution (NaOCl). Only the working part of the NiTi instrument, which is in contact with the patient should be immersed in a NaOCl solution concentrate at NOT more than 5%.
- Exercise caution in the apical area and in canals that divide, and/or exhibit abrupt curvatures or recurvatures.
- Irrigate abundantly and frequently the canal throughout the procedure and after each instrument used (according to good dental practices).
- Always use minimal apical pressure. Never force the files down the canal.
- When instrument does not easily progress, clean and inspect the cutting flutes, then irrigate, recapitulate with a manual file and reirrigate.
- For shaping extremely curved canals it is safer to use the file only to shape one canal in order to reduce the risk of breakage. Pay attention to the following good practices:
 - Use a new file and discard it after the canal was treated (single canal use).
 - Use manual instead of rotary files.
 - Use small size, flexible or/and NiTi files (this will help avoid canal transportation).
 - Visually inspect the working part for all the defects listed in the former paragraph during use (i.e after each wave).
 - Avoid the standard reaming continual rotational motion and instead use small angle motions (filing motion, watch winding oscillation motion, or balanced force technique) in order to limit the rotational bending fatigue on the instruments and improve their expected life.

5) ADVERSE REACTIONS

In the present technical state, no adverse reaction has been reported so far.

6) STORAGE CONDITIONS

Keep products in a dry and clean environment, away from sources of moisture and direct sunlight.

7) STEP BY STEP INSTRUCTIONS

Glide Path creation with WaveOne® Gold Glider

- 1) Review different horizontally angulated radiographs to diagnostically determine the width, length, and curvature of any given root canal.
- 2) Create straight-line access to the canal orifice(s) with emphasis on flaring, flattening, and finishing the internal axial walls.
- 3) Irrigate abundantly and frequently after each instrument used (according to good dental practices).
- 4) In the presence of a viscous chelator, scout the root canal with hand files, at least an ISO 010, determine the working length by using an electronic apex locator in combination with radiographs, irrigate and confirm patency.
- 5) With gentle inward pressure, let the WaveOne® Gold Glider file passively progress through any secured region of the canal. Use the WaveOne® Gold Glider file in one or more passes until the full working length is reached.
- 6) Irrigate the expanded glide path, recapitulate and irrigate and again.
- 7) Reconfirm the working length, before shaping the canal with the WaveOne® Gold shaping system.

Shaping procedure with WaveOne® Gold

- 1) Review different horizontally angulated radiographs to diagnostically determine the width, length, and curvature of any given root canal.
- 2) Create straight-line access to the canal orifice(s) with emphasis on flaring, flattening, and finishing the internal axial walls.
- 3) Irrigate abundantly and frequently after each instrument used (according to good dental practices).
- 4) With irrigant in the canal, create and confirm a reproducible glide path using an ISO 010 hand file until full working length is reached.
- 5) Expand and secure this glide path with at least an ISO 015 file by using either a manual or a dedicated mechanical file, such as PathFile®, ProGlider® or the dedicated WaveOne® Gold Glider file.
- 6) ALWAYS initiate the shaping procedure with PRIMARY file (025.07 red) in the presence of sodium hypochlorite.
- 7) Use gentle inward pressure and let PRIMARY file passively progress through any region of the canal that has a confirmed glide path. After shaping 2-3 mm of any given canal, remove and clean the PRIMARY file.
- 8) Irrigate, recapitulate with an ISO 010 hand file and re-irrigate.
- 9) Continue with PRIMARY file, in 2-3 passes, to pre-enlarge the coronal 2/3 of the canal.
- 10) Use a brushing motion on the outstroke to eliminate coronal interferences or to enhance shaping results in canals that exhibit irregular cross-sections.
- 11) In more restrictive canals, use an ISO 010 hand file, in the presence of viscous chelator, negotiate to the terminus of the canal. Gently work this file until it is completely loose at length.
- 12) Establish working length, confirm patency and verify the glide path.
- 13) Expand and secure this glide path with at least an ISO 015 file by using either a manual or a dedicated mechanical glide path file.

- 14) Carry PRIMARY file to full working length in one or more passes. Upon reaching length, remove the file to avoid over-enlarging the foramen.
- 15) Inspect the apical flutes:
 - If they are loaded with dentinal debris, then the shape is finished.
 - If PRIMARY file is loose at length with no dentinal debris in the apical flutes, continue shaping with MEDIUM file (035.06 green) and/or LARGE file (045.05 white) until the apical flutes are loaded.
- 16) If the PRIMARY does not progress, then use SMALL file (020.07 yellow) in one or more passes to working length and then use PRIMARY file to working length to optimize the shape.
- 17) Use dedicated paper points to dry and dedicated gutta-percha points to obturate the root canals.

8) HYGIENE, DISINFECTION, CLEANING AND STERILIZATION

- WaveOne® Gold instruments are single use devices and cannot be reused. Re-use can increase the risk of cross contamination or breakage.
- Disinfection, cleaning and sterilization is not applicable.
- Products shall be disposed according to local regulations for the safe disposal of sharp and contaminated devices.

9) ADDITIONAL INFORMATION

- Any serious incident in relation to the product should be reported to the manufacturer and the competent authority according to local regulations.
- Sterility cannot be guaranteed if packaging is open, damaged or wet.
- To get a free printed copy of IFU please see section “Get a print copy of IFU” on website <https://www.dentsplysirona.com/en/service-contact/ifu.html>.
- Explanation of non-harmonized symbols for IFUs and labels, see IFU Symbols (<https://www.dentsplysirona.com/en/service-contact/ifu/ifu-symbols.html>).

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DENTSPLY DeTrey GmbH