AH Plus[®] AH Plus Jet[®] Root Canal Sealing Material

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Sigillante canalare

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CAUTION: For dental use only.

USA: Rx only.

Content	Page
1 Product description	2
2 Safety notes	3
3 Step-by-step instructions	5
4 Hygiene	7
5 Lot number, expiration date and correspondence	8

1 Product description

The intended purpose of **AH Plus®** root canal sealing material and **AH Plus Jet®** root canal sealing material is to be used as permanent root canal sealer.

AH Plus® root canal sealing material and **AH Plus Jet®** root canal sealing material are based on epoxy-amine resin, offering the following features:

- Long-term sealing properties.
- Outstanding dimensional stability.
- Self-adhesive properties.
- Very high radiopacity.

AH Plus® root canal sealing material and **AH Plus Jet®** root canal sealing material do not stain teeth.

1.1 Indications

- Permanent obturation of root canals of teeth of the secondary dentition in combination with root canal points.

1.2 Contraindications

- Use with patients who have a history of allergic reaction to epoxy resins or amines or any of the other components (see Composition and Warnings for further details).

1.3 Delivery forms

- AH Plus® in tubes for manual mixing of paste A and paste B.
- AH Plus Jet® mixing syringe offering a more precise, convenient and faster procedure.

1.4 Composition

Paste A (amber color)

- Calcium tungstate
- Araldite GY 250 epoxy resin
- Zirconium oxide
- Araldite GY 285 epoxy resin
- Highly dispersed silicon dioxide
- Iron oxide pigments

Paste B (white color)

- Calcium tungstate
- Zirconium oxide
- Dibenzylidiamine
- Amantadine
- Highly dispersed silicon dioxide
- Polydimethylsiloxane
- Tricyclodecane-diamine

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

The material contains epoxy resins Araldite GY 250 epoxy resin (Bisphenol-A epoxy resin; 4,4'-Isopropylidenediphenol, oligomeric reaction products with 1-chloro-2,3-epoxypropane; CAS No. 25068-38-6) and Araldite GY 285 epoxy resin (formaldehyde, oligomeric reaction products with 1-chloro-2,3-epoxypropane and phenol; CAS No. 9003-36-5) and amines Amantadine (1-Amino adamantane; CAS No. 768-94-5) and Tricyclodecane-diamine (Octahydro-4,7-methano-1H-indenedimethylamine; CAS No. 68889-71-4) which may be irritating to skin, eyes, and oral mucosa and may cause allergic contact dermatitis, allergic sensitization, or systemic allergic reactions in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes rinse immediately with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.
- **Do not re-use contaminated tubes or syringes** to avoid cross-contamination.
- **Do not use improperly mixed material** as wrong mix ratio or inhomogeneous mixing may lead to inadequate curing of the material and leaching of monomers or oligomers.
- **Prevent ingestion.** If accidental **swallowing** occurs, have patient drink plenty of water. If nausea or illness develops, seek medical attention.
- **Do not place too much sealer** to prevent overfilling of the root canal, damage of periapical tissue, and acute inflammation of periapical tissues resulting in (temporary) post-operative pain or extrusion into the mandibular nerve canal.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the root canal filling. Use adequate isolation such as rubber dam.
- The syringes cannot be reprocessed. To prevent from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringes are handled with clean/disinfected gloves. Discard syringes if contaminated.
- As additional precautionary measure, the syringe may be protected from gross contamination but not from all contamination by applying the single use Disposa-Shield® protective barrier¹.
- AH Plus Jet® root canal sealing material mixing tips and Disposa-Shield® protective barrier are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination or malfunction.
- Make sure that AH Plus Jet® root canal sealing material mixing tips are correctly and firmly attached to the syringe and the intraoral tip to the mixing tip.
- Interactions:
 - If refrigerated, allow material to reach room temperature prior to use.

2.3 Adverse reactions

Following adverse reactions or undesirable side-effects might occur. See chapter Warnings and referenced chapters for details how to prevent or react.

- Acute inflammation of periapical tissues resulting in (temporary) post-operative pain (see Step-by-Step chapter 3.3).
- Irritation and possible corneal damage.
- Irritation of skin or possible allergic contact dermatitis.
- Inflammation of mucous membranes.
- Nausea or illness.
- Dys- or anaesthesia could occur if material is overfilled and extruded into the nerve canal (see Warnings).
- Ingestion or aspiration of mixing or intraoral tip (see Precautions and Step-by-Step chapter 3.2.2).

In general, root canal treatment and obturation might be associated with discharging sinus, intermittent dull ache, and intermittent discomfort of biting and, therefore, could occur when this material is used, too.

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store at temperatures from 10 °C to 24 °C (50 °F-75 °F). Use the product at room temperature.
- Do not use after expiration date.

¹ Disposa-Shield® Infection Control Barrier for low speed, long handpiece, REF A88006.

3 Step-by-step instructions

3.1 Preparation

1. Prior to the application of the material prepare, clean, and dry the root canals to be filled using state-of-the-art endodontic techniques.

3.2 Dosage and mixing

3.2.1 AH Plus® root canal sealing material (tubes)

Appropriate paste condition.

For Paste B slight separation may occur. This condition does not negatively affect the setting time of the mixed product. If the dispensed material is solely of clear color, discard the dispensed material, to ensure convenient handling of the mixed paste.

1. Using a metal spatula², mix equal volume units (1:1) of paste A (amber color) and paste B (white color) of AH Plus® root canal sealing material on a glass slab or the mixing pad supplied with the package. Mix to a homogeneous consistency.
2. Tightly close tubes after use.
3. Do not exchange caps of tubes. The white cap belongs to paste A; the colored cap belongs to paste B.

3.2.2 AH Plus Jet® root canal sealing material (mixing syringe)

Appropriate mixing ratios.

The syringe filling volumes of pastes A (amber color) and B (white color) vary slightly. To ensure appropriate mixing, bleed the syringe prior to the first use.

The AH Plus Jet® syringe cannot be reprocessed. To prevent the AH Plus Jet® syringe from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe are handled with clean/disinfected gloves.

1. Remove the cap by turning it 90° counter-clockwise and pulling it.
2. Attach the mixing tip³ to the syringe, aligning the notch.
3. Turn the tip 90° clockwise and secure the mixing tip in place.
4. Rotate and adjust the tip in angle in order to meet treatment requirements.
5. Use Disposa-Shield® protective barrier when cross-contamination cannot be avoided.
6. Carefully apply steady pressure onto the plunger. The two pastes are automatically mixed in equal volume units.
7. After having finished remove mixing tip by turning it 90° counter-clockwise and pulling it off (together with protective barrier if any).
8. Replace the cap on the syringe head, aligning the notch, then turn it 90° clockwise.

² Mixing ratio by weight is 1 g of paste A to 1.18 g of paste B.

³ AH Plus Jet® Mixing Tips, EU REF 60620116, US REF 667006.

3.3 Application

Overfilling.

Damage of periapical tissue and acute inflammation of periapical tissues resulting in (temporary) post-operative pain.

1. Do not place too much sealer to prevent overfilling of the root canal. For condensation and warm gutta-percha obturation techniques apply only a light coating of sealer onto the canal walls.
2. If overfilling occurs the material is usually tolerated very well by the surrounding tissue. If however material is pressed into the mandibular canal, immediately apply state of care measures.

Clogging.

Separation of mixing tip from syringe.

1. Do not allow material to dry inside.
2. Do not reuse mixing tip.
3. Always make sure mixing tip is properly connected to syringe before application. The material should extrude easily.

3.3.1 Condensation and warm gutta-percha techniques

1. As a standard technique, the material is used in combination with gutta-percha or other root canal points.
2. For these filling techniques where most of the canal is obturated by endodontic point material, apply a light coating of sealer with a paper point or file onto the canal walls to the working length and/or apply a light coating of sealer onto the surface(s) of the root canal filling point(s).
3. Insert root canal filling point(s) into the canal according to standard procedure.

3.3.2 Master-Point-Technique

1. Select a gutta-percha point (or alternatively a paper point or a reamer) of the size of the last instrument used during apical preparation.
2. Wet the canal walls with the material through a pumping or simultaneously rotating movement in a counter-clockwise direction of the point/reamer. Alternatively, apply the material onto the tip of a Lentulo spiral.
3. Advance the Lentulo spiral slowly to the apex running at very low speed. Avoid the formation of air bubbles in the material and overfilling of the canal.
4. Withdraw Lentulo very slowly still running at low speed.
5. Dip disinfected and dry master point into the material and insert it into the canal with a slow pumping motion.

3.3.3 Working and setting time

- Working time is 4 hours at 23 °C⁴.
- Setting time is 24 hours at 37 °C⁵.

3.4 Removal of root canal filling

1. If the material is used in combination with (gutta-percha) points, root canal fillings can best be removed using thermo-mechanical techniques.

⁴ Internal test method.

⁵ Measured according to ISO 6876:2012 (E). The complete polymerization needs 7 days at 37 °C.

4 Hygiene

4.1 AH Plus Jet® syringe

Cross-contamination.

Infection.

1. The syringe cannot be reprocessed.
2. To prevent the syringe from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe and finger grip are handled with clean/disinfected gloves. Do not reuse syringe if contaminated.
3. Dispose of contaminated syringe in accordance with local regulations.

4.2 AH Plus Jet® mixing tips and Disposa-Shield® protective barrier

The use of protective barriers is an additional precautionary measure against gross contamination but not against all contamination.

1. Right after use, carefully remove protective barrier together with mixing tip without contaminating the device.

Cross-contamination.

Infection.

1. Disposa-Shield® protective barrier and AH Plus Jet® root canal sealing material mixing tips are intended for single use only. Do not reuse in order to prevent cross-contamination or malfunction.
2. Discard in accordance with local regulations.

4.3 Cleaning

1. Clean spatulas, mixing slabs and instruments immediately after use with alcohol or acetone.

4.4 Disposal

Dispose of in accordance with local regulations.

5 Lot number (), expiration date () and correspondence

1. Do not use after expiration date.
ISO standard is used: “YYYY-MM” or “YYYY-MM-DD”
2. The following numbers should be quoted in all correspondence:
 - Reorder number (REF)
 - Lot number
 - Expiration date
3. Any serious incident in relation to the product should be reported to the manufacturer and the competent authority according to local regulations.
4. Device Identification (Basic UDI-DI): ++D010EFM02PZ

A summary of the safety and clinical performance (SSCP) for this product can be found (upon activation) at <https://ec.europa.eu/tools/eudamed> by searching using the Basic UDI-DI number listed above and at <https://www.dentsplysirona.com/ifu> using the reference number (REF).

© Dentsply Sirona 2023-04-25

[These Instructions for Use are based on Master Version 13]

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Dentsply DeTrey GmbH