DE

Diagram of the Ankylos® Chirurgie Kit with German labels for various instruments and components, including drill bits, handles, and motor modules.

EN

Diagram of the Ankylos® Surgical Kit with English labels for instruments like twist drills, ratchets, and conical reamers, along with their respective order numbers.

FR

Diagram of the Ankylos® Kit chirurgical with French labels for tools such as twist drills, ratchets, and conical reamers, including part numbers.

ES

Diagram of the Ankylos® Juego quirúrgico with Spanish labels for surgical instruments like twist drills, ratchets, and conical reamers, with corresponding reference numbers.

IT

Diagram of the Ankylos® Kit Chirurgico with Italian labels for instruments including twist drills, ratchets, and conical reamers, listing product codes.

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The number of holes in the modules depends on the kit.

Ankylos® Surgical Kit manual ABC Order no. 3103 6503
Ankylos® Surgical Kit manual AB Order no. 3103 6513
Ankylos® Surgical Kit motor AB Order no. 3103 6523

Install silicone gasket

Cleaning and sterilization instructions
Please observe the Instructions for Use of the individual instruments and the Dentsply Sirona Implants Instructions for Sterilization and Instrument Care. Before using the tray for the first surgical treatment it must undergo a complete processing and sterilization cycle together with the instruments.

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Ankylos® Juego quirúrgico manual AB N° de ref. 3103 6513
Ankylos® Juego quirúrgico motor AB N° de ref. 3103 6523
Ankylos® Kit chirurgico manuale AB Cod. prod. 3103 6513
Ankylos® Kit chirurgico motore AB Cod. prod. 3103 6523