Multilingual MRI Safety Information

Contents

EN (English) – Magnetic Resonance Imaging (MRI) Safety Information	1
BG (Български) – Ядремно-Магнитен Ресонанс (ЯМР) Информация за безопасност	2
CS (Čeština) – Bezpečnostní informace pro magnetickou rezonanci (MR)	3
DA (Dansk) – Magnetisk resonans-billeddannelse (MR) Sikkerhedsoplysninger	4
DE (Deutsch) – Sicherheitshinweis zu Magnetresonanztomographie (MRT)	5
EL (ελληνικά) – Πληροφορίες ασφάλειας σε περιβάλλον απεικόνισης μαγνητικού συντονισμού (MRI).....	6
ES (Español) – Información de seguridad acerca de la resonancia magnética (RM).....	7
ET (Eesti) – Magnetresonantstomograafia (MRT) uuringut puudutav ohutusteave	8
FI (Suomi) – Magneettikuvauksen turvallisuustiedot.....	9
FR (Français) – Informations de sécurité Imagerie par Résonance Magnétique (IRM)	10
HR (Hrvatski) – Sigurnosne informacije o magnetskoj rezonanciji (MR).....	11
HU (Magyar) – Mágneses rezonancia képalkotás (MRI) Biztonsági információk.....	12
IT (Italiano) – Informazioni di sicurezza per la risonanza magnetica (RM)	13
LT (Lietuvių) – Magnetinio rezonanso tomografija (MRT) saugos informacija	14
LV (Latviešu) – Informācija par magnētiskās rezonanses attēlveidošanas (MRI) drošumu	15
NB (Norsk) – Sikkerhetsinformasjon om magnetresonanstomografi (MR)	16
NL (Nederlands) – Magnetische resonantie (MRI)-veiligheidsinformatie.....	17
PL (Polski) – Informacje dotyczące bezpieczeństwa w przypadku obrazowania metodą rezonansu magnetycznego (MRI)	18
PT (Português) – Imagiologia por ressonância magnética (IRM) Informações de segurança	19
RO (Română) – Informații de siguranță Imagistică prin Rezonanță magnetică (IRM)	20
SK (Slovenčina) – Informácie o bezpečnosti zobrazovania magnetickou rezonanciou (MR).....	21
SL (Slovenščina) – Slikanje z magnetno resonanco (MRI) Varnostne informacije	22
SV (Svenska) – Säkerhetsinformation om magnetröntgen (MRI)	23
TR (Türkçe) – Manyetik Rezonans Görüntüleme (MRG) Güvenlik Bilgisi	24

EN (English) – Magnetic Resonance Imaging (MRI) Safety Information

MR Conditional

A patient with the Dentsply Sirona Implants assortment may be safely scanned under the following conditions. Failure to follow these conditions may result in injury to the patient.

Name/Identification of device	Dentsply Sirona Implants assortment
Nominal value(s) of Static Magnetic Field [T]	1.5 T or 3 T
Maximum Spatial Field Gradient [T/m and gauss/cm]	40 T/m (4000 gauss/cm)
Radio Frequency (RF) Excitation	Circularly Polarized (CP)
Radio Frequency (RF) Transmit Coil Type	Whole body transmit coil, Head RF transmit-receive coil
Maximum Whole Body Specific Absorption Rate (SAR) [W/kg]	2.0 W/kg (Normal Operating Mode)
Maximum Head SAR [W/kg]	3.2 W/kg (Normal Operating Mode)
Limits on Scan Duration	2.0 W/kg whole body average SAR for 15 minutes of continuous RF (a sequence or back-to-back series/scan without breaks) followed by a 5-minute cooling period for an hour-long scan.
MR Image Artifact	The presence of this implant may produce an image artifact. Some manipulation of scan parameters may be needed to compensate for the artifact.

If information about a specific parameter is not included, there are no conditions associated with that parameter.

MR Safe

The Dentsply Sirona Implants assortment with dental components made of polymers, glass-ceramics, and ceramics is MR Safe.

Subject to change without prior notice.

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