





# Reproducibility of V-3D-Shades within the Kiss Concept

V-3D-SHADES	Opaque	Dentine mixture				Incisal
		Dentine basic powders		Power Chroma additional powders		
						
		Portioner large	Portioner large	Portioner small	Portioner small	
1 M 1	B1	B1	-	-	-	S1
1 M 2	A1	A1	A2 (small)	-	-	S1
2 L 1,5	B2	B2	-	-	-	S1
2 L 2,5	B2	B2	B3	-	-	S1
2 M 1	D2	D2	-	-	-	S2
2 M 2	A2	A2	A3	-	-	S2
2 M 3	B3	B3	-	PC 1	PC 3	S4
2 R 1,5	A2	A2 (2x)	-	PC 1	PC 4	S2
2 R 2,5	A2	A3	A2	PC 1	PC 4	S2
3 L 1,5	C2	C2	-	-	-	S5
3 L 2,5	B3	B3	-	-	-	S4
3 M 1	C1	C1	C2	-	-	S1
3 M 2	A3	A3	-	-	-	S3
3 M 3	B4	B4	-	PC 3	PC 5	S6
3 R 1,5	D3	D3 (2x)	-	PC 5	-	S4
3 R 2,5	A3,5	A3,5 (2x)	-	PC 3	PC 5	S3
4 L 1,5	C3	C3	C4 (small)	-	-	S5
4 L 2,5	A4	A4 (2x)	-	PC 4	PC 6	S5
4 M 1	C3	C3	C4	-	-	S5
4 M 2	A3,5	A3,5 (2x)	-	PC 4	PC 6	S3
4 M 3	A4	A4	-	PC 4	PC 6	S5
4 R 1,5	A3,5	A3,5 (2x)	-	PC 5 (2x)	-	S3
4 R 2,5	A4	A4	-	PC 4	PC 5	S5
5 M 1	C4	C4	-	PC 4	PC 6	S6
5 M 2	A4	A4	-	PC 4	PC 6	S5
5 M 3	A4	A4	-	PC 4 (large)	PC6 (large)	S5

The values listed here are intended for orientation only and should be regarded only as guidelines. The shades could differ. We have compiled and checked all values and other data with care. However, we cannot under any circumstances be liable for your results.



DeguDent GmbH  
Rodenbacher Chaussee 4  
63457 Hanau-Wolfgang  
Germany  
[www.dentsplysirona.com](http://www.dentsplysirona.com)

REV 2021-06