# Cavitron® Ultrasonic Inserts

## Directions for Use and Infection Control Procedures

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## DIRECTIONS FOR USE

Cavitron® Ultrasonic Inserts are to be used with magnetostrictive ultrasonic scalers.

### Indications for Use

Used for ultrasonic procedures:

- All general supra and subgingival scaling applications.
- Periodontal debridement for all types of periodontal diseases.

### Contraindications

Cavitron Inserts should not be used for restorative dental procedures involving condensation of amalgam.

### Warnings

- 2 • Tips with high usage which exceed the warranty period should be

discarded to avoid possible in-use breakage and possible injury

- Handle insert with care. Improper handling of insert, specifically the insert tip, may result in injury and/or cross contamination.
- Failure to follow properly validated sterilization processes and approved aseptic techniques for Cavitron Ultrasonic Inserts may result in cross contamination.
- Cavitron® Ultrasonic Inserts with metal grips are not compatible with the Steri-Mate® 360 Handpiece.

### Precautions

It is the responsibility of the Dental Healthcare Professional to:

- Operate the inserts only at the recommended power settings.
- Check water spray to verify fluid is reaching the working end of the insert tip. Adjust the water lavage control until the water (lavage) flows with a rapid drip or small spray (approximately 35mL). Higher water flow settings provide cooler irrigation.
- Not to bend or reshape insert. Inserts should be handled with care

at all times. Ultrasonic inserts tips that have been bent, reshaped or otherwise damaged are susceptible to in-use breakage and should be discarded and replaced immediately.

- Understand that Cavitron Ultrasonic Scaling Units and Cavitron Ultrasonic Inserts are designed and tested for safety and efficacy as a system. Use of any other brand ultrasonic insert with Cavitron equipment has not been tested and may have a negative effect on the safety and effectiveness of your Cavitron equipment and their warranties. Please review the Warranty section before use.
- Understand that like bristles of a toothbrush, ultrasonic inserts “wear” with use. Refer to the Efficiency Wear Indicators supplied with your Cavitron Ultrasonic insert.

### Adverse Reactions

None Known.

## INSERT MARKING

## FDA'S UNIQUE DEVICE IDENTIFICATION (UDI)

### Unit Power Settings

- Slimline® 10S, L&R should only be used up to medium power.
- All other Cavitron Inserts may be used up to maximum power.

### Operating Instructions

1. Select the appropriate insert for the tooth or root to be treated.

2. Fill entire handpiece with water.
3. Lubricate O-ring (black or green rubber material) with water.
4. Gently twist the insert down into the handpiece until the insert is fully seated in place.
5. Use the lowest effective power setting.
6. Adjust the water control until the water (lavage) flows with a rapid drip or small spray.
7. Use very light pressure with a light pen or modified pen grasp.
8. Use a light but stable intraoral or extraoral fulcrum.

9. Adapt the insert so the point is not directed into root surface
10. Use a light, smooth gentle touch.

11. Keep the insert tip in motion at all times, using overlapping horizontal, vertical or oblique brush-like strokes.
12. Use the inactivated insert to explore/evaluate root surface.

## INFECTION CONTROL PROCEDURES

### Warnings

- All devices should be cleaned and sterilized before each use. For the countries that require a three-step process, in which disinfection is required prior to sterilization, all devices must be cleaned, disinfected and sterilized before use.
- High-level disinfection by itself is not appropriate for these devices.

### Limitations on Processing

- Repeated processing has minimal effect; end of life is normally determined by wear and damage due to use.
- Always use a pH neutral instrument cleaning solution (such as

Dentsply Sirona Resurge® Instrument Cleaning Liquid which is recommended)/ disinfecting detergent or solution. Product can be damaged by alkaline and acidic detergents.

- Failure to follow the cleaning solution recommended concentration rates and contact times may result in rapid deterioration of the material.
- Cold liquid disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use.

### Instructions

Point of use:

- Ultrasonic inserts must be removed from the handpiece before processing.
- Do not allow residue or any form of contamination to dry on device. Remove excess soil/gross debris by rinsing under running water.

- Following use, device should be reprocessed as soon as is reasonably practical.

### Mechanical Cleaning: Ultrasonic Cleaners

- Follow manufacturer instructions for instrument cleaning solution, observing concentration rates and contact times.

**Note:** Some instrument cleaning solutions also disinfect.

- May be cleaned using an ultrasonic water bath for up to 15 minutes.
- Prepare an enzymatic instrument cleaning solution per manufacturer's instructions paying attention to concentration rates and contact times. A pH neutral instrument cleaning solution such as Dentsply Sirona Resurge® Instrument Cleaning Liquid is recommended for ultrasonic bath.
- Adhere to the ultrasonic cleaning bath manufacturer's recommended liquid level.
- The use of an instrument cassette during an ultrasonic bath is recommended to reduce the risk of instrument damage.

- Place the device into the ultrasonic bath and cover for the indicated contact time; the instruments must be sufficiently covered.
- Remove the instruments from the ultrasonic cleaning bath. Rinse the insert under tepid (potable) running water for 30 seconds to remove cleaning agent: a disposable syringe may be used to rinse the holes/cannulas.
- Inspect the insert for visible debris. If debris remains, manually clean using a nylon brush wetted with the cleaning solution to remove it. Rinse the insert under tepid (potable) running water for 30 seconds to remove cleaning agent.
- Dry the insert using a dry, disposable lint-free cloth or use a dental instrument dryer. An insert left standing wet may stain, corrode, or clog.

### **Cleaning and Disinfection: Automated - Instrument Washer-Disinfector**

- To remove gross debris, wipe with a disposable lint free cloth while rinsing under tepid (potable) running water for 30 seconds. A Disposable syringe may be used to rinse the holes/cannulas. Pay special attention to rinse all holes/cannulas.
- Follow manufacturer instructions for instrument washer solution, observing concentration rates and contact times as well as the instrument washers recommended cleaning and/or thermal disinfection cycle (1 minute at 90°C).
- The use of an instrument cassette during an automated cleaning/disinfection is recommended to reduce the risk of instrument damage.
- Place the device into the instrument Washer Disinfector and begin the washer disinfector cycle. Washer disinfector must conform to ISO 15883.
- After the drying cycle ends remove the instruments from the

Washer Disinfector. An Insert left standing wet may stain, corrode, or clog.

### **Cleaning: Manual**

- Prepare an enzymatic instrument cleaning solution per manufacturer's instructions paying attention to concentration rates. A pH neutral instrument cleaning solution such as Dentsply Sirona Resurge® Instrument Cleaning Liquid is recommended.
- Fully immerse the insert in the instrument cleaning solution for the recommended contact time.
- Rinse the insert under tepid, potable running water for at least 30 seconds. A disposable syringe may be used to rinse the holes and cannulas.
- Inspect the insert for visible debris. If debris remains, repeat the cleaning procedure using a nylon brush wetted with the cleaning solution to remove it. Rinse the insert under tepid (potable) running water for 30 seconds to remove cleaning agent.

- Dry the insert using a dry, disposable lint-free cloth or use a dental instrument dryer. An insert left standing wet may stain, corrode, or clog.

### **Maintenance, inspection and testing**

- Visually inspect to ensure all contamination is removed.
- Visually inspect all O-rings for wear. Replace worn O-rings (Refer to O-ring Replacement Kit Instructions). Lubricate O-rings with water before use.
- Check for distortion, damage and wear (Refer to Efficiency Wear Indicator Instructions). Discard damaged, worn or corroded instruments.

### **Sterilization**

Place bagged instruments into the steam sterilizer per manufacturer's instructions.

STEAM STERILIZATION CYCLES	STERILIZING TEMPERATURE	STERILIZING CYCLE TIME
*Dynamic Air Removal	270°F / 132°C	3 minutes
Gravity Displacement	270°F / 132°C	10 minutes

\*The steam sterilization parameters specified above are for Dynamic Air Removal steam sterilizers (pre-vacuum and steam flush pressure pulse).

#### Alternate method

Place non-bagged instruments into the steam sterilizer and run at the listed cycles.

**Note:** Instruments sterilized unbagged should be used immediately.

#### Drying

To dry, use the drying cycle of the sterilizer. Set cycle for 20-30 minutes.

#### Storage

To maintain sterility, instruments should remain bagged until ready for use.

## WARRANTY

Cavitron® Ultrasonic Inserts manufactured by Dentsply Sirona are warranted against defects arising from faulty materials or workmanship for ninety days from the date of purchase. This warranty extends to inserts purchased from an authorized Dentsply Sirona distributor and only to the original purchaser and is valid only for inserts purchased in the United States (U.S.) and for those countries without specific warranty requirements. For inserts purchased outside the U.S. in countries with specific warranty requirements, those applicable country warranties will apply.

This warranty is subject to the following conditions:

1. Inserts must not be subjected to abuse or improper use or application.

2. Inserts must not be subjected to abuse or improper cleaning, disinfection, or sterilization procedures.
3. Bending or reshaping the inserts will void the warranty.
4. Using inserts in units other than Cavitron Ultrasonic Systems will void the warranty.

There are no warranties, expressed or implied beyond the description on the face hereof. Dentsply Sirona neither assumes, nor authorizes any person to assume for it, any other liability for incidental and consequential damages resulting from the use of the insert.

To receive reimbursement for any defective Cavitron Ultrasonic Insert, you must send the defective product within the warranty period to the dealer or manufacturer with specification of the defect and receipt of purchase.

#### **For Technical Support**

Call Dentsply Sirona Preventive Product Services at 1-800-989-8826.

#### **Learn more about Cavitron® Ultrasonic Inserts**

For more information about Cavitron Ultrasonic Inserts, please visit [dentsplysirona.com](http://dentsplysirona.com).

#### **Disposal**

Dispose of inserts in accordance with state and local laws.

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Dentsply DeTrey GmbH