# Cavitron® Jet Plus System

## Quick Maintenance Guide

**IMPORTANT: Turn OFF water supply to Cavitron system or disconnect water quickdisconnect from the water supply when not in use.**

### Start-Up Procedures (beginning of day):

1. Open the manual shut-off valve on the dental office water supply system.
2. With the Cavitron Jet Plus System OFF, unscrew the powder bowl cap. Verify the powder bowl is empty. Turn the system ON for 15 seconds to eliminate residual moisture in the lines. Turn the system OFF.
3. Shake the powder bottle well to create an even consistency of powder mixture.
4. Pour enough powder into the bowl for the procedure to be performed. Do not fill above the top of the center tube.
5. Secure the cap on the powder bowl.
6. Install a sterilized Jet-Mate Handpiece onto the handpiece cable.
7. Set the Power Level Control to minimum and the Lavage Control to maximum.
8. Turn the system ON.
9. If powder fluffing is observed when the Tap-On™ Foot Pedal is not in use, this would indicate an air leak. To correct, turn the System OFF, remove the Powder Cap, clean any residual powder from the O-ring seal and threads, replace the Powder Cap, tighten, and turn the System back ON.
10. Hold the sterilized handpiece (without an insert or nozzle insert installed) over a sink or drain. Activate the Purge Control button.
  - The Purge button will light for two minutes indicating proper activation of the purge function.
  - If the Purge button is activated with an insert present in the handpiece, the button will blink for 3 seconds and disable. Remove the insert from the handpiece and press the Purge button again.
  - The Purge function can be interrupted at any time during the two minute cycle by pressing the Purge button again or by pressing the Tap-On™ Foot Pedal.
11. After completing the purge cycle, place a sterilized 30K Cavitron® Ultrasonic Insert into the handpiece and set the Power Level Control and Lavage Control to your preferred operating position for ultrasonic scaling. For air polishing, place a sterilized Jet Air Polishing Insert into the handpiece and adjust the Power Level Control to Prophy Mode, and the Powder Flow and Lavage Controls to your preferred operating positions.

### Between Patients:

1. Remove the used Cavitron® Ultrasonic Insert or Jet Air Polishing Insert. Clean and sterilize following the Infection Control Procedures that were enclosed with your insert.
2. Hold the handpiece over a sink or drain and activate Purge function as described in Step 10 of the Start-Up procedure.
3. After the purge cycle is complete, turn the System to the OFF (O) position.
4. Remove the Jet-Mate handpiece, clean and sterilize following the procedures outlined in the Cavitron Systems Infection Control Procedures Booklet that was enclosed with your system.
5. Disinfect the surfaces of the cabinet, Power Cord, Handpiece Cable, Tap-On™ Foot Pedal and Cable assembly (if applicable), Water Supply and Air Supply lines as instructed in the Cavitron Systems Infection Control Procedures Booklet. **DO NOT SPRAY DISINFECTANT SOLUTION DIRECTLY ON SYSTEM SURFACES\*.**

6. Inspect the handpiece cable for any breaks or tears.
7. If using a closed water supply or DualSelect Dispensing System, check for adequate fluid volumes for the next patient.
8. Check the powder bowl for sufficient powder for the next procedure.
9. When ready to use, place a sterilized Jet-Mate Handpiece onto the handpiece cable assembly and insert a sterilized ultrasonic insert or air polishing insert into the handpiece and adjust system controls as preferred.

**NOTE: Refer to the Infection Control Procedure Booklet supplied with your system for disinfectant products that have been tested for compatibility.**

### Shut-Down Procedures (end of day):

Follow the "Between Patients" maintenance procedures, steps 1 through 6. If quick-disconnect is not attached to the end of the hose, it is recommended to close the manual shut-off valve on the dental water supply system. If a quick-disconnect is attached to the end of the hose:

- Disconnect the water supply line from the dental office water supply.
- Relieve the water pressure by pressing the tip of the connector into an appropriate container and allow water to drain.

1. Unscrew the powder bowl cap.
2. Remove the powder bowl from the unit and discard the unused powder.
3. Holding the open end of the powder bowl away from you, activate the system for 15 seconds to clear the bowl. A high volume evacuator can be used to remove any residual powder.
4. Remove the o-ring seal from the powder bowl cap and using a soft dry cloth, wipe residual powder from the cap, the o-ring and the powder bowl threads. Be careful not to scratch or otherwise damage the cap.
5. Return the o-ring to the cap and secure the cap on the powder bowl.

### Extended periods of closure:

Complete "Shut-Down Procedures (end of day)", in addition:

1. Remove the hose from the system, by pushing on the outer ring of the system's water inlet and gently pull out the waterline.
2. As an additional precaution, blowing air through the waterline (using an air/water syringe) may also be helpful when preparing for extended inactive use of a Cavitron Ultrasonic Scaling System

### Handpiece Cable

While the Cavitron Handpiece Cables are designed to offer optimum flexibility and endurance, improper handling may lead to a shortened useful life of the cable assembly, due to excessive wear on the wires and/or internal tubing. To prevent premature failure of the cable assembly, please use the following steps:

1. Do not wrap the cable tightly around the Cavitron system or any apparatus.
2. Avoid overstretching or pulling the cable.
3. If possible, minimize frequent bending/folding that exceeds normal usage during dental procedure.
4. For detachable handpiece cables, hold by the plug body when detaching from the Cavitron system.
5. Gently loop the cable in a continuous loose circle after each use.

## Waterline Maintenance Instructions

### Dental Unit Waterline Overview:

Dental Unit Waterline Maintenance is a key component to a facility's effective Infection Control and Prevention Protocol. The Centers for Disease Control and Prevention (CDC) recommends that dental unit water used in nonsurgical procedures measure less than or equal to 500 colony forming units of heterotrophic bacteria per milliliter ( $\leq 500$  CFU/mL) of water, the standard set for drinking water by the Environmental Protection Agency (EPA). All dental unit waterlines should be treated, shocked, and tested following CDC recommended guidance. If outside the US, standards may differ, check your local regulations on safe drinking water.

### Before Use

Dentsply Sirona recommends that you perform waterline testing on your new Cavitron scaling system prior to use on a patient.

## INDEPENDENT WATER BOTTLE/RESERVOIR CONNECTION

If connecting a Cavitron Scaling System waterline to a dental chair, where the water is delivered via an independent water bottle / reservoir, follow the dental chair manufacturer's Instructions for Use for treating and shocking your waterlines.

1. The waterline maintenance products listed below have been tested for material compatibility with Cavitron Ultrasonic Scaling Systems:

### Waterline Treatment Products

- a-dec ICX™ Tablet Waterline Treatment
- Hu-Friedy Waterline filter
- ProEdge Dental Products BluTab® waterline tablets
- Crosstex DentaPure® cartridge
- Sterisile CITRISIL™ Dental Waterline Microbial Contaminant Control Tablet
- Alpro Alpron Liquid Concentrate
- Dentsply Sirona Dentosept® S water disinfectant.

### Waterline Shock Products

- Crosstex DentaPure® cartridge

2. Follow manufacturer instructions for use for the waterline treatment products listed above.

NOTE: Ensure that Cavitron Scaling System is connected to the dental chair when performing the shock procedure recommended in the chair manufacturer's IFU.

## MUNICIPAL WATER CONNECTION

If connecting a Cavitron Scaling System waterline directly to a Municipal water valve, treat the waterlines weekly with a 1:10 Sodium Hypochlorite 5.25% Solution with the following steps.

1. Connect the Cavitron Scaling System to the Cavitron DualSelect Dispensing System following the DualSelect system's Instructions for Use.
2. Prepare a 1:10 Sodium Hypochlorite Solution by adding a 5.25% Sodium Hypochlorite to the designated line on the orange disinfectant bottle.
3. Fill remainder of disinfectant bottle with tap water
4. Place the orange disinfectant bottle in position "A".
5. Close the shield and set the Selector Knob to position "A"
6. Adjust the lavage/water control at the base of the handpiece to the maximum water flow and begin to flush the 1:10 Sodium Hypochlorite Solution through the system for at least 30 seconds, ensure that the solution is exiting the handpiece.
7. Set the Selector Knob to Open H<sub>2</sub>O position and open the shield.
8. Remove the orange disinfectant bottle from position "A" and place it in position "B" and close the shield. Switch selector knob to position "B" and operate the unit for at least 10 seconds.
9. Set selector knob to Open H<sub>2</sub>O position, open the shield, and remove the disinfectant bottle.
10. The system should be left undisturbed, allowing the prepared solution to sit in the waterline, for a minimum of 10 minutes but no more than 30 minutes.
11. As a precaution, place a sign on the scaling system indicating it is being treated and should not be used
12. After the minimum 10-minute wait time has passed, fill another bottle with clean water, place it in position "A" and close the shield. Set the Selector Knob to position "A" and operate the Cavitron System at maximum waterflow for at least 2 minutes or until all Sodium Hypochlorite odor disappears.
13. Set selector knob to Open H<sub>2</sub>O position, open the shield. Remove the clean water bottle from position "A" and place it in position "B". Close the shield and set selector knob to "B". Once again, operate the Cavitron system at maximum waterflow for at least 2 minutes, until all the water drains and/or all the Sodium Hypochlorite odor disappears.
14. Ensure that all the Sodium Hypochlorite solution has been completely flushed from the system prior to use on a patient.
15. Set the selector Knob to "Open H<sub>2</sub>O", open the shield and remove the empty bottle from position "B".
16. Place empty bottle in position "A", close the shield and set selector knob to position "A".
17. Operate Cavitron System at maximum waterflow for at least 30 seconds to air purge line dry.
18. Set selector knob to "open H<sub>2</sub>O" position, open shield and remove empty bottle for position "A" and place in position "B".
19. Operate Cavitron System at maximum waterflow for at least 30 seconds to air purge line dry.

**NOTE - ALL Sodium Hypochlorite solution MUST BE FLUSHED FROM THE SYSTEM BEFORE IT IS READY FOR PATIENT USE**

## Water Filter Maintenance Guide

WATER QUALITY:	MUNICIPAL	WELL	HARD	SOFT	BOTTLE SYSTEMS
Recommended Evaluation and/or Replacement:	1 x month	2 x month	2 x month	1 x month	2 x month
Other Considerations:					If using chemical cleaners, the system should be flushed appropriately to prevent damage to equipment. Please refer to the Cavitron Maintenance Guide for recommended procedure.

### KEY INDICATORS THAT YOUR CAVITRON WATER FILTER NEEDS TO BE REPLACED:

- Water flow is reduced to the insert and/or handpiece
- If particulates are visible on the filter
- Discoloration of the filter
- Light indicator on your unit is visible. Valid on Cavitron Touch (G1000) and 300 Series (G310) ultrasonic scaling units only.

### PLEASE NOTE:

- A 10-pack of replacement filters (part number 90158) is available from your local Dentsply Sirona distributor.
- The Cavitron water filter is not intended to be the primary water filtration for the office. It is meant to protect the Cavitron system and components.
- If your Cavitron water filter needs to be replaced on a highly frequent basis, you should consider an overall water filtration system for your office to minimize damage to your dental equipment.
- Please refer to the scheduled maintenance guide for your specific Cavitron system for complete maintenance recommendations.

### TO REPLACE YOUR CAVITRON WATER FILTER:

1. Verify that the system is turned OFF.
2. Locate the blue waterline on the back of the unit.
3. Disconnect the water supply hose from the dental office water supply. If a quick disconnect connector is attached to the end of the hose, relieve the water pressure by pressing the tip of the connector in an appropriate container and drain the water.
4. Grasp the fittings on either side of the filter disk and twist counterclockwise. Remove the filter section from either side of the water hose.
5. Install the replacement filter onto the water hose fittings. The filter should be positioned to match up with the correct hose fitting.
6. Hand tighten one hose fitting onto filter in a clockwise direction. Tighten second hose onto filter in clockwise direction. Reconnect the water supply line, operate unit to bleed the air and test for leaks.

## Air Supply Line Filter Maintenance

Water build up in the air supply line filter should be drained. This can be accomplished by turning the knob on the bottom of the filter counter-clockwise to open. After draining, turn the knob fully clockwise to close (some filters drain by depressing the Schrader valve stem at the bottom of the filter). If the inner filter element becomes discolored or dirty, a new filter assembly should be installed. Replacement filter assemblies are available by ordering Part Number 90088 from your local Dentsply Sirona distributor.

1. Verify that the system is turned OFF.
2. Disconnect the air supply line from the dental office air source.
3. Using a 7/16" wrench, loosen the nuts on the side fittings of the filter. Unscrew the nuts and slide them down the hose. Disconnect hoses from the filter and discard the used filter.
4. Insert the short hose into the input port of the filter and the long hose into the output port of the filter. Slide the nuts up the hoses and screw onto the fittings. Tighten using pliers or a wrench.
5. Turn the System ON, operate the system and check for leaks.

## Powder Bowl Maintenance

Remove residual prophy powder from the cap and bowl threads using a soft brush (toothbrush). If not removed, caked powder in threads can result in thread wear and powder bowl cap disengagement.

1. Turn the System OFF.
2. Allow the powder bowl to depressurize and unscrew the Powder Cap.
3. Empty powder from the bowl and use the high suction to remove any residual powder in the bowl.
4. Turn the System ON and check for strong air flow from the center tube of the powder bowl.
5. If no or low air flow is present, turn the System OFF.
6. Unscrew the knurled ring at the bottom of the bowl assembly and remove the fitting assembly.
7. Using the Jet Air Polishing Insert nozzle cleaning wire tool, clean clogged powder from the fitting assembly. Turn the System ON and check for strong air flow. Turn the system OFF.
8. Check that the o-ring is properly positioned in the groove of the fitting assembly and reassemble the fitting assembly to the bowl. Tighten knurled ring. Place powder bowl into System.
9. Fill the powder bowl with fresh prophy powder and test for flow and leaks.
10. Remove residual powder from thread on the cap and bowl with a soft, dry cloth.

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Dentsply LLC
Dentsply DeTrey GmbH