Cavitron Select® SPS™ System

Quick Maintenance Guide

IMPORTANT: Turn OFF water supply to Cavitron system or disconnect water quick disconnect from the water supply when not in use.

	Cavitron Select SPS Ultrasonic Scaler	Cavitron Select SPS Ultrasonic Scaler with Cavitron Select Reservoir System
Start-Up Procedures (beginning of day)	<ol style="list-style-type: none"> 1. Open the manual shut-off valve on the dental office water Supply system. 2. Install a sterilized Steri-Mate Handpiece onto the handpiece cable. 3. Set the Power Level Control to minimum and the Lavage Control to maximum. 4. Turn the system ON. 5. Hold the sterilized handpiece (without insert installed) over a sink or drain. Activate the foot control and flush the line for 2 minutes. 6. Place a sterilized 30K Cavitron Ultrasonic Insert into the handpiece and set the Power knob Control and Lavage Control to your preferred operating position for ultrasonic scaling. 	<ol style="list-style-type: none"> 1. Carefully lock Cavitron Select Reservoir Bottle into Cavitron Select Reservoir Pump. 2. Fill Cavitron Select Reservoir Bottle with tap, distilled water, or medicament. Replace Cavitron Select Bottle cap. 3. Connect Cavitron Select Scaler to Cavitron Select Reservoir Pump and Bottle (see Section 8.5 for complete instructions). 4. Open the manual shut-off valve on the dental office water supply system. 5. Install a sterilized Steri-Mate Handpiece onto the handpiece cable. 6. Set the Power Level Control to minimum and the Lavage Control to maximum. 7. Turn the system ON. 8. Hold the sterilized handpiece (without insert installed) over a sink or drain. Depress the Foot Control and flush the waterline for 2 minutes. 9. Place a sterilized 30K Cavitron Insert into the Handpiece. Set Power level knob and Lavage Control to your preferred operating position for ultrasonic scaling.
Between Patients	<ol style="list-style-type: none"> 1. Remove the used Cavitron Ultrasonic Insert. Clean and sterilize Following the Infection Control Procedures that were enclosed with your insert 2. Hold the handpiece over a sink or drain and flush the waterline of the unit as above at maximum water flow for 30 seconds. 3. Turn the System to the OFF position. 4. Remove the Steri-Mate handpiece, clean and sterilize following the procedures outlined in the Cavitron Systems Infection Control Procedures Booklet that was enclosed with your system. 5. Disinfect the surfaces of the cabinet, Power Cord, Handpiece Cable, Foot Control and Cable assembly (if applicable), Water Supply lines as instructed in the Cavitron Systems Infection Control Procedures booklet. DO NOT SPRAY DISINFECTANT SOLUTION DIRECTLY ON SYSTEM SURFACES*. 6. Inspect the handpiece cable for any breaks or tears. 7. If using a closed water supply or DualSelect Dispensing System, check for adequate fluid volumes for the next patient. 8. When ready to use, place a sterilized Steri-Mate Handpiece onto the handpiece cable assembly and insert a sterilized ultrasonic insert into the handpiece and adjust system controls as preferred. 	<ol style="list-style-type: none"> 1. Follow steps 1-6 listed in Between Patients (Cavitron Select SPS Ultrasonic Scaler). 2. With the power OFF, unscrew cap and fill Cavitron Select Reservoir Bottle with tap, distilled water, or medicament. 3. Do not overfill Bottle with liquid into the neck of the Bottle. Securely screw cap back on Bottle. 4. When ready to use, place a sterilized Steri-Mate Handpiece onto the handpiece cable assembly and insert a sterilized ultrasonic insert into the handpiece and adjust system controls as preferred.
Shut-Down Procedures (end of day)	<p>Follow the "Between Patients" maintenance procedures, Steps 1 through 6. If a quick disconnect is attached to the end of the hose:</p> <ol style="list-style-type: none"> 1. Disconnect the water supply line from the dental office water supply. 2. Relieve the water pressure by pressing the tip of the connector into an appropriate container and allow water to drain. <p>If quick-disconnect is not attached to the end of the hose, it is recommended to close the manual shut off valve on the dental water supply system.</p>	<ol style="list-style-type: none"> 1. Remove the used Cavitron Ultrasonic Insert. Clean and sterilize following the Infection Control Procedures that were enclosed with your insert. 2. Turn the System OFF. 3. Detach Cavitron Select SPS Scaler from Reservoir System. 4. Detach Cavitron Select Reservoir Bottle from Pump (see Section 8 for complete instructions). 5. Empty entire contents of Cavitron Select Reservoir Bottle. 6. Replace empty Cavitron Select Reservoir Bottle to Pump. 7. Carefully lock Cavitron Select Reservoir Bottle into Cavitron Select Reservoir Pump. 8. Connect Cavitron Select SPS Scaler to Cavitron Select Reservoir System (see Section 6 for complete instructions). 9. Turn the System ON.

	Cavitron Select SPS Ultrasonic Scaler	Cavitron Select SPS Ultrasonic Scaler with Cavitron Select Reservoir System
Shut-Down Procedures (end of day), continued		<ol style="list-style-type: none"> 10. Hold the Handpiece (without the Cavitron Insert installed) upright over a sink or drain. 11. To air purge the waterline, operate the Cavitron Select SPS Scaler for at least 30 seconds. (this will eliminate fluid to prevent it from settling in the Cavitron Select SPS Scaler for extended periods of time. This may also help decrease the chance of bio-film formations in the Cavitron Select SPS waterlines.) 12. Turn the System OFF.
Extended periods of closure	Refer to "Shut-Down Procedures (end of day)" maintenance, in addition: <ol style="list-style-type: none"> 1. Remove the hose from the system, by pushing on the outer ring of the system's water inlet and gently pull out the waterline 2. As an additional precaution, blowing air through the waterline (using an air/water syringe) may also be help when preparing for extended inactive use of a Cavitron Ultrasonic Scaling System 	
Water Maintenance Instructions	<p>Dental Unit Waterline Overview:</p> <p>Dental Unit Waterline Maintenance is a key component to a facility's effective Infection Control and Prevention Protocol. The Centers for Disease Control and Prevention (CDC) recommends that dental unit water used in nonsurgical procedures measure less than or equal to 500 colony forming units of heterotrophic bacteria per milliliter (≤ 500 CFU/mL) of water, the standard set for drinking water by the Environmental Protection Agency (EPA). All dental unit waterlines should be treated, shocked, and tested following CDC recommended guidance. If outside the US, standards may differ, check your local regulations on safe drinking water.</p> <p>Before Use:</p> <p>Dentsply Sirona recommends that you perform waterline testing on your new Cavitron scaling system prior to use on a patient.</p>	
	INDEPENDENT WATER BOTTLE/ RESERVOIR CONNECTION	CAVITRON SELECT SPS RESERVOIR CONNECTION
	<p>If connecting a Cavitron Scaling System waterline to a dental chair, where the water is delivered via an independent water bottle / reservoir, follow the dental chair manufacturer's Instructions for Use for treating and shocking your waterlines.</p> <ol style="list-style-type: none"> 1. The waterline maintenance products listed below have been tested for material compatibility with Cavitron Ultrasonic Scaling Systems: <ul style="list-style-type: none"> Waterline Treatment Products a-dec ICX™ Tablet Waterline Treatment Hu-Friedy Waterline filter ProEdge Dental Products BluTab® waterline tablets Crosstex DentaPure® cartridge Sterisile CITRISIL™ Dental Waterline Microbial Contaminant Control Tablet Alpro Alpron Liquid Concentrate Dentsply Sirona Dentosept® S water disinfectant. Waterline Shock Products Crosstex DentaPure® cartridge 2. Follow manufacturer instructions for use for the waterline treatment products listed above. <p>NOTE: Ensure that Cavitron Scaling System is connected to the dental chair when performing the shock procedure recommended in the chair manufacturer's IFU.</p>	<ol style="list-style-type: none"> 1. Follow Steps listed in "Shut-Down Procedures (end of day)" (Cavitron Select SPS Ultrasonic Scaler with Cavitron Select Reservoir System) section then: 2. Prepare a 1:10 Sodium Hypochlorite Solution by mixing 1 part 5.25% Sodium Hypochlorite with 10 parts tap water. 3. Remove cap and pour the 1:10 Sodium Hypochlorite Solution into the Cavitron Select SPS Reservoir Bottle. Do not overfill with solution (see Section 4.2). 4. Securely replace cap on Bottle. 5. Turn the System ON. Adjust the lavage/water control at the base of the handpiece to the maximum water flow and begin to flush the 1:10 Sodium Hypochlorite Solution through the system for at least 30 seconds. 6. Turn the System OFF. The system should be left undisturbed, allowing the prepared solution to sit in the waterline for a minimum of 10 minutes but no more than 30 minutes. It is recommended that a sign be placed on the scaling system: Do Not Use System is Being Treated 7. After the minimum 10-minute wait time, unscrew cap and discard disinfectant solution. 8. Fill Cavitron Select Reservoir Bottle with tap or distilled water. Do not overfill with liquid (see Section 4.2). 9. Turn the System ON. Flush clean water through the system for 2 minutes or until all Sodium Hypochlorite odor disappears. 10. Turn off system. Detach Cavitron Select SPS scaler from Select Reservoir System. 11. Detach Cavitron Select Reservoir Bottle from Pump (see Section 8). 12. Autoclave Cavitron Select Reservoir Bottle per Infection Control Procedure booklet supplied with your system. <p>Ensure that all the Sodium Hypochlorite solution has been completely flushed from the system prior to use on a patient.</p>
	MUNICIPAL WATER CONNECTION	
<p>If connecting a Cavitron Scaling System waterline directly to a Municipal water valve, treat the waterlines weekly with a 1:10 Sodium Hypochlorite 5.25% Solution with the following steps.</p> <ol style="list-style-type: none"> 1. Connect the Cavitron Scaling System to the Cavitron DualSelect Dispensing System following the DualSelect system's Instructions for Use. 2. Prepare a 1:10 Sodium Hypochlorite Solution by adding a 5.25% Sodium Hypochlorite to the designated line on the orange disinfectant bottle. 3. Fill remainder of disinfectant bottle with tap water 		

<p>Water Maintenance Instructions, continued</p>	<p>Cavitron Select SPS Ultrasonic Scaler</p> <ol style="list-style-type: none"> 4. Place the orange disinfectant bottle in position "A". 5. Close the shield and set the Selector Knob to position "A" 6. Adjust the lavage/water control at the base of the handpiece to the maximum water flow and begin to flush the 1:10 Sodium Hypochlorite Solution through the system for at least 30 seconds, ensure that the solution is exiting the handpiece. 7. Set the Selector Knob to Open H₂O position and open the shield. 8. Remove the orange disinfectant bottle from position "A" and place it in position "B" and close the shield. Switch selector knob to position "B" and operate the unit for at least 10 seconds. 9. Set selector knob to Open H₂O position, open the shield, and remove the disinfectant bottle. 10. The system should be left undisturbed, allowing the prepared solution to sit in the waterline, for a minimum of 10 minutes but no more than 30 minutes. 11. As a precaution, place a sign on the scaling system indicating it is being treated and should not be used 12. After the minimum 10-minute wait time has passed, fill another bottle with clean water, place it in position "A" and close the shield. Set the Selector Knob to position "A" and operate the Cavitron System at maximum waterflow for at least 2 minutes or until all Sodium Hypochlorite odor disappears. 13. Set selector knob to Open H₂O position, open the shield. Remove the clean water bottle from position "A" and place it in position "B" . Close the shield and set selector knob to "B". Once again, operate the Cavitron system at maximum waterflow for at least 2 minutes, until all the water drains and/or all the Sodium Hypochlorite odor disappears. <p>NOTE: Ensure that all the Sodium Hypochlorite solution has been completely flushed from the system prior to use on a patient.</p>	<p>Cavitron Select SPS Ultrasonic Scaler with Cavitron Select Reservoir System</p>																		
<p>Waterline Filter Maintenance</p>	<p>Your Cavitron system is equipped with a waterline filter that needs to be evaluated and replaced periodically.</p> <table border="1" data-bbox="342 1314 1544 1610"> <thead> <tr> <th data-bbox="342 1314 534 1392">WATER QUALITY:</th> <th data-bbox="534 1314 699 1392">MUNICIPAL</th> <th data-bbox="699 1314 841 1392">WELL</th> <th data-bbox="841 1314 984 1392">HARD</th> <th data-bbox="984 1314 1117 1392">SOFT</th> <th data-bbox="1117 1314 1544 1392">BOTTLE SYSTEMS</th> </tr> </thead> <tbody> <tr> <td data-bbox="342 1392 534 1478">Recommended Evaluation and/or Replacement:</td> <td data-bbox="534 1392 699 1478">1 x month</td> <td data-bbox="699 1392 841 1478">2 x month</td> <td data-bbox="841 1392 984 1478">2 x month</td> <td data-bbox="984 1392 1117 1478">1 x month</td> <td data-bbox="1117 1392 1544 1478">2 x month</td> </tr> <tr> <td data-bbox="342 1478 534 1610">Other Considerations:</td> <td data-bbox="534 1478 699 1610"></td> <td data-bbox="699 1478 841 1610"></td> <td data-bbox="841 1478 984 1610"></td> <td data-bbox="984 1478 1117 1610"></td> <td data-bbox="1117 1478 1544 1610">If using chemical cleaners, the system should be flushed appropriately to prevent damage to equipment. Please refer to the Cavitron Maintenance Guide for recommended procedure.</td> </tr> </tbody> </table> <p>Key indicators that your Cavitron water filter needs to be replaced:</p> <ul style="list-style-type: none"> • Water flow is reduced to the insert and/or handpiece • If particulates are visible on the filter • Discoloration of the filter <p>PLEASE NOTE:</p> <ul style="list-style-type: none"> • A 10-pack of replacement filters (part number 90158) is available from your local Dentsply Sirona distributor. • The Cavitron water filter is not intended to be the primary water filtration for the office. It is meant to protect the Cavitron system and components. • If your Cavitron water filter needs to be replaced on a highly frequent basis, you should consider an overall water filtration system for your office to minimize damage to your dental equipment. • Please refer to the scheduled maintenance guide for your specific Cavitron system for complete maintenance recommendations. 		WATER QUALITY:	MUNICIPAL	WELL	HARD	SOFT	BOTTLE SYSTEMS	Recommended Evaluation and/or Replacement:	1 x month	2 x month	2 x month	1 x month	2 x month	Other Considerations:					If using chemical cleaners, the system should be flushed appropriately to prevent damage to equipment. Please refer to the Cavitron Maintenance Guide for recommended procedure.
WATER QUALITY:	MUNICIPAL	WELL	HARD	SOFT	BOTTLE SYSTEMS															
Recommended Evaluation and/or Replacement:	1 x month	2 x month	2 x month	1 x month	2 x month															
Other Considerations:					If using chemical cleaners, the system should be flushed appropriately to prevent damage to equipment. Please refer to the Cavitron Maintenance Guide for recommended procedure.															

	Cavitron Select SPS Ultrasonic Scaler	Cavitron Select SPS Ultrasonic Scaler with Cavitron Select Reservoir System
Pump Lavage Filter Maintenance	Not applicable	<p>When the pump lavage filter becomes discolored, the filter should be replaced to prevent reduced water flow to the Cavitron Select SPS System. A 10-pack of replacement filters is available by ordering Part Number 90191 from your local Dentsply Sirona distributor.</p> <ol style="list-style-type: none"> 1. Empty the contents of the Cavitron Select Reservoir bottle and disconnect it from the pump. 2. Remove the old filter and discard. 3. Insert the small end of the new filter until filter is seated.
Handpiece Cable	<p>While the Cavitron Handpiece Cables are designed to offer optimum flexibility and endurance, improper handling may lead to a shortened useful life of the cable assembly, due to excessive wear on the wires and/or internal tubing. To prevent premature failure of the cable assembly, please use the following steps:</p> <ol style="list-style-type: none"> 1. Do not wrap the cable tightly around the Cavitron system or any apparatus. 2. Avoid overstretching or pulling the cable. 3. If possible, minimize frequent bending/folding that exceeds normal usage during dental procedure. 4. For detachable handpiece cables, hold by the plug body when detaching from the Cavitron system. 5. Gently loop the cable in a continuous loose circle after each use. 	
O-Ring Maintenance	<p>Before placing Cavitron Inserts into the handpiece, performing the following steps will increase the O-ring life on Cavitron Inserts:</p> <ul style="list-style-type: none"> • Fill entire handpiece with water. • Lubricate O-ring (black or green rubber material) with water. • Gently twist the insert down into the handpiece until the insert is fully seated in place. 	

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