

Palodent® 360

Circumferential Matrix System

TIPS AND TRICKS



Size availability

Choose a band that slightly exceeds the height of the occlusal circumference.



Tight contacts?


Use a Palodent® Plus retaining ring or pre-wedge to create separation before placing the matrix.

Small teeth

Pre-tighten the band prior to placement to minimize excess slack.

Thumbwheel position

Place the thumbwheel anywhere along the secure tooth wall to achieve the best field of view and accessibility.

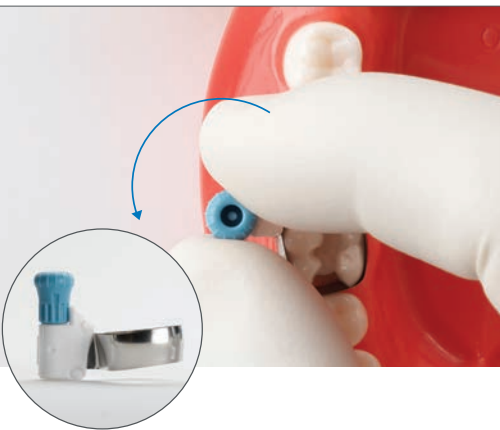

Where space or grip is limited

Grasp the thumbwheel and move the white retaining body forward in a ratcheting motion. Release the thumbwheel, allowing the retaining body to return back to center. Repeat until the band is snug.



Palodent® products are designed to work together

Use a Palodent® Plus ring and/or wedges to further stabilize the band and create the desired form.



Removing the matrix

Reverse rotation on the thumbwheel (counterclockwise) and wiggle slightly back and forth to loosen matrix band.