

Document Detail

Type: 0060-GR DT
Document No.: 0060-GR0021116LI[1]
Title: Purevac SC Dispenser Visual DFU
Comment
Status: CURRENT
Effective Date: 08-Jun-2018

Approvals

<u>Role</u>	<u>Sign-off By</u>	<u>Sign-off Date</u>	
DPD Document Control	Fred Elslager	08-Jun-2018 2:10 pm	GMT
DPD Research and Development	Michael Stephenson	08-Jun-2018 2:11 pm	GMT
DPD Operations-Manufacturing	Matthew Bahoric	08-Jun-2018 2:12 pm	GMT
DPD Quality Engineer	Maeve Bowers	08-Jun-2018 2:12 pm	GMT
DPD Marketing	Kyle Muse	08-Jun-2018 2:11 pm	GMT
DPD ERP Coordinator	Natalie Weiss	08-Jun-2018 2:17 pm	GMT
0060-Preventive UDI Approver	Donata Kline	08-Jun-2018 3:20 pm	GMT
DPD Quality Assurance/Regulatory Management	Donata Kline	08-Jun-2018 3:21 pm	GMT
DPD Document Control	Fred Elslager	08-Jun-2018 6:36 pm	GMT

Reference

<u>Document No.</u>	<u>Title</u>	<u>Relation</u>
0060-0021116FG	Purevac SC Evacuation System Cleaner Dispenser	Master Document

Artwork Specification

Rev. 1

Part Number:	0021116LI
Title	Purevac SC Evacuation Cleaner Dispenser (Quick Start Guide)
Description	Instructions for Use
Material:	70# smooth white
Trim Size:	8.5" X 11"
Binding:	NA
Colors:	4 Color
Finished Size:	8.5" x 11"
Languages:	English
Graphics Revision	Rev. 1 (05/18)

Notes:

1. Package to prevent damage.
2. Artwork is maintained by DENTSPLY Professional, York PA 17404

Revision History:


Rev	Description of change	Author	Dept	CHO
1	Initial release	D. Kierce	Marketing	CHO-105352

Purevac® SC Evacuation System Cleaner Dispenser



Instructions For Use

It's even easier to dispense Purevac SC Evacuation System Cleaner with the Purevac SC Evacuation System Cleaner Dispenser. It creates a turbulent mixture of air and Purevac SC solution to clean out your evacuation line system in dental practices with or without an amalgam separator. A convenient handle allows you to transport it to multiple operatories.

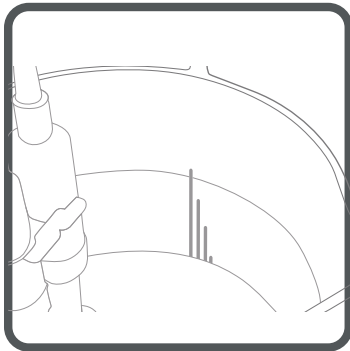


1. Fill the Purevac SC Evacuation System Cleaner Dispenser with room temperature tap water up to the appropriate fill marker on the inside of the unit. Use one (1) quart or one (1) liter per operator, plus one (1) quart or one (1) liter for cuspidors.

NOTE: FIRST TIME USERS SHOULD ADD 2 OZ (60 mL) OF PUREVAC SC TO ONE QUART (1 LITER) OF WATER.




2. Add Purevac SC Evacuation System Cleaner. Pump 1 oz (30 mL) of Purevac SC solution per quart or liter of water into the dispenser. 1 oz (30 mL) = 1 pump.

3. For suction lines with US fittings, attach the enclosed adaptors onto the plastic fittings on the dispenser. These adaptors are not required for metric-sized fittings.

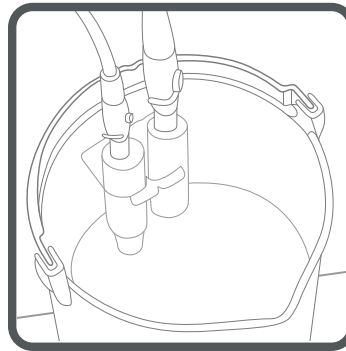

4. Place the high volume evacuation and saliva ejectors onto the plastic fittings.

5. Turn the suction lines on. Aspirate solution until the next fill marker is visible on the dispenser. Turn off the vacuum lines, remove them from the dispenser fittings and return them to the hangers. Turn the suction back on to both lines. Keep suction on in first operator and repeat steps 3 and 4 for each remaining operator.

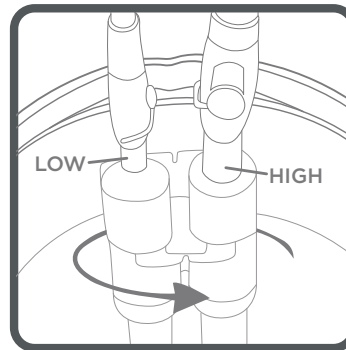

6. Turn off the suction in the first operator.

7. Pour any remaining solution to clean cuspidors. Repeat the process for additional operatories.

For more information: dentsplysirona.com/preventive.