

ACHIEVING ZEN DURING A ROOT CANAL? CHALLENGE ACCEPTED.



Shaping • Cleaning • Obturation

WaveOne® Gold

EXPERIENCE THE CALMING CONFIDENCE OF WAVEONE® GOLD

Root canals can be challenging. But the WaveOne® Gold treatment solution has everything you need to approach your next case with calmness and tranquility. The result? An endodontic treatment that is a simpler (and more profitable) part of your practice. So go ahead – take on the next root canal case with confidence.

A comprehensive treatment solution in 3 simple steps.



SHAPING

Safe shaping in half the time.¹ A short sequence for most of your cases including 1 glidepath and 1 shaping file. Crafted to be more forgiving, due to reciprocating motion and gold heat treatment.²



CLEANING

Efficient cleaning. That's what the irrigation needle as well as the EndoActivator® are designed for.




OBTURATION

Biocompatible seal and optimal fit. No matter if you prefer a warm or cold obturation technique our biocompatible Bioceramic Sealer supports both. Enjoy the ease with Conform Fit® GuttaPercha (cold) or GuttaCore® (warm).



DS MOTORS

Because every angle, every acceleration, every speed matters to deliver the best, our motors provide the genuine reciprocating movement for WaveOne® Gold.

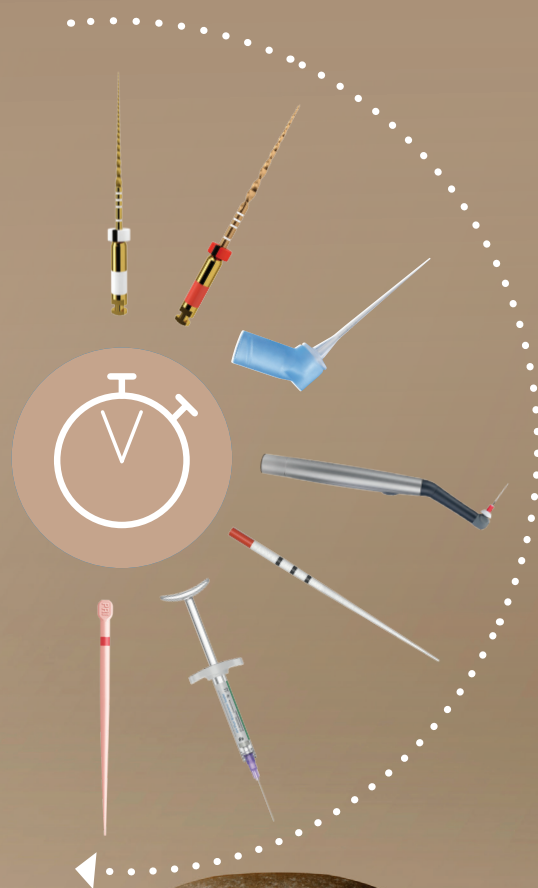

1. vs. ProTaper® Universal
2. vs. conventional systems

SHAPING

CHALLENGE: SAFE SHAPING IN HALF THE TIME.¹



**One short sequence
for most of your cases²**



Simple to use and easy to adopt

- Short learning curve³
- Good control of instrument progression⁴



Time-saving

- Halves shaping time¹
- Zeros out reprocessing & documentation
- Optimal cutting efficiency⁵



Safety. A priority.

- Forgiving due to reciprocating motion and gold heat treatment
- Up to 200 scientific studies
- More than 30 million treated cases

WaveOne® Gold
Glider


015

2%

WaveOne® Gold
Primary file

025

7%

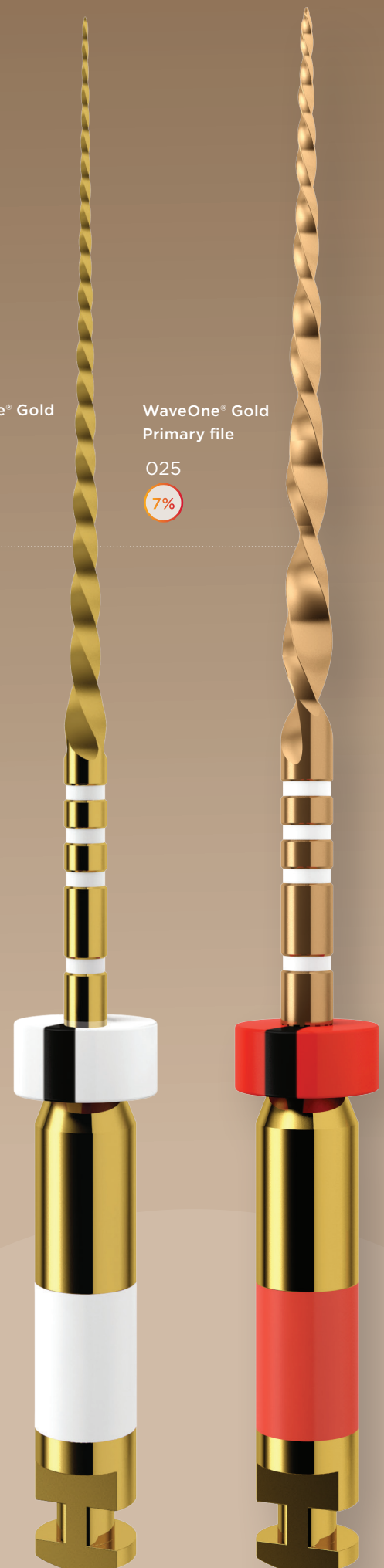

BUILDING ON
MAILLEFER
HERITAGE

1. vs. ProTaper Universal

2. Single glide path file and single shaping file in most of the cases

3. User evaluation

4. vs. WaveOne

5. Higher cutting efficiency than WaveOne and cutting efficiency preserved thanks to single-use

CLEANING

CHALLENGE:
**EFFICIENT
CLEANING.**

IRRIGATION NEEDLE




Efficient

- Advanced technology with two lateral openings aimed directly at the dentinal walls
- Soft polypropylene allows the needle to curve and to deliver irrigation solution close to the apex



Safe


- Closed end reduces risk of extrusion at the apex
- Plastic body makes it atraumatic for hard tissues

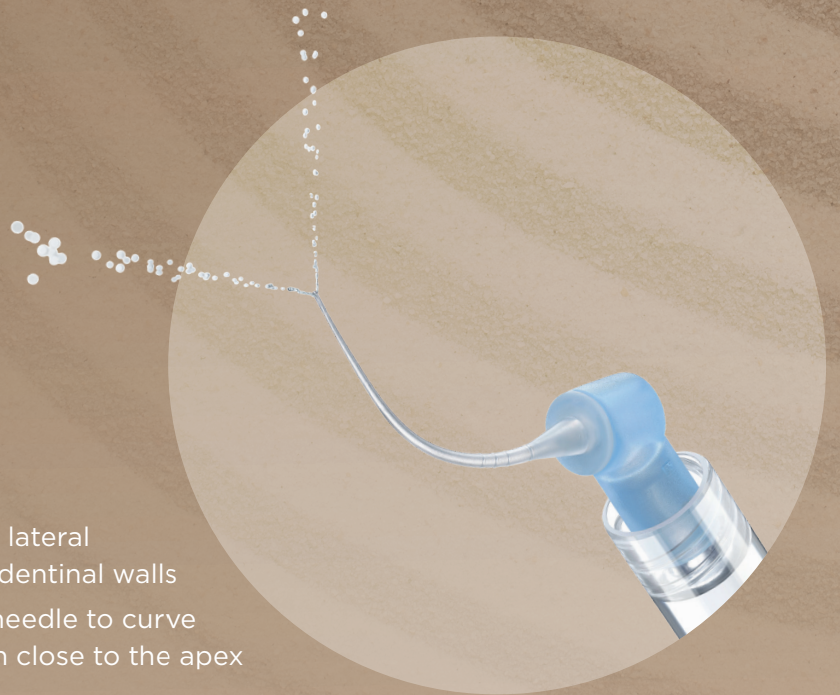


ENDOACTIVATOR®

More opened dentinal tubules

Passive soaks aren't enough. Activating irrigant solution produces a surface with more opened dentinal tubules, providing better obturation of lateral and accessory canals.⁶

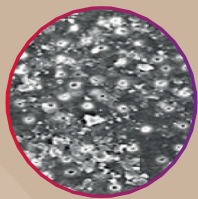

Unactivated
dentin surface

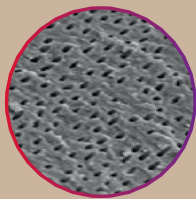

After irrigant
activation

SmartLite Pro EndoActivator™

COMING
SOON



6. A quantitative and qualitative analysis of ultrasonic versus sonic endodontic systems on canal cleanliness and obturation Valerie Kanter, DMD, Emily Weldon, DMD, Uma Nair, DMD, MDS, Claudio Varella, DDS, MS, Keith Kanter, DDS, Kenneth Anusavice, PhD, and Roberta Pileggi, DDS, MS, Gainesville, FL UNIVERSITY OF FLORIDA. (Oral Surg Oral Med Oral Pathol Oral Radiol Endod 2011;112:809-813). Support Claim 4.

OBTURATION

CHALLENGE:

BIOCOMPATIBLE SEAL AND OPTIMAL FIT.

AH PLUS® BIOCERAMIC SEALER



Biocompatible

Providing an ideal environment for Hydroxyapatite formation paving the way for the body's self healing.⁷



Adapting to your preference

Compatible with cold and warm obturation techniques.



Cold Obturation

CONFORM FIT® GUTTA-PERCHA



- Take advantage of the unsurpassed fit of the injection-moulded Conform Fit® Gutta-Percha points that match the unique shapes created by our WaveOne® Gold files.⁷
- Convenient tab and color markings for easy identification.

Warm Obturation

GUTTACORE®

- The confidence of a 3D fill, with the ease of a single insertion.
- Our obturators feature a cross-linked Gutta-percha core that sends the surrounding warm, flowable gutta-percha three-dimensionally to follow the natural canal anatomy.

Superior apical gutta-percha point filling volume⁷

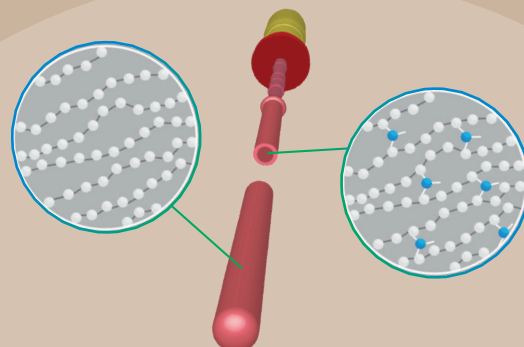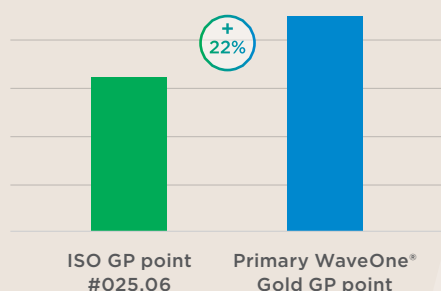


⁷ Internal data on file. For more information, contact Consumables-Data-Requests@dentsplysirona.com

WaveOne® Gold

Product References

WaveOne® Gold Reciprocating Glide Path File

Glider - 015, 02 variable taper

	3 x 21 mm	B ST WIGG 321 015
	3 x 25 mm	B ST WIGG 325 015
	3 x 31 mm	B ST WIGG 331 015
	6 x 21 mm	B ST WIGG 621 015
	6 x 25 mm	B ST WIGG 625 015
	6 x 31 mm	B ST WIGG 631 015

WaveOne® Gold Reciprocating Shaping Files

Small - 020, 07 variable taper

	3 x 21 mm	A 0753 221 G0S 03
	3 x 25 mm	A 0753 225 G0S 03
	3 x 31 mm	A 0753 231 G0S 03
	6 x 21 mm	A 0756 221 G0S 03
	6 x 25 mm	A 0756 225 G0S 03
	6 x 31 mm	A 0756 231 G0S 03

Primary - 025, 07 variable taper

	3 x 21 mm	A 0753 221 G0P 03
	3 x 25 mm	A 0753 225 G0P 03
	3 x 31 mm	A 0753 231 G0P 03
	6 x 21 mm	A 0756 221 G0P 03
	6 x 25 mm	A 0756 225 G0P 03
	6 x 31 mm	A 0756 231 G0P 03

Medium - 035, 06 variable taper

	3 x 21 mm	A 0753 221 G0M 03
	3 x 25 mm	A 0753 225 G0M 03
	3 x 31 mm	A 0753 231 G0M 03
	6 x 21 mm	A 0756 221 G0M 03
	6 x 25 mm	A 0756 225 G0M 03
	6 x 31 mm	A 0756 231 G0M 03




Large - 045, 05 variable taper

	3 x 21 mm	A 0753 221 G0L 03
	3 x 25 mm	A 0753 225 G0L 03
	3 x 31 mm	A 0753 231 G0L 03
	6 x 21 mm	A 0756 221 G0L 03
	6 x 25 mm	A 0756 225 G0L 03
	6 x 31 mm	A 0756 231 G0L 03

WaveOne® Gold Reciprocation Shaping Files (Cont.)



Assortment Reciprocating File

Small/Primary/Medium/Large

	4 x 21 mm	A 0754 221 G9A 03
	4 x 25 mm	A 0754 225 G9A 03
	4 x 31 mm	A 0754 231 G9A 03

WaveOne® Gold Blister Sequence:

k-file 10, WaveOne Gold Glider, WaveOne Gold Primary

	3 x 25 mm	B ST W1GS 325 AST
		
		

Irrigation Needle 30 Gauge

	x5	B NS DSIN 522 30G
	x40	B NS DSIN D22 30G

Paper Points for WaveOne® Gold

Box of 180 x Sterile Paper Points

	A 175W 000 00S 03 Small
	A 175W 000 00P 03 Primary
	A 175W 000 00M 03 Medium
	A 175W 000 00L 03 Large
	A 175W 000 AST 03 Assortment

Conform Fit® Gutta-Percha Points for WaveOne® Gold

Box of 60 x Conform Fit Gutta Percha Points

	B 00 WGGP F00 SML Small
	B 00 WGGP F00 PRI Primary
	B 00 WGGP F00 MED Medium
	B 00 WGGP F00 LRG Large
	B 00 WGGP F00 AST Assortment

GuttaCore® for WaveOne® Gold


Blister (5 obturators + 1 size verifier)

	B NS GCWG 500 SML Small
	B NS GCWG 500 PRI Primary
	B NS GCWG 500 MED Medium
	B NS GCWG 500 LRG Large


Box of 5 Blisters

	B NS GCWG C00 SML Small
	B NS GCWG C00 PRI Primary
	B NS GCWG C00 MED Medium
	B NS GCWG C00 LRG Large


SmartLite Pro EndoActivator™

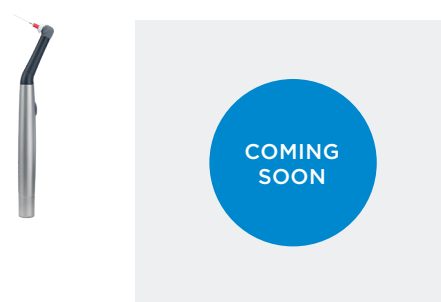




Oven

	230V EU	B 00TH PR230V EU
	230V UK	B 00TH PR230V UK
	115V US	B 00TH PR115V US

WaveOne® Gold Treatment Solution Kit

	B 00 W1GO TSO KIT
	4 WaveOne® Gold Sequence Blister
	2 WaveOne® Gold Glider 3-files Sterile 25mm
	2 WaveOne® Gold Small 3-files Sterile 25mm
	2 WaveOne® Gold Primary 3-files Sterile 25mm
	2 WaveOne® Gold Medium 3-files Sterile 25mm
	1 WaveOne® Gold Large 3-files Sterile 25mm
	1 WaveOne® Gold Conform-Fit™ Gutta-Percha Assortment
	1 WaveOne® Gold Paper Points Sterile - Primary
	2 Irrigation Needle (5 units per box)
	1 WaveOne® Gold Tip Card

			
Files	Maillefer Instruments Holding Sàrl Chemin du Verger 3, CH-1338 Ballaigues	Dentsply DeTrey GmbH De-Trey-Straße 1, D-78467, Konstanz	2797
Gutta-Percha & Absorbent Points	Dentsply Tulsa Dental Specialties 608 Rolling Hills Drive, Johnson City, TN 37604	Dentsply DeTrey GmbH De-Trey-Straße 1, D-78467, Konstanz	2797
Irrigation Needle	Produits Dentaires SA Rue des Bosquets 18, CH-1800 Vevey	PD Dental EU 74200 Thonon-les-Bains, France	1639
AH Plus® Bioceramic Sealer	Maruchi , 2-208, Medical Industry Complex Bldg., 42-10, Taejanggongdan-gil, Wonju-si, Gangwon-do, KR-26311	Biogeri Am Pfeilshof 12, DE-22303 Hamburg	0197

