Retainer Brite® Dental Appliance Cleaner

ENG

INSTRUCTIONS FOR USE

INTENDED USE:

Product is intended to clean a variety of removable dental appliances.

INDICATIONS FOR USE:

Product is indicated for patients who are using removable dental appliances such as retainers, aligners, dentures, sports mouthguards, nightguards, bruxing and TMJ appliances, which need to be cleaned at home.

PERFORMANCE CHARACTERISTICS AND CLINICAL BENEFITS:

Retainer Brite® Cleaning Tablets are specially formulated to help protect removable appliances by helping remove plaque and tartar, killing common odor-causing bacteria* and preserving clarity and cleanliness of the appliance.

Clinical benefits of orthodontic treatment may include overall improvement of tooth health and correction of malocclusion.

CONTRAINDICATIONS:

This product is contraindicated for patients allergic to Persulfates.

WARNINGS:

Contains persulfates which are a known allergen.

Keep out of reach of children.

Use only as directed.

If product comes into contact with eyes, wash eyes thoroughly with water for several minutes.

Do not put tablets or cleaning solution in mouth. If ingested, seek medical assistance immediately. In the USA, please call Poison Control Center at 1-800-222-1222. In the UK, please call NHS 111.

Report serious incidents that have occurred in relation to the device to device manufacturer and the Competent Authority in Member State of residence, or other applicable regulatory agency.

PRECAUTIONS:

Do not allow wire appliances to soak for more than 15 minutes. Remove promptly and rinse thoroughly with water.

Wash hands thoroughly before and after handling tablets.

Do not mix with other cleaners.

DO NOT REUSE, once dissolved tablet cannot be reused. Solution will not clean as intended if reused.

Store in a cool dry place.

ADVERSE REACTIONS:

Allergic reactions.

STEP BY STEP INSTRUCTIONS:

Use daily for best results.

1

Drop one tablet into enough warm (not hot!) water to cover appliance.

2

Place appliance into bubbling blue solution.

3

After 15 minutes** appliance is clean and odor free!

4

Rinse appliance and hands thoroughly with running water.

**Do Not allow wire appliances to soak more than 15 minutes. Remove promptly and rinse thoroughly with water.

SAFE DISPOSAL OF THE DEVICE:

Dispose of down the drain, as you would your mouthwash or toothpaste. Flush drain with plenty of running water.

*Kills 99% of common odor-causing bacteria including: Staph aureus, E. coli, P. aeruginosa and F. nucleatum. Data on file. Independent 2016 U.S. lab testing (ASTM method E2315-03).

INGREDIENTS: Sodium Bicarbonate, Citric Acid, Sodium Carbonate, Potassium Persulfate Compound, Corn Syrup Solids, Sodium Percarbonate, Sodium Sulfate (may contain silica), Sorbitol, Tetraacetylenediamine (TAED), PEG-180, Sodium Lauryl Sulfoacetate, Flavor, PEG-8, Magnesium Stearate, FD&C Blue #1 Aluminum Lake (CI 42090), FD&C Blue #2 (CI 73015)

Manufacturer: **Dentsply Sirona Orthodontics Inc.**
7290 26th Court East Sarasota, FL 34243 USA

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