Neu ab:
New as of:
Nouveau depuis: 05.2022
Nuevo a partir de:
Nuovo a partire da:

Sidexis 4 Viewer

Version 4.4

Installationsanleitung


Installation instructions

Instructions d'installation

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Deutsch / English / Français / Español / Italiano

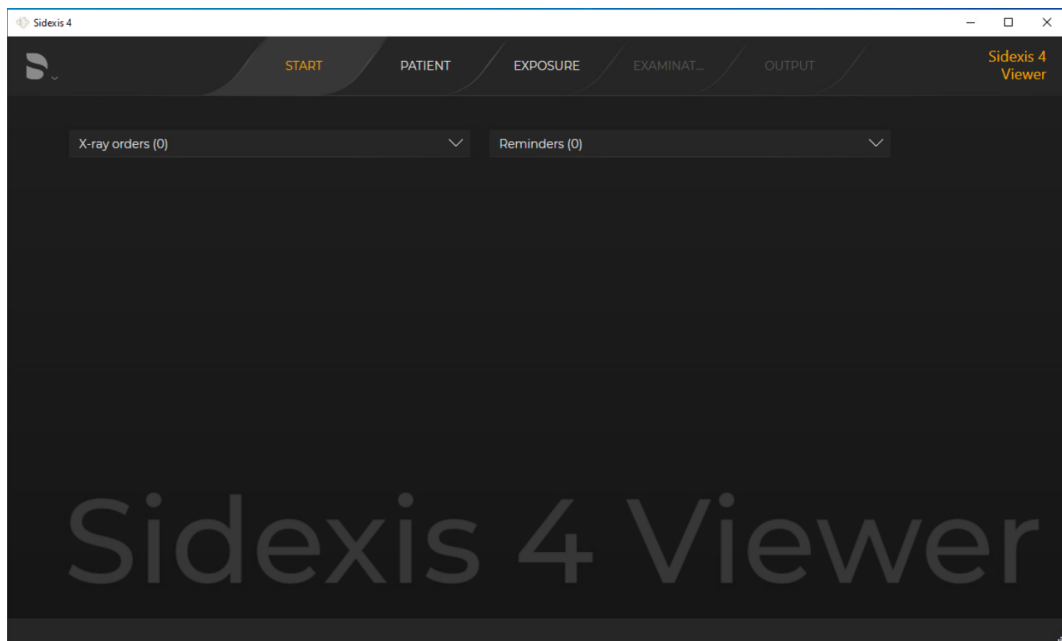Neu ab:

05.2022


Sidexis 4 Viewer

Version 4.4

Installationsanleitung

Deutsch

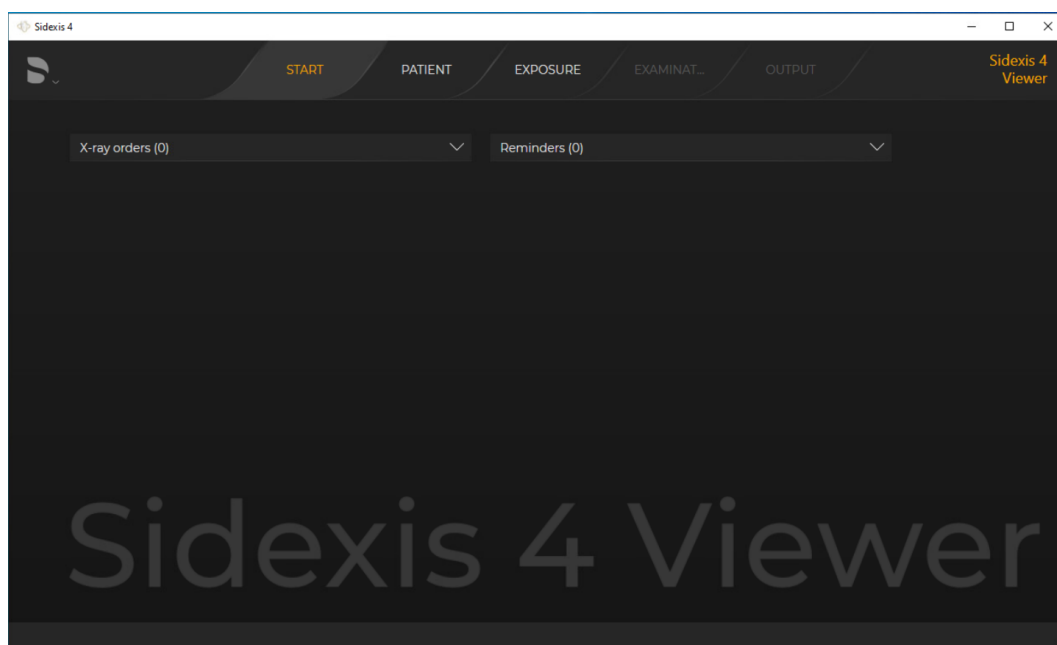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1 The Sidexis 4 Viewer


The Sidexis 4 Viewer is used to examine X-ray images and findings from Sidexis 4 if the full version of Sidexis 4 is not available.

The images are examined in the same way as with Sidexis 4, but the functionality is severely limited in comparison with the full version. The following functions are not available in Sidexis 4 Viewer:

- Exporting media
- Connecting to any type of unit (X-ray units, film scanners, treatment centers)
- Deleting patients and media
- Connecting to a practice management system (via SLIDA interface)
- Connecting to a clinical PACS
(compatibility with Sidicom WLS, Sidicom QR, Sidicom PS)

2 No Sidexis 4 Viewer or Sidexis 4 installed


- ✓ Neither Sidexis 4 nor Sidexis 4 Viewer are installed on the computer.
- 1. Insert the CD/DVD into the drive of your computer.
 - ↳ The content of the CD/DVD is shown in the directory.
- 2. Click on the file "Start Sidexis 4 Viewer.exe".
 - ↳ If no "Microsoft.NET Framework" is installed on your computer, a dialog box will open.
In this case click on the "Accept and install" button and, once installation is complete, click on the "OK" button.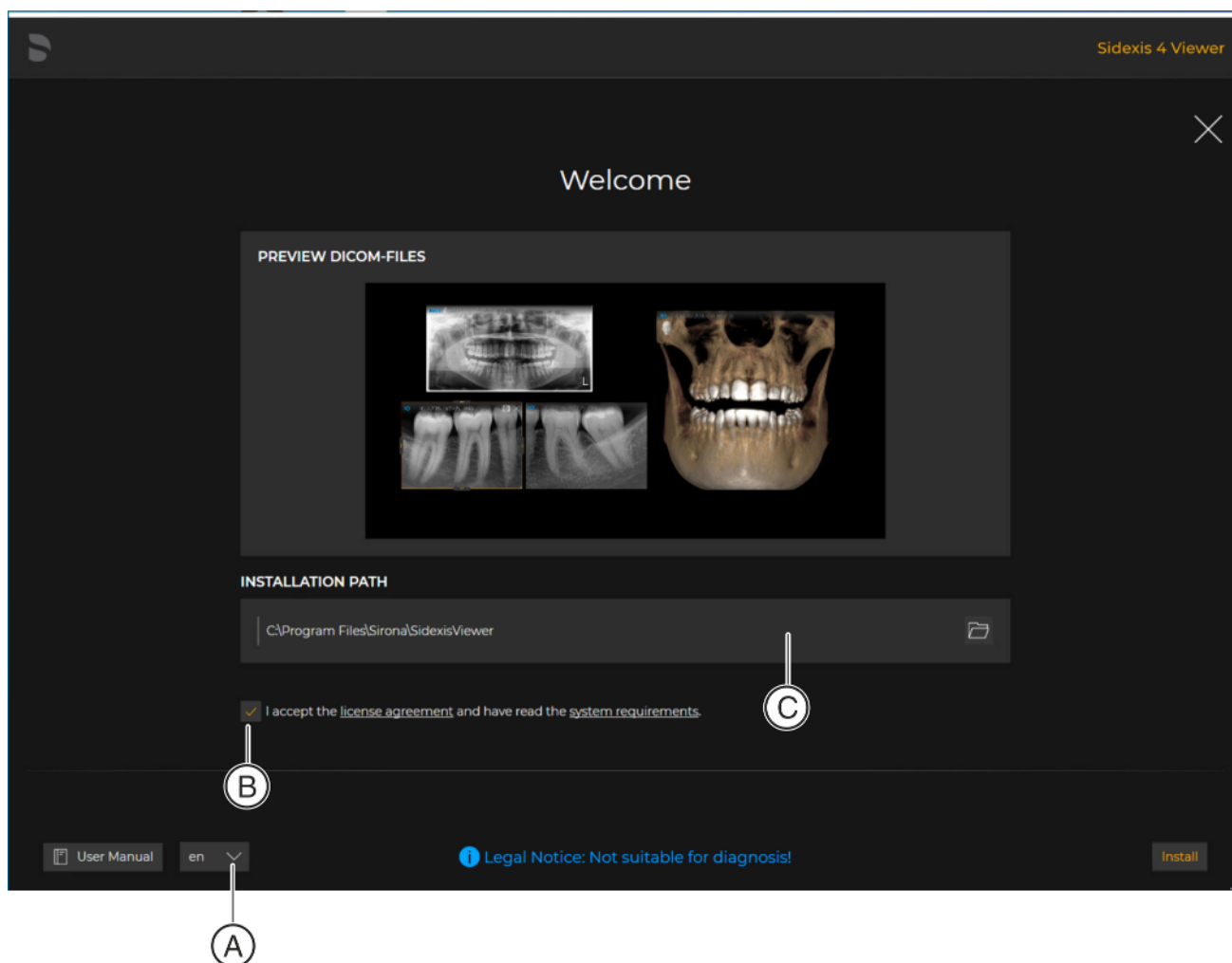

Installation menu of the Sidexis 4 Viewer

- ↳ The Sidexis 4 Viewer installation is prepared, and the installation menu opens.
In the installation menu, you are shown a preview of the data.
- 3. If desired, you can change the language via the drop-down list (A).
- 4. Click on the "System requirements" link and check them.
- 5. Click on the "License conditions" link. Read through the license conditions for Sidexis 4 Viewer and, if you accept the conditions, click on the checkbox (B).

6. If desired, you can also change the installation path by entering it in the input window (C) or selecting the installation path by clicking on the folder icon.
7. Start the installation by clicking on the *"Install"* button.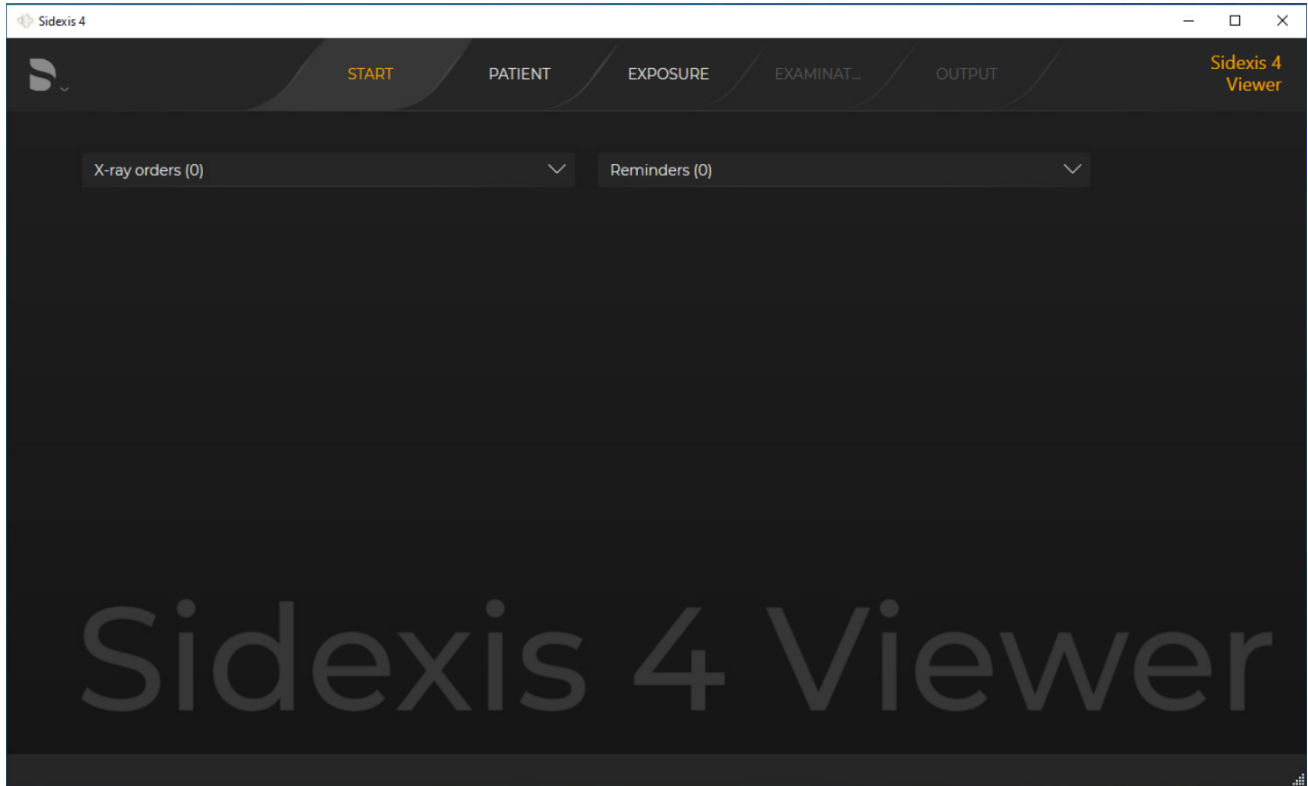


Sidexis 4 Viewer

- ↳ Sidexis 4 Viewer is installed and opened.
- ↳ After Sidexis 4 Viewer has opened, the import of the data records provided starts automatically.
IMPORTANT: This automatic import can take several minutes.


3 Sidexis 4 Viewer or Sidexis 4 available

- ✓ Sidexis 4 or Sidexis 4 Viewer is already installed on the computer.
- 1. Insert the CD/DVD into the drive of your computer.
 - ↳ The content of the CD/DVD is shown in the directory.
- 2. Click on the file "Start Sidexis Viewer.exe".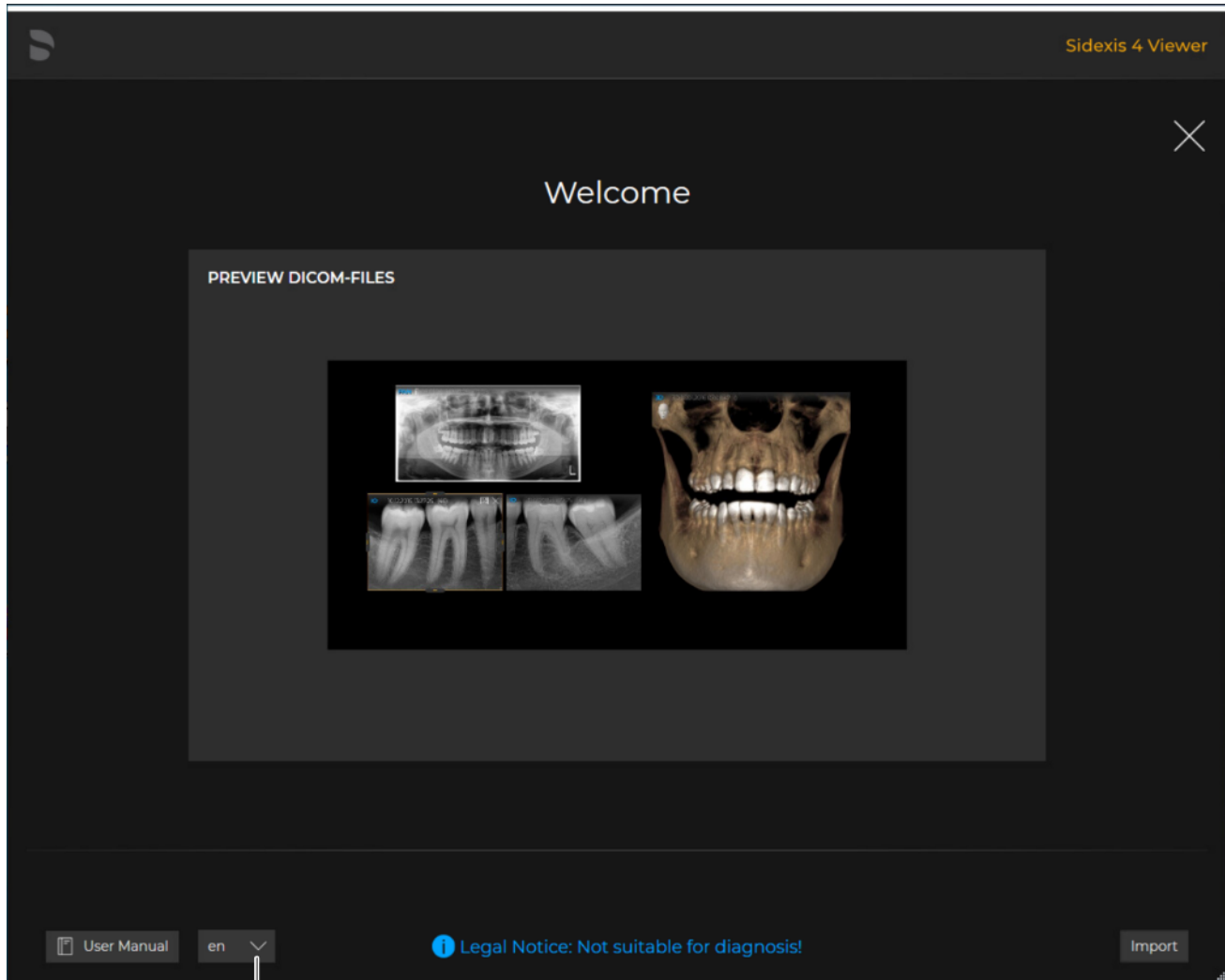

Import menu of the Sidexis 4 Viewer

- 3. If desired, you can change the language via the drop-down list (A).
- 4. Start the import of data by clicking on the "Import" button.

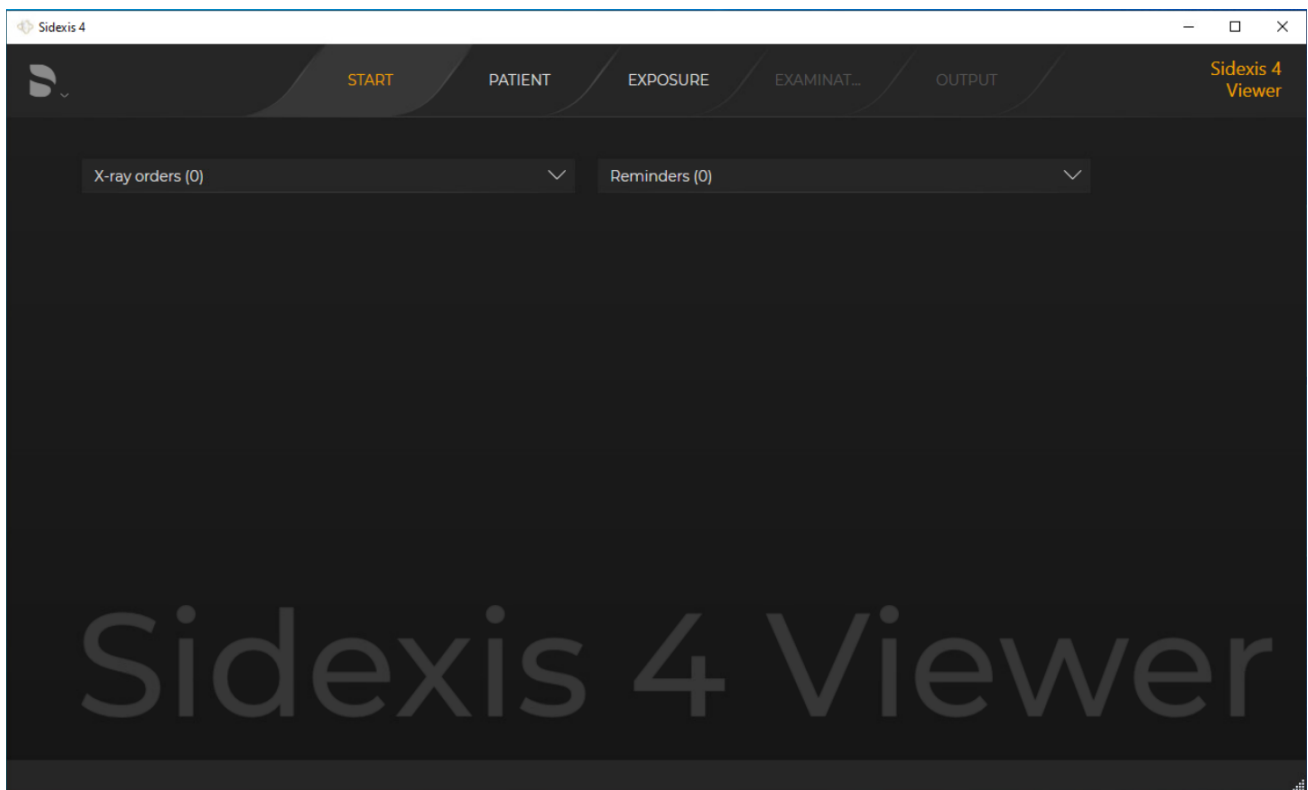Sidexis 4 Viewer

- The Sidexis 4 Viewer or Sidexis 4 client opens.
- After Sidexis 4 Viewer has opened, the import of the data records provided starts automatically.
IMPORTANT: This automatic import can take several minutes.

We reserve the right to make any alterations which may be required due to technical improvements.

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© Sirona Dental Systems GmbH
SIRONA Dental Systems GmbH