

Update Description

The service pack CEREC SW 4.0.5 updates existing CEREC SW 4.0.4 installations and only supports CEREC Omnicam. CEREC SW 4.0.5 does not support CEREC Bluecam and is not compatible to CEREC SW 4.0, 4.0.1, 4.0.2 and 4.0.3.

The service pack contains the following changes:

- **Reduction of response times**
The service pack optimizes the response times of the software especially in the Acquisition Phase and for the transition from Acquisition to Model Phase.
- **Interruption of scan when changing the picture catalogue**
In some cases the scanning process was interrupted after changing the picture catalogues. This will be fixed by this service pack.
- **“Wait”-cursor during the transition from “Acquisition to Model”**
The cursor is introduced to visualize the background calculations in connection with phase transition. The service pack also stops the Windows message indicating that the software does not react.
- **“Shredder” renamed to “Recycle Bin”**
The service pack renames the “Shredder” to “Recycle Bin”.
- **Flickering during camera calibration is fixed**
The service pack stops the flickering during the camera calibration process which occurred from time to time.
- **Picture catalogues which could not be deleted now fixed.**
In certain circumstances the pictures in the picture catalogues were not deleted by moving them to the “Recycle Bin”. The service pack fixes this issue.
- **Buccal registration after editing the model**
This service pack enables buccal registration after editing of the model.

An existing CEREC SW 4.0.4 installation is required for this service pack. We strongly recommend this update to all 4.0.4 Omnicam users.