

CEREC SW 5.3.2

Upgrade Description (EN)

Dentsply Sirona is pleased to announce the new update CEREC SW 5.3.2. The installation of the "CEREC SW 5.3.2" update does not require a previously installed version. The automatic update to "CEREC SW 5.3.2" is only offered if a CEREC SW 5.3.x is installed.

Changes made to previous version 5.3.1:

- With CEREC SW 5.3.2, bridge designs can be uploaded from the manufacturing phase to DS Core via "Export to DS Core".
- Bug fixes for operating system and graphics card driver support.
- Bug fixes for SW stability

The CEREC SW 5.3.2 installation routine includes the DI&CAD/CAM Connector v.2.0.2.

CEREC SW 5.3.2 is compatible with inLab CAM connected with DS Core.

Installation & License

- CEREC SW 5.3.2 is a full version and does not need any preinstalled CEREC SW versions.
- To upgrade from a CEREC SW 5.x.x to CEREC SW 5.3 the CEREC SW 5.3 version license is required.
- Download CEREC SW 5.3.2 from the Customer Support Portal or the CAD/CAM Downloads page on www.dentsplysirona.com
- Perform a data backup before starting the installation.
- Uninstall previous CEREC software versions.
- For installation, please unpack the downloaded CEREC SW 5.3.2 files and copy the CEREC SW 5.3.2 folder on your acquisition center. To start the installation, double-click the Setup.exe and follow the installation setup.
- The use of the correct graphic card driver is necessary for ensuring an optimal functionality of your CEREC Software 5.3.2. At the startup of your software, it will automatically check whether your driver is correct. Should you receive a notification, please contact customer service.

Hardware support note:

- Scans from the Primescan 2 can be opened and processed with CEREC SW 5.3.2. This can be done on a CAD/CAM workstation as well as on a Primescan AC.
- CEREC SW 5.3.2 supports Omnicam and Primescan scanners (AC models with touchscreen and Windows 10).
- CEREC SW 5.3.2 supports pre-2019 Omnicam units (without touchscreen with performance upgrade)



All CEREC SW downloads can be found on:

[CAD/CAM downloads: Find here! | Dentsply Sirona Global](#)