



Connect Software 5.2.9

Update Description (EN)

Dentsply Sirona is pleased to announce the new Connect SW 5.2.9 update. The Software will be available from February 13th, 2025 for all Connect 5.x users.


Changes made to previous version 5.2.8:

General bugfixes:

- Include margin line in automatic DXD export
- General bugfixes for integrated workflow with the DS Core cloud platform
- Support operating system and graphic card drivers

Installation notes:

Automatic update: The update notification for Connect SW 5.2 will pop-up automatically. Start the update process directly by pressing "Download" or start the process later. If you choose to start the update process later, you must go to the Windows taskbar.

- Open the Dentsply Sirona AutoUpdate Center by clicking on the corresponding icon: 
- The Connect SW 5.2.9 is listed in the update list of the Dentsply Sirona AutoUpdate Center. After the download is complete, the installation of Connect SW 5.2.9 can be started.
- Perform a backup of all patient data before starting the installation process.
- Do not perform an installation on more than one acquisition unit or computer at the same time if these are within the same network.
- Before starting the installation process, please check the **patient database** directory in the software.
- (Configuration/Settings/Patient Database). The path should always point to a directory, never to the root of a drive, i. e. never to "D:\", but rather to "D:\Data"
- The use of the correct graphic card driver is necessary for ensuring an optimal functionality of your Connect Software 5.2.7. At the startup of your software, it will automatically check whether your driver is correct. Should you receive a notification, please contact customer service.

Hardware support note:

Connect SW 5.2.9 supports Omnicam (with guided scanning) and Primescan (no guided scan required) scanners but requires the newer AC models with touchscreen and Windows 10 to run. Pre-2019 Omnicam units (without touchscreen) will require a Windows 10 update in order to use this software. For more information on upgrading these devices, please contact your dealer representative.

All Connect SW downloads can be found here:

<https://www.dentsplysirona.com/en/explore/cerec/downloads.html>

<https://www.dentsplysirona.com/en/explore/digital-impression/downloads.html>

<https://www.dentsplysirona.com/en/discover/discover-by-brand/connect-software.html>

<https://www.dentsplysirona.com/en/explore/3d-printing/primeprint/software-downloads.html>

<https://my.cerec.com/en/products/connect-sw-5-2.html>