

ENDO ACTIVATOR[®]

Safe • Effective • Simple

**AN EASY TO USE & SAFE SOLUTION
TO ACTIVATE YOUR IRRIGATION**



15/02 # 25/04 # 35/04



Distributed by:

DENTSPLY
MAILLEFER

ENDO ACTIVATOR[®]

Safe • Effective • Simple

PRODUCT FEATURES

EndoActivator[®] Tips

- Easy snap-on/snap-off design
- Strong, flexible, medical-grade polymer composition
- Color-coded by size
- 22 mm lengths with depth gauge rings at 18, 19 and 20 mm

Handpiece/Driver

- Contra-angled design for easy access to posterior teeth
- 3-speed sonic motor provides options of 2.000, 6.000 and 10.000 cpm
- Ergonomic, cordless, and battery-operated handpiece



U.S. and International Patents Pending

TECHNIQUE SEQUENCE*

- Prepare canal to produce a fully tapered shape
- Fill pulp chamber with NaOCl, EDTA, or other final rinse solution
- Select the Activator that manually fits loosely within 2 mm of working length
- Place the barrier sleeve over the Driver to protect the entire handpiece
- Attach the Activator over the barrier-protected Driver
- Place the attached Activator tip into the prepared root canal
- Depress the on/off switch to activate solution
(Note: Switch defaults to 10.000 cpm, as indicated for cleaning procedures)
- Use a pumping action to move the Activator in short, 2-3 mm vertical strokes
- Hydrodynamically agitate the intracanal solution for 30-60 seconds
- Irrigate, then use intracanal suction to eliminate loose debris
- Repeat the above steps for each intracanal irrigant used

*Sequence for cleaning root canal systems.
For other applications, see Directions For Use.



Endo Inventions, LLC
122 S. Patterson Avenue, Ste. 206
Santa Barbara, CA 93111 USA



EMERGO EUROPE
Prinsessegracht 20, 2514 AP,
The Hague, The Netherlands
Phone: +31.70.345.8570
Fax: +31.70.346.7299

Distributed by:

DENTSPLY
MAILLEFER

CE
2797

Dentsply Maillefer • CH-1338 Ballaigues • Switzerland • www.dentsplymaillefer.com

SPR/F19 02111.E/02 / 2010 - updated 01/2021